EN

Clamp version for the Life & Mobility products (will be clamped around the oval frame tubing)

Fitting the docking system (Fig. 1.0)

Position the clamping jaw (1 & 2) around the lower frame tube (9) from the outside and the bar (4) from the inside and screw both parts together with the screws (5). Make this attachment on both sides in the same way.

Position the two clamps (B-C) exactly opposite each other (Fig.2.0) in such a way that both clamps are at the same height.


To do this use a reference point from which you can measure (e.g. from the axle plate of the wheelchair to the clamp). If necessary undo the screws (5) a little and move both clamps (B-C) so that they are at the same level. Then re-tighten the screws (5) (Torque setting: 3 Nm).

Mounting the control unit (Fig. 3.0)

Take the complete control unit (T) through the rear zip fastening of the back pack. Now undo the screws (1.4) – position the gripper clamp (1.2) from the outside and the beam clamp (1.1) from inside around the backrest tube. (Torque setting: 3 Nm).

Please note: The above list is an indication of wheelchair models Roxx & Canto the R20 can be mounted on. However year of build, frame shape, back accessories, connecting bars between the side frame, type of push handles, special settings, frame accessories are of influence. Therefore it is essential to try and test it to be 100% sure! Or ask your supplier for advice.

Attention: In case of a reclining or angle adjustable back the backpack weight of the R20 (~2kg) may affect the stability and could cause a tipping backward. Therefore we strongly recommend to use anti tips.

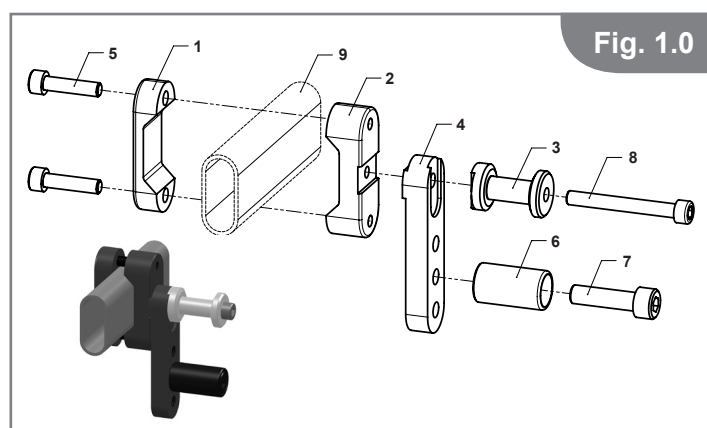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