



Totstander

User Manual



Supporting
Children



SECTION 01: INTENDED USE

The Leckey Totstander has been designed for used at home or in school for upright standing. The stander is available in one sizes catering for the needs of children with disabilities from the age of 1-5. The maximum user weight is 22kg.

SECTION 02: DECLARATION OF CONFORMITY

James Leckey Design Ltd as manufacturer with sole responsibility declares that the Leckey Totstander conforms to the requirements of the MDR EU 2017/745 Guidelines and EN12182 Technical aids for disabled persons, general requirements and test methods.

SECTION 03: TERMS OF WARRANTY

The warranty applies only when the product is used according to the specified conditions and for the intended purposes, following all manufacturers' recommendations (also see general terms of sales, delivery and payment). A three year warranty is provided on all Leckey manufactured products and components.

SECTION 04: PRODUCT HISTORY RECORD

Your Leckey product is classified as a Class 1 Medical device and as such should only be prescribed, set up or reissued for use by a technically competent person who has been trained in the use of this product. Leckey recommend that a written record is maintained to provide details of all setups, reissue inspections and annual inspections of this product.

SECTION 05: PRODUCT TRAINING RECORD

(Parents, Teachers & Carers)

Your Leckey product is a prescribed Class 1 Medical Device and as such Leckey recommend that parents, teachers and carers using the equipment should be made aware of the following sections of this user manual by a technically competent person. Leckey recommend that a written record is maintained of all those who have been trained in the correct use of this product.

SECTION 06: SAFETY INFORMATION

1. Always read instructions fully before use.
2. Users should not be left unattended at any time whilst using Leckey equipment.
3. Only use Leckey approve components with your product. Never modify the product in any way. Failure to follow instructions may put the user or carer at risk and will invalidate the warranty on the product.
4. If in any doubt to the continued safe use of your product or if any parts should fail, please cease using the product and contact our customer services department or local dealer as soon as possible.
5. We would recommend that users are never transferred straight from seating to standing without preparation and stretching. This is best done whilst lying on a mat but can be carried out in a wheelchair or seating system if required.
6. Carry out all positional adjustments and ensure that they are securely fastened before you put the user into this product.
7. When placing the user into a standing frame, for safety reasons, always secure the pelvic band first. The chest and knee straps should then be fastened.
8. When the product is stationary ensure that all castors are locked and facing away from the base as this will improve product stability.
9. Never adjust the overall height of the Totstander while the frame is in use. Other finer positional adjustments may be carried out safely when the user is in the frame.
10. Never leave the product on a sloping surface, greater than 5 degrees.
11. The product contains components which could present a choking hazard to small children. Always check that locking knobs and bolts within the child's reach are tightened and secure at all times.
12. Leckey products comply with fire safety regulations in accordance with EN12182. However the product contains plastic components and therefore should be kept away from all direct sources of heat including

naked flames, cigarettes, electric and gas heaters.

13. Clean the product regularly. Do not use abrasive cleaners. Carry out maintenance checks on a regular basis to ensure your product is in good working condition.
14. The product is designed for indoor use and when not in use should be stored in a dry place that is not subjected to extremes of temperature. The safe operating temperature range of the product is +5° to +40°.

SECTION 07: HOW TO UNPACK AND ASSEMBLE

Check parts

Congratulations on purchasing your Lecky Totstander. When you receive your Totstander it will be boxed and wrapped in one parcel. Your Totstander comes as standard with chest support, knee plate, knee cups, pelvic strap, wooden base, inner poles and sandals.

Carefully remove all the parcels from the box and check you have all the parts you have ordered.



Keep polythene bags away from children

SECTION 08: PRODUCT CARE

Clinical Set-up

Adjusting the height

To adjust the height of the Totstander, loosen the lowest knobs **A** on the upright bars, adjust to the desired height and retighten.

Adjusting the height of the knee cups. Loosen the knobs **B** (see photo below), set knee cups to desired and retighten knobs.



Attaching the Pelvic Band

The pelvic belt should be attached to the user first, again if practical on a mat. The pelvic straps are attached around the user's pelvis the longer straps are then slipped through the outer poles of the frame and secured firmly behind the user with the Velcro on the belt. To ensure that you can easily identify the strap that goes around the user and the one that attaches to the frame, a Leckey label is sewn on to the one that goes around the frame.



SECTION 9: TECHNICAL INFORMATION

Product Code **LFS/T-MW**

The Totstander comes with Meadow Fabric as the sole option.

Totstander		
Chest Band Height		
min	470	18.5
max	680	27
Base		
min	450	18
max	450	18

For more information please visit **leckey.com**



Leckey,
19c Ballinderry Road
Lisburn, BT28 2SA
Northern Ireland

[+44] 28 9260 0750
hello@leckey.com

leckey.com



LS919-01

