# Elegance

Directions for use  
Gebrauchsanweisung  
Notice D'Utilisation  
Manuale Per l'Uso  
Manual De Usuario

# BREEZY

INNOVATE.  
BUILD FOR THE FUTURE

## Foreword (1.1.1)

Dear Customer,

We are very happy that you have decided in favour of a high-quality product from SUNRISE MEDICAL. This user's manual will provide numerous tips and ideas so that your new wheelchair can become a trustworthy and reliable partner in your life.

Maintaining **close links** with our customers is of great importance to us at Sunrise Medical: we therefore like to keep you up-to-date with the latest developments at Sunrise Medical. Keeping close to our customers also means fast service when you need replacement parts or accessories, or just have a question about your wheelchair – and with as little red tape as possible.

We want you to be satisfied with our products and service. Sunrise Medical therefore constantly works to continuously develop its products. For this reason, changes can occur in our range of products with regard to shape, technology and fittings. Consequently, no claims can be construed from the data or pictures contained in this user's manual.

SUNRISE MEDICAL has been awarded the ISO- 9001 Certificate, which affirms the quality of our products at every stage, from R & D to production.

Please contact your local, authorised SUNRISE MEDICAL dealer if you have any questions concerning the use, maintenance, or safety of your wheelchair.

In the case that there is no authorised dealer in your area or you have any questions, you can contact Sunrise Medical either in writing or by telephone at:

**Sunrise Medical GmbH**  
**High Street Wollaston**  
**West Midlands DY8 4PS**  
**England**

Phone +44 (0) 1384 446622

Fax +44 (0) 1384 446644

[www.sunrisemedical.com](http://www.sunrisemedical.com)

## Table of Contents

Foreword	3
Safety Tips	4
Wheelchair components	5
Handling	6
Options	
Step Tubes	6
Brakes	6
Footplate	7
Castors	7
Axle Plates	7
Backrest	7
Armrest	8
Lap belt instructions	8
Anti-Tip Tubes	9
Seat	9
Crutch Holder	9
Travel Wheels	9
Tyres and Mounting	9
Possible Problems	9
Maintenance and Care	10
Technical Data	10
Nameplates / Guarantee	10
Torque	10

This manual gives information on all features sold across different countries, without stating whether they are actually available in your country or are optional or standard features. For this information please refer to the orderform/prescription form or your prescriber/supplier.

## Safety Tips (3.1.2)

The engineering and construction of this wheelchair have been designed to provide maximum safety. International safety standards currently in force have either been fulfilled or exceeded. Nevertheless, users may put themselves at risk by improperly using their wheelchairs. For your own safety, the following rules must absolutely be observed. Unprofessional or erroneous adaptation or adjustment work could increase the risk of accidents. As a wheelchair user, you are also part of the daily traffic on streets and sidewalks, just like anyone else. We would like to remind you that you are thus also subject to any and all traffic laws. For this reason, you should always wear light clothing in the dark as much as possible so that you can be seen more easily and make sure that the reflectors are not covered. Be careful during your first ride in this wheelchair. Get to know your wheelchair. Test what effect changing the center of gravity would have on your wheelchair. Practice using your wheelchair on level ground as well as going uphill and downhill.

### CAUTION:

The wheel locks are not intended to brake your wheelchair. They are only there to ensure that your wheelchair does not begin rolling unintentionally. When you stop on uneven ground, you should always use the wheel locks to prevent such rolling. Always set both wheel locks; otherwise, your wheelchair could tip over.

### The wheel locks have not been designed to be used as brakes for a moving wheelchair.

Always make use of elevators and ramps. If these are not available, you can reach your destination with the aid of two attendants.

**They should grip the wheelchair at securely mounted parts.** If your wheelchair is equipped with anti-tip tubes, these should be folded away. A wheelchair should never be lifted with an occupant; it should only be pushed.

Before each use, the following should be checked:

- Quick-release axles on the rear wheels
- Velcro on seats and backrests
- Tyres, tyre pressure, and wheel locks.

### Wheel locks will not function properly if tyre pressure is too low or the interval between tyres and wheel locks is too large.

### The maximum user weight is 125 kilograms (20 stone).

This wheelchair should be used exclusively to convey **one person** at a time in the seat. Any other use beyond this limit is in violation of safety regulations.

### Users should not sit in the wheelchair while riding in any and all means of transportation.

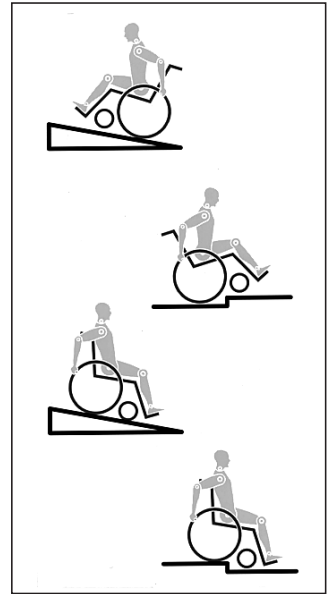
Only permanently installed seats and seat belts will offer sufficient protection in hazardous situations. When vehicles are in motion, unoccupied wheelchairs should be secured using appropriate means. Before changing any of the adjustments of this wheelchair, it is important to read the corresponding section of the user manual. It is possible that potholes or uneven ground could cause this wheelchair to tip over, especially when riding uphill or downhill. When riding over a step or up an incline frontally, the body should be leaning forward.

## Transportability of wheelchairs and users in vehicles

A wheelchair secured in a vehicle will not provide the equivalent level of safety and security of a vehicle seating system. It is always recommended that the user transfers to the vehicle seating. It is recognised that this is not always practical for the user to be transferred and in these circumstances where the user must be transported whilst in the wheelchair the following advice should be followed.

1. Confirm that the vehicle is suitably equipped to transport a passenger in a wheelchair and has access that is suitable for your wheelchair type.
2. Sunrise Medical Ltd recommends:
  - a) That the wheelchair be secured by a 4-part (webbing) Tie Down Restraint system, conforming to ISO 10542. These restraints generally comprise of 4 individual straps that are attached to each corner of the wheelchair
  - b) That the occupant should be restrained independently of the wheelchair by a lap and diagonal safety belt, conforming to ISO 10542. This is similar to a car seat belt that comes across the user's shoulder and around the waist.

- c) Any part of the wheelchair accessories such as kerb climbers that can easily be detached should be removed and stored in the vehicle luggage compartment during transportation.
- d) The wheelchair should always be transported in the forward facing direction.
- e) Sufficient space should be available around the wheelchair to enable clear access to attach, tighten and release the wheelchair and occupant tie down restraints and safety belts.
- f) User should be provided with a headrest that is suitable for use with the wheelchair in transportation.
- g) The tie-down restraints should be fitted to the main frame of the wheelchair as indicated in the diagram on the following page, and not to any attachments or accessories, e.g. not around the spokes of wheels, brakes or footrests.
- h) The tie-down restraints should be attached as close as possible at an angle of 45 degrees and tightened securely in accordance with the manufacturer's instructions.
- i) Pelvic restraint, posture belts or lap belts supplied with the wheelchair should always be used in addition to, but never as a substitute for the approved passenger restraint belts.
- j) The safety of the user during transportation depends upon the diligence of the person securing the tie-down restraints and they should have received appropriate instructions and/or training in their use.

The wheelchair should be restrained with 4-point tie down webbing straps, with non-adjustable front straps and adjustable rear straps, which typically use Karabiner clips/S hooks and tongue and buckle attachments.

The attachment points to the chair are the inner front side frame just above the castor and the rear side frame. The straps are fitted around the side frames at the intersection of the horizontal and vertical frame tubes. (See illustration on following page)

The tie down symbol on the wheelchair frame indicates the position of the wheelchair restraint straps. The straps are then tensioned after the front straps have been fitted to secure the wheelchair.

The wheelchair must be facing forward and the manual brakes firmly applied. The wheelchair occupant should be secured using a 3-point double inertia reel belt.

It is recommended that a lap/pelvic belt similar to the example shown previously in the section headed seat belts, be used in conjunction with the vehicle occupant restraints.

### The seat/pelvic belt should never be used in place of the occupant lap and diagonal belt when travelling in a vehicle.

## Transportability – positioning of wheelchair tie down restraints on wheelchair

Position of the front wheelchair tie down restraint and the tie down label. The position is the same for both the Breezy Elegance self-propelling and the attendant push wheelchairs.

Position of the rear wheelchair tie down restraint and the tie down label on the self-propelling wheelchair.

Position of the rear wheelchair tie down restraint and the tie down label on the attendant push wheelchair.

## Wheelchair components (4.1.1)

We at SUNRISE MEDICAL have been awarded the ISO 9001 Certificate, which affirms the quality of our products at every stage, from R & D to production. This product meets relevant National and International Standards to satisfy compliance with EU directives. Optional equipment and accessories are available at extra charge.

### Wheelchairs:

1. Push handles
2. Backrest upholstery
3. Sideguards
4. Seat sling
5. Footrest
6. Castors
7. Quick-release axles
8. Wheel locks
9. Handrim
10. Rear wheel

**Handling**

**Folding up (6.1)**

First remove the seat cushion from the wheelchair and flip up the (platform) footplate or the individual footplates. Take hold of the sling from underneath in the middle and pull upwards. The wheelchair will then fold up. To fold your wheelchair until it is as small as possible, e.g., to put into a car, you can remove the footrests (depending on the model). For this purpose, open the latch from the outside and swing the footrest to the side. Then the footrest only has to be drawn out of the frame tubing.

02.1

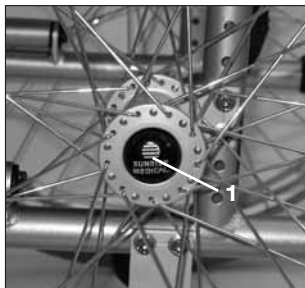
02.2

**Unfolding**

Press down on the seat tubing (see picture). Your wheelchair is then unfolded. Now snap the seat tubing into position in the seat saddle. This can be facilitated by slightly tipping your wheelchair. In this way, the weight can be taken off of one rear wheel so that the wheelchair can be more easily unfolded. Be careful not to get your fingers caught in the cross-tube assembly.

**Quick-Release Axles for Rear Wheels (6.2)**

The rear wheels are equipped with quick-release axles. The wheels can thus be installed or removed without using tools. To remove a wheel, simply depress the quick-release button on the axle (1) and pull it out.

400

**CAUTION:** Hold the quick-release button on the axle depressed when inserting the axle into the frame to mount the rear wheels. Release the button to lock the wheel in place. The quick-release button should snap back to its original position.

**Options – Step Tubes**

**Step Tubes (7.1)**

Step tubes are used by attendants to tip a wheelchair over an obstacle. Simply step on the tube to push a wheelchair, for example, over a kerb step.

401

**Options – Brakes**

**Wheel Locks (7.10)**

Your wheelchair is equipped with two wheel locks. They are applied directly against the tyres. To engage, press both wheel-lock levers forward against the stops (Fig. 1 and 2). To release the wheel, pull the levers back to their original positions.

402.1

1

Braking power will decrease with:

- Worn tyre tread
- Tyre pressure that is too low
- Wet tyres
- Improperly adjusted wheel locks.

The wheel locks have not been designed to be used as brakes for a moving wheelchair. The wheel locks should therefore never be used to brake a moving wheelchair. Always use the hand-rims for braking. Make sure that the interval between the tyre and wheel locks complies with given specifications (Fig. 1 and 2). To readjust, loosen screw (1) and set the appropriate interval. Tighten screw (see the page on torque)

402.2

2

**CAUTION:**

After each adjustment of the rear wheels, check the interval to the wheel locks and readjust if necessary.

**Extension for the wheel lock Lever (7.11)**

The longer lever helps to minimise the effort needed to set the wheel locks.

403

**CAUTION:**

Mounting the wheel lock too close toward the wheel will result in a higher effort to operate. This might cause the wheel lock extension lever to break!

Leaning onto the wheel lock extension lever while transferring will cause the lever to break! Splashing water from tyres might cause the wheel lock to malfunction.

**Drum Brakes (7.13)**

Drum brakes permit safe and convenient braking for an attendant. They can also be set with the aid of a locking lever (1) to prevent rolling. This lever must snap audibly into place. Drum brakes are not dependent on the aid of air pressure inside the tires. Your wheelchair cannot be moved when the drum brakes are set.

404

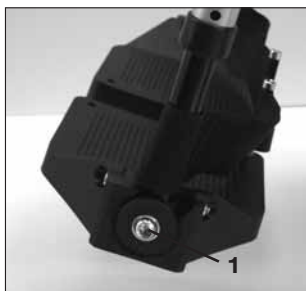
**CAUTION:**

Drum brakes should only be adjusted by authorized dealers.

**Options – Footplate**

**Footrest (7.21.3)**


The footrests can be flipped up to make it easier to get into and out of the chair. By loosening the screw (1) the angle can be adjusted in the area of ± 15°. Then re-tighten the screw (see Torque).

405

**Lower leg length**

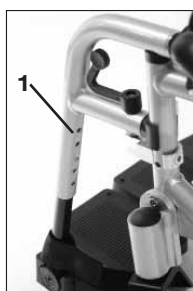
By loosening the screw (1) the lower leg length can be adjusted. Loosen the screws and adjust the tube with the footrest to the desired position. Then re-tighten the screw (see page on Torque). Make sure that the gap to floor is not below the minimum of 2.5 cm.

406 a

**Legrests and Latch (7.23.1)**

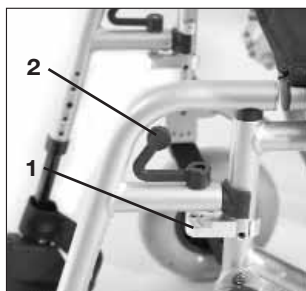The legrest can be swung in and outwards. Fit your legrest by inserting it with the footplate(s) pointing toward the outside. Then rotate it toward the inside until it locks into place (1). To remove the legrest, depress the lever (2), swing the footrest towards the outside and lift to remove it. Make sure that the legrest is properly locked into place.

407 a


**CAUTION:**

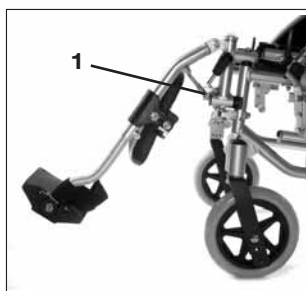The legrests should not be used to lift or to carry the wheelchair.

**Elevating legrest (7.24.1)**

**To elevate:**

Lift your lower leg and press the release button. Pull the legrest upwards and adjust it to the desired height. As soon as you release the button, the legrest will click back into place.

416.3

**To lower:**

Lift your lower leg and press the release button to unlock the legrest to lower it (1). As soon as you release the button, the legrest will click back into place.

**CAUTION:**

The elevating legrest is not allowed to be adjusted into the lowest position to avoid any interference with the castors.

**CAUTION:**

Keep hands clear of the adjustment mechanism between the frame and the movable parts of the legrest while elevating or lowering the legrest.

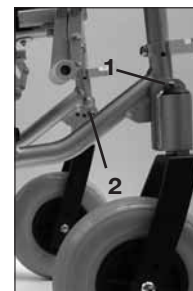
**CAUTION:**

Legrests are not to be used for lifting or carrying the wheelchair with an occupant.

**Options – Castor**

**Adjustment of seat height and seat angle (7.51.1.3)**

By loosening the screws (1, 2) you can adjust the appropriate chair angle. Adjust the castor so that, as far as possible, the axle is at 90° to the floor. After adjustment, all the screws must be re-tightened (see the page on Torque). The front seat height can also be adjusted by altering the castor in the castor fork (Seat heights 47, 50, 52 cm) and by altering the castor connector (Seat height 42, 45 cm). For detailed adjustability possibilities, please see figure 1 and figure 2 on page 10.

408

**Options – Axle Plates**

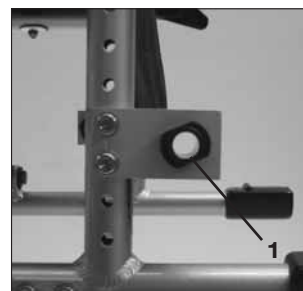
**Horizontal Axle Position (7.51.1.2)**

The axle plate can be positioned forward for maximum maneuverability or rearward for increased stability.

**CAUTION:**

Wheel locks must be adjusted to suit the new position.

The longer the wheel base, the greater the stability of the wheelchair. Turn the axle bracket to the rear (1) in order to extend the wheel base.

409

**CAUTION: Wheel locks must be adjusted to suit the new position.**

**Options – Backrest**

**Adjustable Backrest**

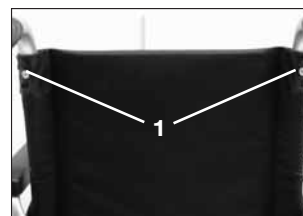
**Upholstery**

The sling can be adapted to any degree of tension by adjusting the Velcro fasteners. The padding in the backrest can be removed through an interior opening, allowing you to cushion the backrest according to your individual needs.

**Adjustable back height (7.77.3)**

The backrest can be set to two different back heights (41-43 cm). Undo the screws (1) and adjust the back to the required height. Tighten the screws back up (see Torque).


410

**Half folding backrest (7.78)**

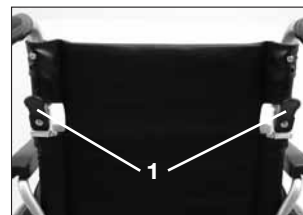For folding the back down press the latches (1).

**CAUTION:**

Make sure that the backrest locks in correctly when it is set upright again. When adjusting be careful not to get your fingers caught.

410.1

**Options – Armrest**

**Standard sideguard, flip-up, removable with short or long arm pads (7.81.3)**

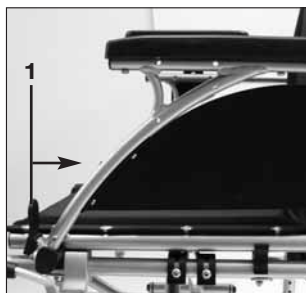
With the sideguard which is stepped down at the front it is possible to move up closer to a table. To flip them up press the lever (1) forwards, so that the sideguard is released.

**ATTENTION!**  
Sideguards, and the armrests are not designed to be used to lift or carry the wheelchair.


**Sideguard, flip-up, removable with short or long arm pads, height-adjustable**

The arm pad can be height-adjusted in the following way. Push the lever (1) and adjust the arm pad (2) to the desired height. Release the lever and push the arm pad (2) down until you hear it click into place.

**ATTENTION!**  
Sideguards, and the armrests are not designed to be used to lift or carry the wheelchair.



411

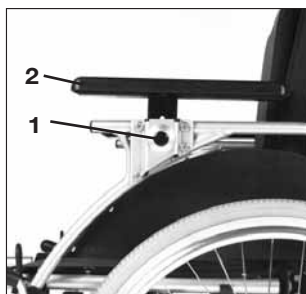411.2

**Step 5**

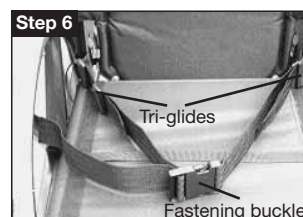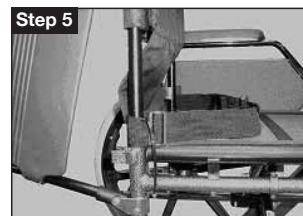To secure the positioning belt to the chair. Take one of the cable ties supplied and thread through the positioning belt strap. Then wrap around the frame and fasten together. **DO NOT** over tighten positioning belt. Repeat steps 3-5 with both ends of positioning belt.

**Step 6**

To adjust the positioning belt use the tri-glide buckles and the fastening buckle.

**Step 7**

The positioning belt should be adjusted so the fastening buckle is situated in the middle of the pelvis.

**Step 8**

The positioning belt should be adjusted so a hand can be placed between the user and the belt.

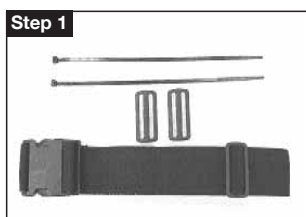
**Options – Lap belt instructions**

**Note:**

Lap belts are fitted as positional aids, and are not suitable as transportation restraints.


**Step 1**

Positioning belt kit. The Positioning belt kit contains, one positioning belt, three tri glide buckles and two cable ties.

**Step 2**

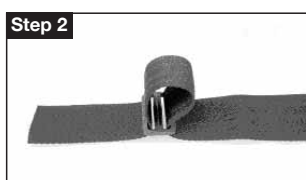To assemble the positioning belt. Take the positioning belt strap and thread through the tri-glide. Repeat on both ends of positioning belt.

**Step 3**

To fit the positioning belt to the chair. Take the positioning belt strap and wrap around back post.

**Step 4**

Thread the positioning belt back through the tri-glide to form a loop around the back post. Thread positioning strap through tri-glide as shown in step 5.

The positioning belt can be used on the Quickie RXS, Quickie 2, Breezy SLTL and Quickie 2 Millennium wheelchairs.

Quickie RXS

Quickie 2

Quickie 2 Millennium

Breezy SL/TL

**Advice to client**

The positioning belt must only be fitted by an approved Sunrise Medical dealer / agent. The positioning belt should only be adjusted by a professional, or a Sunrise Medical approved dealer / agent.

The positioning belt must be checked on a daily basis to ensure they are adjusted correctly (see step 7) and are free from any obstruction or adverse wear.

Sunrise Medical does not encourage the transportation of any person in a vehicle using this positioning belt as a method of restraint. Please see Sunrise Medical transit booklet for further advice on transportation.

**Maintenance:**

Check lap belt, and securing components, at regular intervals for any sign of frays, or damage. Replace if necessary.

**NOTE:**

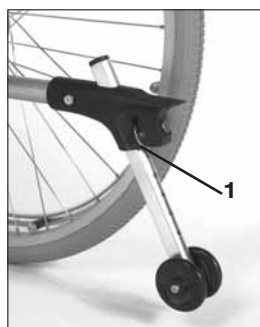
The lap belt should be adjusted to suit the end user as detailed above. Sunrise Medical recommend that the length and fit of the belt be checked on a regular basis to reduce the risk of the end user inadvertently re-adjusting the belt to an excessive length.

## Options – Anti-Tip Tubes

### Anti-tip tubes (7.93.1)

The anti-tip tubes provide additional safety for inexperienced users when they are still learning how to operate their wheelchairs. Anti-tip tubes prevent a wheelchair from tipping over backwards. By pulling the interlock (1), the anti-tip tube can be height adjusted. There should be a gap of 2,5 cm to the floor. To go up or down a step (e.g. kerb) the anti-tip tubes must be swung upwards, to prevent them from touching the ground.

413 a

## Options – Seat

### Standard seat sling (7.101.1)

The seat sling has Velcro® fastening on each side. This makes it possible to re-tension it easily. To re-tension the seat sling, fold the wheelchair slightly. Remove the screws (1).

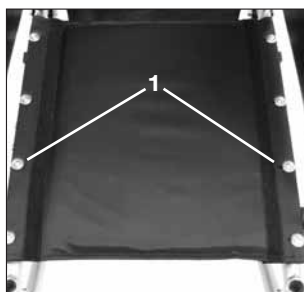
By opening the Velcro® fastening the seat sling can now be re-adjusted. To re-assemble, follow the procedure in reverse order. Make sure that after the adjustment work, all screws are re-tightened (see page on Torque).

#### ATTENTION!

**In order to ensure safety, at least 50% of the Velcro® surface must be in contact.**

### Seat depth (7.102.2)

By loosening the screws (1) the position of the seat sling can be changed and therefore the seat depth can be altered. Re-tighten the screws afterwards (see Torque).

414

## Options – Crutch Holder

### Crutch Holder (7.110)

This device permits crutches to be transported directly on a wheelchair. It has a Velcro loop (1) to fasten crutches or other aids.

#### CAUTION:

**Never try to use or even remove the crutches or other aids while moving.**

## Options – Travel Wheels

### Travel Wheels (7.113)

Travel wheels should be used wherever your wheelchair would be too wide if the rear wheels were used (e.g., in airplanes, buses, etc.). After the rear wheels have been removed with the aid of the quick-release axles, the transit wheels can immediately be used to continue riding. The transit wheels are mounted so that they are approx. 3 centimeters above the ground when not in use. They are thus out of the way when riding, transporting, or when tipping to pass over obstacles (e.g., kerbs, steps, etc.).

#### CAUTION:

**Your wheelchair does not have any wheel locks when the transit wheels are being used.**

## Tyres and Mounting

### Tyres and Mounting (8.1)

Always make sure you that you maintain the correct tyre pressure, as this can have an effect on wheelchair performance. If the tyre pressure is too low, rolling resistance will increase, requiring more effort to move the chair forward. Low tyre pressure also has a negative impact on maneuverability. If the tyre pressure is too high, the tyre could burst.

The correct pressure for a given tyre is printed on the surface of the tyre itself.

Tyres can be mounted the same way as an ordinary bicycle tyre. Before installing a new inner tube, you should always make sure that the base of the rim and the interior of the tyre are free of foreign objects. Check the pressure after mounting or repairing a tyre. It is critical to your safety and to the wheelchair's performance that regulation air pressure be maintained and that tyres be in good condition.

## Possible Problems

### Wheelchair pulls to one side (9.1.1)

- Check tyre pressure
- Check to make sure wheel turns easily (bearings, axle)
- Check to make sure both castors are making proper contact with the ground

### Castors begin to wobble (9.2.1)

- Check to make sure all bolts are secure; tighten if necessary (see the page on torque)
- Check to make sure both castors are making proper contact with the ground

### Wheelchair / Cross-tube assembly does not snap into position in the seat saddle (9.3)

- Chair is still new, i.e., the seat or backrest upholstery is still very stiff. This will improve with continued use.

### Wheelchair is difficult to fold up (9.5.1)

- Screws of the cross-tube brackets are too tight.

### Wheelchair squeaks and rattles (9.6)

- Check to make sure bolts are secure; tighten if necessary (see the page on torque)
- Apply small amount of lubrication to spots where movable parts come in contact with one another

### Wheelchair begins to wobble (9.7.1)

- Check tyre pressure.
- Check to see if rear wheels are adjusted differently.

**Maintenance and Care**

**Maintenance (9.9)**

- Check the tyre pressure every 4 weeks. Check all of the tyres for wear or damage.
- Check the brakes approximately every 4 weeks to make sure that they are working properly and easy to use.
- Change tyres as you would an ordinary bicycle tyre.
- All of the joints that are critical to using your wheelchair safely are self-locking nuts. Check every three months to make sure that all bolts are secure (See the page on torque). Self locking nuts should only be used once and should be replaced after single use.
- Use only mild household cleansers when your wheelchair is dirty. Use only soap and water when cleaning the seat upholstery.
- If your wheelchair should ever get wet, please dry it after use.
- A small amount of sewing-machine oil should be applied to quick-release axles approximately every 8 weeks. Depending on the frequency and type of use, we recommend taking your wheelchair to your authorised dealer every 6 months to have it inspected by trained personnel.

**CAUTION:**

**Sand and sea water (or salt in the winter) can damage the bearings of the front and rear wheels. Clean the wheelchair thoroughly after exposure to these conditions.**

**Technical Data (10.300)**

**Total width:**

- With standard wheels including handrims, narrow fitting: Seat width + 19,5 cm
- With drum brake wheels, handrims, narrow fitting: Seat width + 20 cm

**Folded dimensions:**

- With standard wheels: approx. 33 cm
- Without standard wheels: approx. 17 cm

**Weight in kg:**

- Wheelchair 16 kg
- Footrest (pair) 1.5 kg
- Standard sideguard 1.8 kg
- 24" rear wheel (solid) 1.4 kg

**Maximum load:**

Approved to loads up to 125 kg (20 stone)

**Nameplates / Guarantee**

**Rating plate (11.1.3)**

The rating plate is located on the cross-brace tube or cross-frame tube and also on the back of the user guide. This plate gives the exact type descriptions as well as other technical information. When ordering spare parts or making a complaint please give the following information

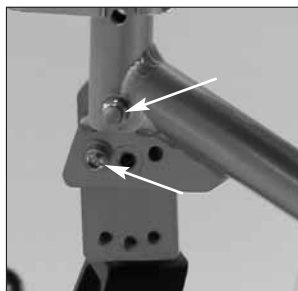
- Serial number
- Order number
- Month/Year

**Guarantee**


You have purchased a high-quality Breezy product. As a sign of our gratitude, we are providing you with a 5-year guarantee on all frame components and the cross-tube assembly as well as 2 years on all other parts. We take no responsibility for any damage resulting from inappropriate or unprofessional assembly and/or repairs, through neglect and wear and tear, as well as any alterations to wheelchair components by the user or by any third party. In such cases, our product responsibility is voided.

**Custom wheelchairs cannot be exchanged.**

**Torque (12.1.1)**

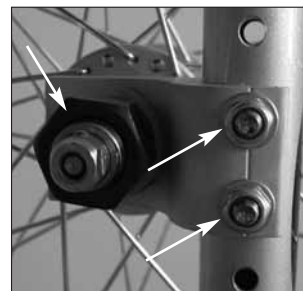415.1

415.2


The torque for the screws shown:

**10 Nm** castor and axle adapter screws must be secured with Loctite again after they have been loosened.

The torque for all screws is 7 Nm.


**Breezy Elegance settings**

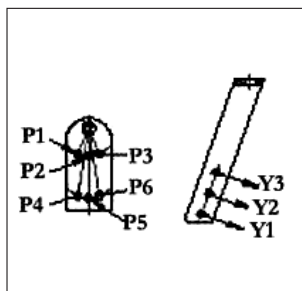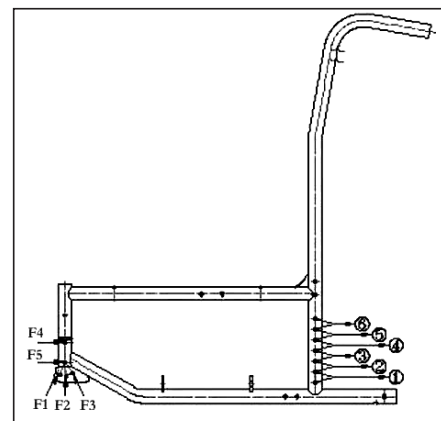Position of perforated plate on rear wheel	Position of caster fork	Adjustment of caster fork A	Adjustment of caster fork B	Front Seat Height	Seat Angle
5	Y3	P2	F4/F2	42cm	0°
6	Y3	P3	F4/F3	43cm	3°
4	Y3	P5	F5/F2	47cm	3°
4	Y2	P6	F5/F3	47cm	6°
3	Y2	P5	F5/F2	49cm	3°
1	Y1	P5	F5/F2	51cm	0°
2	Y1	P5	F5/F2	51cm	3°

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Sunrise Medical AB
Sunrise Medical B.V.
G2 A/S