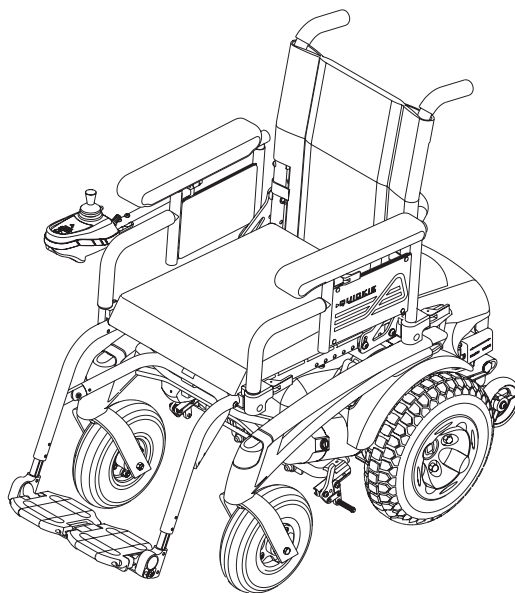
QUICKIE®

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I. INTRODUCTION

SUNRISE MEDICAL LISTENS

Thank you for choosing a Quickie/Sopur wheelchair. We want to hear your questions or comments about this manual, the safety and reliability of your chair and the service you receive from your Sunrise Medical supplier. Please feel free to write or call us at the address and telephone number below:

SUNRISE MEDICAL Ltd.
Sunrise Business Park
High Street
Wollaston
West Midlands DY8 4PS
ENGLAND
+44 (0) 1384 446688

Be sure to return your warranty card, and let us know if you change your address. This will allow us to keep you up to date with information about safety, new products and options to increase your use and enjoyment of this wheelchair. If you lose your warranty card, call or write and we will gladly send you a new one.

SUNRISE MEDICAL has been awarded the ISO 9001 Certificate, which affirms the quality of our products at every stage, from R & D to production.

This product meets all the requirements for radio interference in accordance with EC Guideline 89/336/EG and the Medical Device Directive 93/42/EEC.

The P-220 is designed for a maximum user weight of 140 kg for maximum gradients of 18% (10°) and for in- and outdoor use (Class B). This means the wheelchair is compact and manoeuvrable for some indoor environments and capable of negotiating some outdoor obstacles. European wheelchair standard EN 12184.

FOR ANSWERS TO YOUR QUESTIONS

Your authorised supplier knows your wheelchair best and can answer most of your questions about chair safety, use and maintenance. For future reference, fill in the following:

Supplier: _____

Address: _____

Telephone: _____

Serial number: _____


Date/Purchased: _____

II. TABLE OF CONTENTS

I. INTRODUCTION	2
II. TABLE OF CONTENTS	3
III. YOUR CHAIR AND ITS PARTS	5
IV. NOTICE– READ BEFORE USE	6
V. EMI (ELECTROMAGNETIC INTERFERENCE)	7
A. What Is EMI?	7
B. What Effect Can EMI Have?	7
C. Sources of EMI	7
D. Distance From the Source.....	8
E. Immunity Level	8
F. Report All Suspected EMI Incidents	8
G. EMI From Chair.....	8
VI. GENERAL WARNINGS	9
A. Notice to User.....	9
B. Notice to Attendants.....	9
C. Weight Limit.....	9
D. Controller Settings	10
E. EMI	10
F. Safety Check-List.....	10
G. Changes and Adjustments	10
H. When Seated in a Parked Wheelchair	10
I. Environmental Conditions.....	11
J. Terrain	11
K. Street Use	11
L. Motor Vehicle Safety	12
M. Centre of Balance	12
N. Transfers	12
O. Reaching or Leaning	13
P. Dressing or Changing Clothes	13
Q. Obstacles.....	13
R. Driving in Reverse.....	13
S. Ramps, Slopes and Sidehills	14
T. To Reduce the Risk of a Fall, Tip-Over or Loss of Control	14
U. Ramps at Home and Work	14
V. Wheelchair Lifts	15
W. Kerbs and Single Steps	15
X. Stairs.....	15
Y. Escalators.....	15
VII. WARNINGS: COMPONENTS AND OPTIONS	16
A. Anti-Tip Levers.....	16
B. Armrests.....	16
C. Batteries	16
D. Cushions and Sling Seats	16
E. Fasteners.....	16
F. Footrests	16
G. Free Wheel Mechanism.....	17
H. On/Off Switch	17
I. Pneumatic Tyres	17
J. Positioning Belts	17
K. Push Handles	17
L. Rear Wheel Locks.....	18
M. Seating Systems	18
N. Upholstery Fabric.....	18

VIII. TIPS FOR ATTENDANTS	19
A. To Climb a Curb or Single Step.....	19
B. To Descend a Curb or Single Step	19
IX. SET-UP, ADJUSTMENTS AND USE	20
Notes	20
Tools You Will Need	20
Check-Out	20
A. Power Drive Unit	21
B. Battery Removal	21
C. Folding Backrest.....	21
D. Swing-Away Footrests.....	21
E. Elevating Legrest.....	22
F. Joystick Installation.....	22
G. To Adjust the Height of the Remote Joystick	22
H. Remote Joystick Swing-Away Retractable Mount.....	22
I. Dual-Post Height-Adjustable Armrests	22
J. Single-Post Height-Adjustable Armrests.....	23
K. Seat Depth	23
L. 12" (300mm) Drive Wheel.....	23
M. Dynamic Stabiliser	23
N. Light Unit.....	24
MANUAL WHEELCHAIR CONVERSION KIT.....	24
A. Joystick	24
B. Power Drive Unit	24
C. Wheel Locks.....	24
D. 24" (600 mm) Rear Wheels	24
E. Anti-Tip Tubes.....	24
F. Installation of Axle Tube Assembly.....	25
G. Check-Out.....	25
X. OPERATING GUIDE	26
A. Performance Control Settings.....	26
B. P&G Pilot+/VSI Programming Pad.....	26
C. Thermal Roll-Back.....	26
D. Circuit Breakers	26
E. Joystick Assembly.....	27
F. Free Wheel Mechanism.....	29
XI. BATTERIES	30
A. Introduction	30
B. Battery Charger	30
C. Battery Acid.....	31
D. Connecting Batteries in Battery Compartment.....	31
E. Charging Batteries	32
F. Disposing of Batteries	32
XII. MAINTENANCE	33
A. Notes.....	33
B. Cleaning.....	33
C. Storage Tips.....	33
D. Battery Maintenance	33
E. Pneumatic Tyres	34
F. To Repair or Replace a Tyre	34
G. Ordering Parts	34
H. Maintenance Chart	35
XIII. TECHNICAL SPECIFICATIONS	36
XIV. WIRING DIAGRAM	37
XV. SETTINGS OF THE CONTROLLER	39
XVI. SUNRISE MEDICAL WARRANTY	40

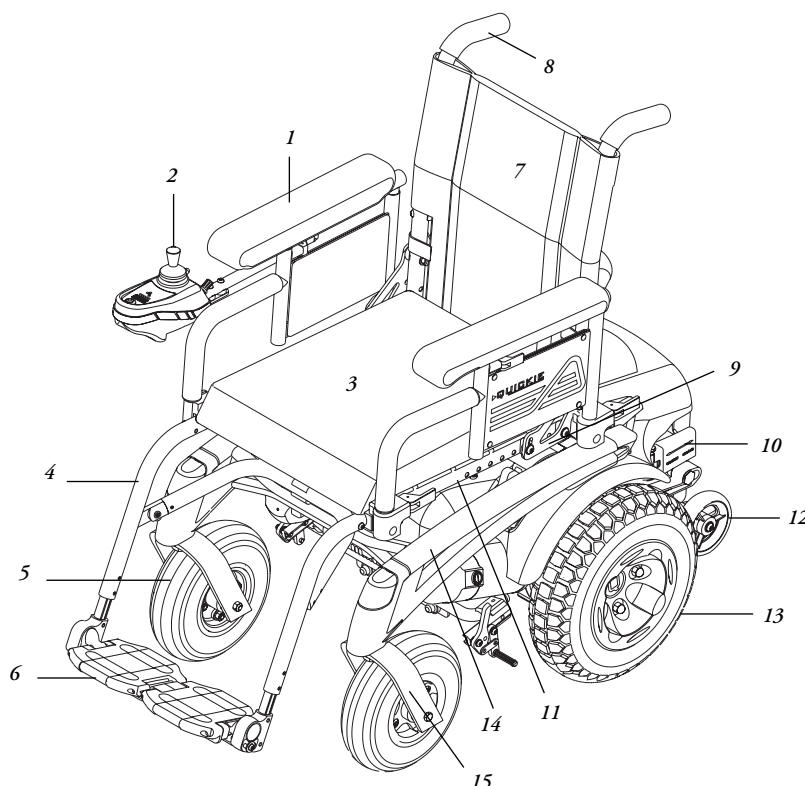III. YOUR CHAIR AND ITS PARTS

1. Armrest
2. Joystick
3. Seat Cushion (optional)
4. Fixed front hanger
5. Caster tyre
6. Two-piece, locking, flip-up footplate
7. Backrest
8. Push handle
9. Backrest hinge plate
10. Freewheel release lever
11. Seat frame
12. Dynamic stabiliser
13. Drive wheels
14. Base frame
15. Caster fork

QUICKIE/SOPUR P-220

Weight:

66 kg with swing-away footrests and armrest, without batteries

Drive Wheels

12" x 4" (300 x 100 mm) Aluminium

Tyre types: standard – pneumatic

Option – 12" x 2" (300 x 50 mm)

Option – airless insert

Joystick

Standard – integral joystick (right-hand or left-hand mount)

Option – Pilot+ control (right-hand or left-hand mount)

Option – swing-away retractable, Omni+ USCM (Universal Specialty Control Module)

Batteries

(2 deep-cycle batteries required to operate chair)

Standard – 12 V/74 Ah

Battery Charger

Standard – 24 V/8 A primary pulse

Colours

Standard – blue, black, red, midnight purple, pearl pink, forest green, burgundy, blue green, yellow, candy teal, candy purple, toxic green, black opal, blue velvet. Custom splatter colours – purple chase.

Seat Frame dimensions

Frame width: standard – 16", 18" (406, 457 mm)

Option – 14", 15", 17", 19", 20" (356, 381, 432, 482, 508 mm)

Seat depth – 14" – 20" (356 – 508 mm)

Seat height: standard – 18" (458 mm), option – 19" (480 mm), 20" (508 mm)

Seat sling: standard – parapack

Option – aluminium seat pan

Option – cushion: 2" (50 mm)

Backrest Folding

Standard – 15.5" (393 mm)

Option – 17.5", 19.5" (445 mm, 495 mm)

Caster Fork

Standard – fixed

Option – Spring suspension (8" (200 mm) tyre only)

Footrest

Standard – swing-away with composite footplates and heel loops, or two-piece locking.

Option – fixed tapered (80°) or fixed straight (70°), elevating legrests, angle-adjustable footplate, extended, toe loops, extension tubes, solid one-piece, two-piece locking

Casters

Standard – 8" (200 mm) pneumatic

Option – 8" (200 mm) airless insert, 8" (200 mm) suspension, 9" (225 mm) pneumatic, 9" (225 mm) airless insert

Armrests

Standard – dual-post, upholstered with standard or full-length pad

Option – dual-post height-adjustable, upholstered with standard or full-length pad

Manual Wheelchair Conversion Kit**Conversion Kit Axles**

Option – quad-release axle nuts

Rear Wheels

Standard – Aluminium

Tyre types: pneumatic, airless insert, full-profile polyurethane, low-profile polyurethane, Kevlar, high-pressure clincher

Handrim

Standard – aluminium

Option – plastic-coated, long tabs vertical or oblique projections

Wheel Locks

Standard – push-to-lock

Option – pull-to-lock 6" or 9" (150 mm or 230 mm) extension handles

All features may not be available with some chair set-ups or in conjunction with another chair feature. Please consult your supplier for more information. Your authorised supplier can also provide you with more information on accessories.

The Quickie/Sopur P-220 is a class B wheelchair as defined in EN 12184.

Department of transport class 3 vehicle.

This chair has not been crash tested.

IV. NOTICE – READ BEFORE USE

A. CHOOSE THE RIGHT CHAIR AND SAFETY OPTIONS

Sunrise Medical provides a choice of many power wheelchair styles, sizes and adjustments to meet the needs of the user. However, final selection of a wheelchair rests solely with you and your health care professional. Choosing the best chair and set-up for your safety depends on such things as:

1. Your size, disability, strength, balance and coordination.
2. Your intended use, and your level of activity.
3. The types of hazards you must overcome in daily use (in areas where you are likely to use your chair).
4. The need for options for your safety and comfort (such as positioning belts or special seat systems).

B. ADJUST CHAIR TO YOUR ABILITY

You need to work with your doctor, nurse or therapist, and your supplier, to fit this chair and adjust (program) the controller settings for your level of function and ability.

C. REVIEW THIS MANUAL OFTEN

Before using this chair you, and each person who may assist you, should read this entire manual and make sure to follow all instructions. Review the warnings often, until they are second nature to you.

D. WARNINGS

The word “WARNING” refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons. The “Warnings” are in four main sections, as follows:

1. V – EMI

Here you will learn about electromagnetic interference and how it can affect your chair.

2. VI – GENERAL WARNINGS

Here you will find a safety checklist and a summary of risks you need to be aware of before you use this chair.

3. VII – WARNINGS – COMPONENTS AND OPTIONS

Here you will learn about your chair. Consult your supplier and your health care professional to help you choose the best set-up and options for your safety.

4. XI – BATTERIES

Here you will learn about battery and charger safety, and how to avoid injury.

NOTE– Where they apply, you will also find “Warnings” in other sections of this manual.

V. EMI (ELECTROMAGNETIC INTERFERENCE)

Heed all warnings to reduce the risk of unintended brake release or chair movement:

1. Beware of the danger from hand-held transceivers. Never turn on or use a hand-held transceiver while power to your chair is on. Use extra care if you believe that such a device may be in use near your chair.
2. Be aware of nearby radio or TV stations, and avoid coming close to them.
3. If unintended movement occurs, turn your chair off as soon as it is safe to do so.

A. WHAT IS EMI?

1. EMI means: electromagnetic (EM) interference (I). EMI comes from radio wave sources such as radio transmitters and transceivers. (A “transceiver” is a device that both sends and receives radio wave signals).
2. There are a number of sources of intense EMI in your daily environment. Some of these are obvious and easy to avoid. Others are not, and you may not be able to avoid them.
3. Powered wheelchairs may be susceptible to electromagnetic interference (EMI) emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitters, two-way radios, and cellular phones, although the wheelchair has been tested in accordance with EMC guidelines.
4. EMI can also be produced by conducted sources or electro-static discharge (ESD).

B. WHAT EFFECT CAN EMI HAVE?

1. EMI can cause your chair, without warning, to:
 - Release its electronic brakes
 - Move by itself
 - Move in unintended directionsIf any of these occurs, it could result in severe injury to you or others.
2. EMI can damage the control system of your chair. This could create a safety hazard, and lead to costly repairs.

C. SOURCES OF EMI

The sources of EMI fall into three broad types:

1. Hand-Held Transceivers:

The antenna is usually mounted directly on the unit. These include:

- Citizens band (CB) radios
- “Walkie-talkies”
- Security, fire and police radios
- Cellular phones
- Lap-top computers with phone or fax
- Other personal communication devices

NOTE– These devices can transmit signals while they are on, even if not in use. The wheelchair should be switched off, when not in use.

2. Medium-Range Mobile Transceivers:

These include two-way radios used in police cars, fire engines, ambulances and taxi cabs.

The antenna is usually mounted on the outside of the vehicle.

3. Long-Range Transceivers:

These include commercial radio and TV broadcast antenna towers, amateur (HAM) radios and alarm systems in department stores. **NOTE** – The following are not likely to cause EMI problems: Lap-top computers (without phone or fax), Cordless phones, TV sets or AM/FM radios, CD or tape players.

D. DISTANCE FROM THE SOURCE

EM energy rapidly becomes more intense as you get closer to the source. For this reason, EMI from hand-held devices is of special concern. (**See C.1**). A person using one of these devices can bring high levels of EM energy very close to your chair without you knowing it.

E. IMMUNITY LEVEL

1. The level of EM is measured in volts per metre (V/m). Every power wheelchair can resist EMI up to a certain level. This is called its "immunity level".
2. The higher the immunity level, the less the risk of EMI. It is believed that a 20 V/m immunity level will protect the power wheelchair user from the more common sources of radio waves.
3. The configuration tested and found to be immune to at least 20 V/m is:
Quickie/Sopur P-220 power wheelchair with a right-hand mounted joystick system, 18" seat width, 18" seat depth, dual-post height-adjustable armrests, fixed tapered legrests with one-piece solid footplate and Gp 24 gel cell batteries.
4. The following specialty input devices have an unknown effect on the immunity level because they have not been tested with the P-220 and the P&G Pilot+ control system:
 - Breath Control
 - Tri-Switch Head Array
 - Proximity Head Array
 - Proportional Mini-Joystick/Chin Control
 - Buddy Button
 - Wafer Board

Individuals with physical limitations requiring the use of a specialty control input device known not to be immune to 20 V/m, or not known, should exercise care around known sources of EMI.

There is no way to know the effect on EMI if you add accessories or modify this chair. Any change to your chair may increase the risk of EMI. Parts not specifically tested or parts from other suppliers have unknown EMI properties.

F. REPORT ALL SUSPECTED EMI INCIDENTS

You should promptly report any unintended movement or brake release. Be sure to indicate whether there was a radio wave source near your chair at the time.

Contact: Sunrise Medical, Customer Service Department at (800) 333-4000.

G. EMI FROM CHAIR

The chair itself can disturb the performance of electromagnetic fields such as emitted by alarm systems of shops.

VI. GENERAL WARNINGS

Heed all warnings in this section. If you fail to do so, a fall, tip-over or loss of control may occur and cause severe injury to you or others.

A. NOTICE TO USER

1. Before using this chair, you should be trained in its safe use by your health care professional.
2. Every wheelchair is different. Take the time to learn the feel of this chair before you begin using.
3. Be aware that you must develop your own methods for the safe use of this chair, that are best suited to your level of function and ability.
4. Have someone help you practice bending, reaching and transfers until you learn how to do them safely.
5. Never try a new manoeuvre on your own unless you are sure it is safe.
6. Get to know the areas where you plan to use your chair. Look for hazards and learn how to avoid them.

B. NOTICE TO ATTENDANTS

Make sure you heed all warnings and follow all instructions in each section of this manual. (Be aware that warnings that apply to the user also apply to you).

NOTES –

1. You need to work with the user, and the user's doctor, nurse or therapist, to develop safe methods best suited to your abilities and those of the user.
2. To manually push the chair you must release the free wheel mechanism.
 - Make sure you have full control over the chair when you release the free wheel mechanism. When you do so, the chair will not have brakes.
3. Propel this chair by the push handles only. They provide secure points for you to hold the rear of the chair to prevent a fall or tip-over.
 - Check to make sure push handle grips will not rotate or slip off.

Helping the user overcome an obstacle:

1. To prevent injury to your back, use good posture and proper body mechanics. When you lift or support the user or tilt the chair, bend your knees slightly and keep your back as upright and straight as you can.
2. Before each manoeuvre, tell the user what you plan to do, and explain what you expect the user to do. This will put the user at ease and reduce the risk of an accident.
3. Go straight up and straight down a curb or stair. If you turn, or climb or descend at an angle, a fall or tip-over is likely.
4. Remind the user to lean back when you tilt the chair backward.
5. Lower the chair slowly. Do not let the chair drop to the pavement or ground. Doing so may damage the chair or injure the user.

C. WEIGHT LIMIT

1. The user plus items carried should never exceed a total weight of 140 kg.
2. Never use this chair for weight training if the total weight (user plus additional weights) exceeds 140 kg.
3. Exceeding the weight limit is likely to damage the seat, frame or fasteners and may cause severe injury to you or others from chair failure.
4. Exceeding the weight limit will void the warranty.

D. CONTROLLER SETTINGS

Be aware that you may need to adjust the controller settings of your chair to reduce the risk of a collision, fall or tip-over.

1. Check and adjust the settings every six to twelve months (or more often, if needed).
2. Consult your supplier to adjust the control settings immediately if you notice any change in your ability to:
 - Control the joystick
 - Hold your torso erect
 - Avoid running into objects

E. EMI

Read Section V to learn about EMI. To reduce the risk of unintended brake release or chair movement:

1. Never turn on or use a hand-held transceiver while power to your chair is on. Use extra care if you believe that such a device may be in use near your chair.
2. Be aware of nearby radio or TV stations, and avoid coming close to them.
3. If unintended movement or brake release occurs, turn your chair off as soon as it is safe.

F. SAFETY CHECK-LIST

Before each use of this chair:

1. Make sure the chair operates smoothly. Check for noise, vibration, or a change in ease of use. (They may indicate low tyre pressure, loose fasteners, or damage to your chair).
 - If you detect a problem, make sure to repair or adjust the chair. Your supplier can help you find and correct the problem.
2. Make sure batteries are charged. Green lights on the charge indicator will light up when charge is full. Yellow lights indicate battery charge level is getting low. Red lights indicate batteries are in immediate need of charging.

G. CHANGES AND ADJUSTMENTS

Never use non-Quickie/Sopur parts or make changes to your chair unless authorised by Sunrise Medical. (Doing so will void the Warranty and may create a safety hazard).

1. If you modify or adjust this chair, it may increase the risk of a fall or tip-over.
2. Modifications which are not authorised by Sunrise Medical constitute remanufacturing of the wheelchair. This voids the warranty. The user then assumes all future liability for the wheelchair.

H. WHEN SEATED IN A PARKED WHEELCHAIR

1. Always turn off all power to your chair when you are parked, even for a moment. This will prevent:
 - Accidental movement from contact with the joystick by you or others.
 - Unintended brake release or movement from EMI sources (See Section V).
2. Make sure that persons who help you (for example shop assistants) are aware of the joystick and do not touch it. If they do, your chair may move suddenly when you do not expect it.

I. ENVIRONMENTAL CONDITIONS

Your chair is not designed for use in a heavy rain storm, or in snowy or icy conditions.

1. Contact with water or excessive moisture can cause an electrical malfunction. The frame, motors and other chair parts are not water-tight and may rust or corrode from the inside. To avoid a chair failure:
 - Minimise exposure of your chair to a rain storm or very wet conditions.
 - Never take your chair into a shower, bath, pool or sauna.
 - Do not use your chair in fresh or salt water (such as at the edge of a stream, lake or ocean).
 - Make sure battery covers are secure.
 - Replace joystick boot if it becomes torn or cracked.
 - Make sure all electrical connections are secure.
 - Dry the chair as soon as you can if it gets wet, or if you use water to clean it.
2. Proceed slowly and use extra care if you must operate your chair on a wet or slick surface.
 - Do so only if you are sure it is safe.
 - Stop if one or both main wheels lose traction. If this occurs, you may lose control of your chair or fall.
 - Never operate your chair on a slope or ramp if there is snow, ice, water or oil film present.
 - When in doubt, have someone help you.
3. When not in use, keep your chair in a clean, dry place.

Extra caution should be used when employing external switches (e.g. Buddy Buttons) or the proximity head array as control devices. These two devices are susceptible to malfunction when wet.

J. TERRAIN

1. This chair is designed for use on firm, even surfaces such as concrete, asphalt and indoor flooring.
2. Do not operate your chair in sand, loose soil or over rough terrain. Doing so may damage wheels, bearings, axles or motors, or loosen fasteners.

K. STREET USE

In most states, power chairs are not legal for use on public roads. Be alert to the danger of motor vehicles on roads or in carparks.

1. At night, or when it is hard to see, use reflective tape on your chair and clothing.
2. At night, or when it is hard to see, always switch on your lights and use your indicators when turning (light unit may not be available in all countries).
3. It may be hard for drivers to see you. Make eye contact with drivers before you proceed. When in doubt, give way until you are sure it is safe.

L. MOTOR VEHICLE SAFETY**⚠ WARNING**

1. Never sit in this chair while in a moving vehicle. In an accident or sudden stop, you may be thrown from the chair.
 - **Wheelchair belts are designed to position the user only and will not protect you in an accident; further injury may result from belts.**
2. Always move to an approved vehicle seat. You must be secured with proper motor vehicle restraints.
3. Never transport this chair in the front seat of a vehicle. It may shift and interfere with the driver.
4. Always secure this chair so that it cannot roll or shift.

M. CENTRE OF BALANCE**⚠ WARNING**

The point where this chair will tip forward, back or to the side depends on its centre of balance and stability.

The centre of balance is affected by:

1. The seat height and seat angle.
2. A change in your body position, posture or weight distribution.
3. Using this chair on a ramp or slope.
4. The use of a backpack or other options, and the amount of added weight.
5. Drive wheel positioning.

To reduce the risk of a fall or tip-over:

1. Consult your supplier for information on modifications authorised by Sunrise Medical, before you modify or adjust this chair.

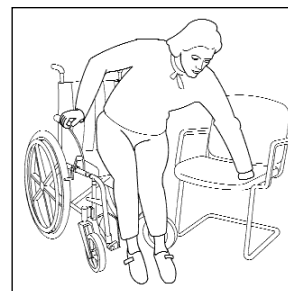
NOTE– You may need to make additional changes to correct the centre of balance.

2. Use extreme care until you know the balance points of this chair and how to avoid a fall or tip-over.

N. TRANSFERS**⚠ WARNING**

It is dangerous to transfer on your own. It requires good balance and agility. Be aware that there is a point during every transfer when the wheelchair seat is not below you. To avoid a fall:

1. Always turn off power before you transfer to or from your chair. If you fail to do so you may touch the joystick and cause your chair to move when you do not expect it.
2. Make sure free wheel mechanism is engaged. This keeps the chair from moving when you transfer.
3. Work with your health care professional to learn safe methods.
 - Learn how to position your body and how to support yourself during a transfer.
 - Have someone help you until you are sure you can do a safe transfer on your own.
4. Move your chair as close as you can to the seat to which you are transferring. If possible, use a transfer board.
5. Rotate the front casters until they are as far forward as possible.
6. Be careful of the footrests. If you can, remove or swing them out of the way.
 - Never stand on footrests when you transfer. Doing so may damage them or cause your chair to tip.
 - Make sure your feet do not “hang up” or get caught in the space between the footrests.
7. Make sure armrests do not interfere.
8. Transfer as far back onto the seat surface as you can. This will reduce the risk that you will miss the seat or fall.

O. REACHING OR LEANING**⚠ WARNING**

Reaching or leaning affects the centre of balance of your chair. If done improperly, a fall or tip over is likely. When in doubt, ask for help or use a device to extend your reach.

To reduce the risk of injury and/or damage to the chair:

1. Never reach or lean if you must shift your weight sideways or rise up off the seat.
2. Never reach or lean if you must move forward in your seat to do so. Always keep your buttocks in contact with the backrest.
3. Never reach with both hands (you may not be able to catch yourself to prevent a fall if you lose your balance).
4. Never try to pick up an object from the floor by reaching down between your knees.
5. Never put pressure on the footrests while reaching. This may cause the chair to tip if you lean too far.
6. Never reach or lean over the top of the seat back. This may damage the backrest and cause you to fall.

IF YOU MUST REACH OR LEAN, DO SO AT YOUR OWN RISK.**Remember to:**

1. Move your chair as close as you can to the object you wish to reach.
2. Rotate the front casters until they are as far forward as possible. This makes the chair more stable.

NOTE– To do this: Move your chair past the object you want to reach, then back up alongside it. Backing up will rotate the casters forward.

3. Turn off all power to your chair. If you fail to do so, you may touch the joystick and cause your chair to move when you do not expect it.
4. Firmly grasp a rear wheel or an armrest with one hand. This will help to prevent a fall if the chair tips.

P. DRESSING OR CHANGING CLOTHES**⚠ WARNING**

Be aware that your weight will shift if you dress or change clothes while seated in this chair. To make the chair more stable, rotate the front casters until they are **forward**.

Q. OBSTACLES (refer to drawings)**⚠ WARNING**

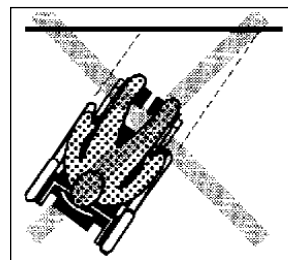
Obstacles you may have to overcome in daily use include door thresholds, lifts, ramps and hazards such as potholes and broken pavement. These can damage your chair and may cause a fall, tip-over or loss of control.

1. Be aware that thresholds are very dangerous. (Even a small change in height may stop a caster wheel and cause your chair to tip). You may need to:
 - Remove or cover threshold strips between rooms.
 - Install a ramp at entry or exit doors.
2. Keep your eyes moving when you ride; scan the area well ahead of your chair.
3. Make sure the floor areas where you use this chair are level and free of obstacles.
4. To help correct your centre of balance:
 - Lean your upper body forward slightly as you go up over an obstacle.
 - Press your upper body backward as you go down from a higher to a lower level.


R. DRIVING IN REVERSE**⚠ WARNING**

Use extra care when you drive your chair in reverse. You may lose control or fall if one of the rear wheels hits an object.

1. Operate your chair slowly and at an even speed.
2. Stop often and check to make sure your path is clear of obstacles.

Wrong kerb approach

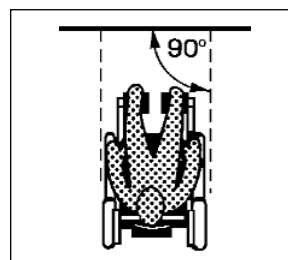Correct kerb approach

S. RAMPS, SLOPES AND SIDEHILLS

⚠ WARNING

The centre of balance of your chair changes when you on a slope.

NOTE– “Slope” includes a ramp or sidehill. Your chair is less stable when it is at an angle. Never use this chair on a slope unless you are sure it is safe. When in doubt have someone help you.

Beware of:

1. Steep slopes. Do not use this chair on a slope steeper than 10°.
2. Wet or slippery surfaces (such as when ice, snow, water or oil film is present). A loss of traction may cause a fall or tip-over.
3. A change in grade on a slope (or a lip, bump or depression). These may cause a fall or tip-over.
4. A drop-off at the bottom of a slope. (A drop-off as small as 3/4" (12 mm) can stop a front caster and cause the chair to tip forward).
5. If the automatic brakes are released, then the movement of the wheelchair may be dangerous.

T. TO REDUCE THE RISK OF A FALL, TIP-OVER OR LOSS OF CONTROL:

⚠ WARNING

1. Never use your chair on a slope unless you are sure you can do so without losing traction.
2. Always go as straight up and as straight down as you can.
 - Do not “cut the corner” on a slope or ramp
 - Do not turn or change direction on a slope.
3. Always stay in the centre of the ramp. Make sure the ramp is wide enough that you are not at risk that a wheel may roll off the side.
4. Lean or press your body uphill. This will help adjust for a change in the centre of balance caused by the slope (Fig. 1).
5. Keep your chair moving at a slow, steady speed. Keep control over the chair at all times.
 - On a descent, do not let your chair accelerate beyond its normal speed.
 - If the chair picks up speed, centre the joystick to slow down or stop.

NOTE– The solid state controller of your chair has a logic system that will help control your speed when driving on a slope or up hill.

- If you stop, re-start slowly.
6. Never use rear wheel locks to slow or stop your chair. This is likely to cause the chair to veer out of control.

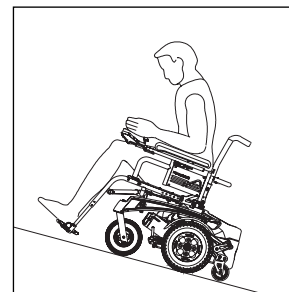
Figure 1

U. RAMPS AT HOME AND WORK

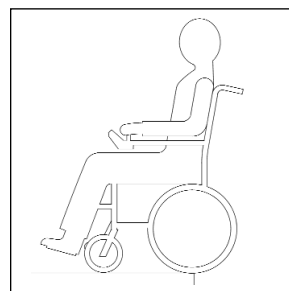
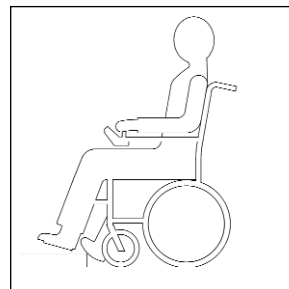
⚠ WARNING

Make sure ramps meet all Building Codes for your area.

1. For your safety, have a licensed contractor build or remodel ramp to meet all standards.
2. **NOTE**– The proper design will vary, depending on such things as: the length and height of the ramp; the need for an intermediate platform; landing size; doors, and the direction of swing, and whether the ramp includes a turn or angle.

At a minimum:

1. Open sides of ramp must have side rails to prevent your chair from going over the edge.
2. Slope must not be steeper than one inch in height for every one foot of slope length (approximately 5°).
3. Ramp surface must be even and have a non-skid surface.
4. You may need to add a section at the top or bottom to avoid a lip or drop-off.
5. Ramp must be sturdy. Add bracing if needed, so that the ramp does not “bow” when you ride on it.



V. WHEELCHAIR LIFTS**⚠ WARNING**

Wheelchair lifts are used in vans, buses, and buildings to help you move from one level to another.


1. Always turn off all power to your chair when you are on a lift. If you fail to do so, you may touch the joystick by accident and cause your chair to drive off the platform. (Be aware that a "roll-stop" at the end of the platform may not prevent this).
2. Make sure there is not a lip or drop-off at the top or bottom of the platform. These may cause a fall or tip-over. When in doubt, have someone help you.
3. Always position the user securely in the chair to help prevent falls while on a lift.
4. Avoid moving forward if a wheel is "hung up" on the lip of the ramp. Backup, reposition the caster for a more direct approach. And slowly try again.

W. KERBS AND SINGLE STEPS (refer to drawings)**⚠ WARNING**

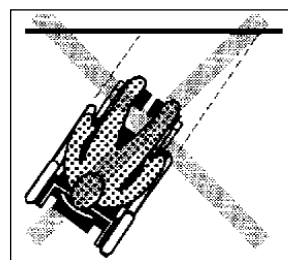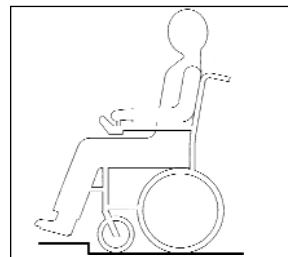1. Your chair is not designed to drive up or down a curb or step more than four (4) inches (10 cm) high. Doing so may:
 - Result in a fall or tip-over.
 - Damage the frame, wheels, axles or other chair parts, or loosen fasteners.
2. To prevent a fall or tip-over, use wheelchair access ramps or have someone help you.
3. If you must climb or descend a curb or step alone, do so at your own risk and use extreme care.
 - Go as straight up or straight down as you can. Never turn or climb or descend at an angle as a fall or tip-over is likely.
 - Proceed slowly, at a steady speed.
4. Make sure that persons who assist you review the "Tips for Attendants" and heed all warnings.
5. When you are driving up or down steps, make sure that the dynamic stabilisers cannot hang up, because they may interfere with the kerbs.

Wrong kerb approach

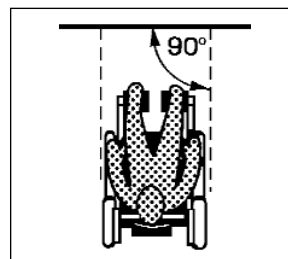Correct kerb approach

X. STAIRS**⚠ WARNING**

Never use this chair to go up or down stairs, even with an attendant. Doing so is likely to cause a fall or tip-over.

Y. ESCALATORS**⚠ WARNING**

Never take this chair on an escalator, even with an attendant. Doing so is likely to cause a fall or tip-over.

VII. WARNINGS: COMPONENTS AND OPTIONS**⚠ WARNING**

Note: If you use parts or make changes not authorised by Sunrise Medical it may create a safety hazard and will void the Warranty.

A. ANTI-TIP LEVERS**⚠ WARNING**

Never remove or alter anti-tip levers. They help keep your chair from tipping over backwards in normal use. Make sure rubber rollers are in good condition.

B. ARMRESTS**⚠ WARNING**

Armrests detach and will not bear the weight of this chair.

1. Never lift this chair by its armrests. They may come loose or break.
2. Lift this chair only by non-detachable parts of the main frame.

C. BATTERIES**⚠ WARNING**

1. Only deep cycle sealed case construction batteries should be used in this device.
2. Always keep batteries upright.
3. Never smoke or hold an open flame near batteries. They are a known explosion hazard.
4. Always wear rubber gloves and safety glasses when you handle batteries.
5. Read all of section XI. Batteries before attempting to change, or charge batteries.

D. CUSHIONS AND SLING SEATS**⚠ WARNING**

1. Quickie/Sopor sling seats, standard foam cushions, and other body supports, are not designed for the relief of pressure.
2. If you suffer from pressure sores, or if you are at risk that they will occur, you may need a special seat system or a device to control your posture.
 - Consult your doctor, nurse or therapist to find out if you need such a device for your well-being.

E. FASTENERS**⚠ WARNING**

Many of the screws, bolts and nuts on this chair are special high-strength fasteners. Use of improper fasteners may cause your chair to fail.

1. Only use fasteners provided by Sunrise Medical.
2. If fasteners become loose, tighten them as soon as you can.
3. Over- or under-tightened fasteners may fail or cause damage to chair parts.
 - See Section IX, "Set-Up, Adjustment and Use", for proper torque settings.

F. FOOTRESTS**⚠ WARNING**

1. At the lowest point, footrests should be at least 2 1/2 inches (65 mm) off the ground. If set too LOW, they may "hang up" on obstacles you can expect to find in normal use. This may cause the chair to stop suddenly and tip forward.
2. To avoid a trip or fall when you transfer:
 - Make sure your feet do not "hang up" or get caught in the space between the footrests.
 - Avoid putting weight on the footrests, as the chair may tip forward.
 - Remove or swing the footrests out of the way, if possible.
3. Never lift this chair by the footrests. Footrests detach and will not bear the weight of this chair. Lift this chair only by non-detachable parts of the main frame.

G. FREE WHEEL MECHANISM**⚠ WARNING**

1. Do not engage or disengage the free wheel mechanism unless power to the chair is off.
2. With the brakes released, the chair will not operate and the battery charge indicator will flash 9 bars rapidly (if power to the chair is on).
3. Be aware that the chair will not have brakes when the free wheel mechanism is in the free-wheel position.
4. Make sure that the person pushing the chair has full control when the free wheel mechanism is disengaged.
5. Both motors must be disengaged, so that the wheelchair can be pushed.

H. ON/OFF SWITCH**⚠ WARNING**

1. Never use the ON/OFF switch to stop the chair except in an emergency. This will result in an abrupt stop, and may cause you to fall.
2. To slow your chair to a stop, return the joystick to neutral.

I. PNEUMATIC TYRES**⚠ WARNING**

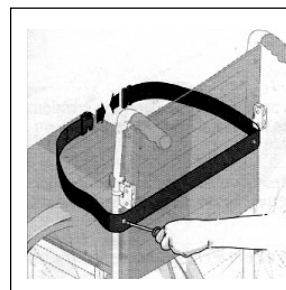
Proper inflation extends the life of your tyres and makes your chair easier to use.

1. Do not use this chair if any of the tyres are under- or over-inflated. Check weekly for proper inflation level, as listed on the tyre sidewall.
2. Low pressure in a tyre may cause the chair to veer to one side and result in a loss of control
3. An over-inflated tyre may burst.
4. Never use a petrol station air pump to inflate a tyre. Such pumps provide air at high volume, and could cause the tyre to burst. To prevent tyre damage:
 - Use a hand pump (or a low volume air pump) to inflate tyres.
 - Use a tyre gauge to check pressure.
5. Driving over sharp objects may cause damage to pneumatic tyres and tubes.

J. POSITIONING BELTS (OPTIONAL)**⚠ WARNING**

Use a positioning belt only to help support your posture. Improper use of such belts may cause severe injury or death.

1. Make sure you are not at risk to slide down in the wheelchair seat. If this occurs, you may suffer chest compression or suffocate due to pressure from the belt.
2. A pelvic wedge or a similar device can help keep you from sliding down in the seat. Consult your health care professional to find out if you need such a device.
3. The belt must be snug, but must not be so tight that it interferes with breathing. You should be able to slide your open hand, flat, between the belt and your stomach.
4. Make sure you can easily remove the belt in an emergency.
5. Never use a positioning belt:
 - In place of a motor vehicle seat belt. In an accident or sudden stop you may be thrown from the chair. A positioning belt will not prevent this, and further injury may result from the belt.
 - As a restraint. A restraint requires a doctor's order.
 - On a user who is comatose or agitated.

**K. PUSH HANDLES****⚠ WARNING**

1. Push handles provide secure points for an attendant to propel and control the chair. This helps to prevent a fall or tip-over.
2. Check to make sure push handle grips will not rotate or slip off.

L. REAR WHEEL LOCKS (OPTIONAL)**⚠ WARNING**

If you request them, we will install rear wheel locks at Sunrise Medical.

1. Rear wheel locks are not designed to slow or stop a moving wheelchair. Never apply them when your chair is moving. Doing so may cause you to veer out of control.
 - Use wheel locks only to keep the rear wheels from rolling when your chair is at a complete stop.
2. Low pressure in a rear tyre may cause the wheel lock on that side to slip and may allow the wheel to turn when you do not expect it.
3. Make sure lock arms embed in tyres at least 3/8" (9.5 mm) when locked. If you fail to do so, the locks may not work.

M. SEATING SYSTEMS**⚠ WARNING**

1. Use of a seating system not approved by Sunrise Medical may alter the centre of balance of this chair. This may cause a fall or tip-over.
2. Never change the seating system of your chair unless you consult your supplier first.

N. UPHOLSTERY FABRIC**⚠ WARNING**

1. Replace worn or torn fabric of seat sling and seat back as soon as you can. If you fail to do so, the seat may fail and cause you to fall. Worn fabrics may increase the potential for a fire hazard.
2. Sling fabric will deteriorate with age and use. Look for fraying, thin spots or stretching of fabric at rivet holes. Replace fabric as required.
3. "Dropping down" into sling seat will weaken fabric and result in the need to inspect and replace sling on a more frequent basis.
4. Be aware that washing may reduce flame retardation of the fabric.

VIII. TIPS FOR ATTENDANTS** WARNING**

1. Persons who help a user do one of the following tasks should review and heed the warnings “Notice to Attendants” and all warnings in this Manual for that task.
2. The “Tips” that follow are suggestions only. Be aware that you will need to learn safe methods best-suited to the user and to your abilities. Consult your health care professional for instructions.

A. TO CLIMB A CURB OR SINGLE STEP** WARNING**

The following is one way to safely help a user climb a curb or single step going forward:

1. Stay behind the chair.
2. Face the curb and tilt the chair up on the rear wheels so that the front casters clear the curb or step.
3. Move forward, placing the front casters on the upper level as soon as you are sure they are past the edge.
4. Continue forward until the rear wheels contact the face of the curb or step. Lift and roll the rear wheels to the upper level.

B. TO DESCEND A CURB OR SINGLE STEP** WARNING**

The following is one way to safely help a user climb a curb or single step going backward:

1. Stay at the rear of the chair.
2. Several feet before you reach the edge of the curb or step, turn the chair around and pull it backward.
3. Proceed carefully. Look over your shoulder and carefully step back until you are off the curb or stair and standing on the lower level.
4. Pull the chair toward you until the rear wheels reach the edge of the curb or step. Then allow the rear wheels to slowly roll down onto the lower level.
5. When the rear wheels are safely on the lower level, tilt the chair back to its balance point. This will lift the front casters off the curb or step.
6. Keep the chair in balance and take small steps backwards. Be sure to look where you are going. Turn the chair around and gently lower front casters to the ground.

IX. SET-UP, ADJUSTMENTS AND USE

NOTES

1. Work surface for set-up:

Use a flat surface, such as a table, to assemble, adjust and check your chair. This makes the steps easier and helps ensure a correct set-up.

2. Fasteners:

- Many of the screws and bolts on this chair are special high-strength fasteners and may have special coatings.
- Many nuts are of the Nylock type. They have a plastic insert to help prevent loosening.
- Only use screws, bolts and nuts provided by Sunrise Medical.

WARNING

1. Use of improper fasteners may cause the chair to fall.
2. Over- or under-tightened fasteners may fail or cause damage to chair parts.
3. If bolts or screws become loose, tighten them as soon as you can. Loose bolts or screws can cause damage to other chair parts causing them to fail.

3. Washers and spacers:

- Note the position of washers and spacers before disassembly.
- To avoid damage to the frame, replace all washers and spacers when you reassemble parts.

4. Torque settings:

- A torque setting is the optimal tightening for a particular fastener. Use a torque wrench that measures inch-pounds to secure screws, nuts and bolts on this chair.

NOTE– Unless otherwise noted, use a torque setting of 120 inch-pounds (13.5 Nm) for all fasteners.

TOOLS YOU WILL NEED

1. Basic Tool Kit:

To set-up, adjust and maintain your chair you will need the following tools:

- 7/16" box and open-end wrench
- 1/2" box and open-end wrench
- 3/4" box and open-end wrench
- 5/32" Allen wrench
- 3/16" Allen wrench
- 1/8" Allen wrench
- Phillips screwdriver #2
- Custom axle wrench (or a 1/2" open-end wrench)

You can obtain a multi-purpose tool kit from Sunrise Medical or buy the tools you need from a hardware store.

2. Torque Wrench

If you plan to adjust and maintain this chair yourself, Quickie/Sopur recommends that you use a torque wrench.

NOTE– The wrench must measure inch-pounds and Nm. You can buy a torque wrench and proper sockets from a hardware store.

CHECK-OUT

Be sure this chair performs to your chosen operational settings. If it does not, turn the chair off immediately and contact your supplier.

NOTE– Repeat this procedure until the chair performs to your specifications.

WARNING

When properly set-up, this chair will operate smoothly. Check to see that all components work properly. If you detect a problem, be sure to correct it before use.

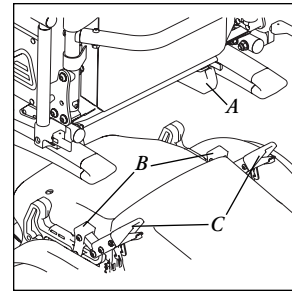
A. POWER DRIVE UNIT

Wheelchair and power drive attachment

- Position seat and main frame over power drive unit.
- Align quick-connect blocks (A) with binding socket (B) located on power drive unit.
- Ensure frame is properly located on power drive unit and the controller cable is clear.
- Press levers (C) down until locked.
- Lever force required can be adjusted by adjusting screw.

⚠ WARNING

Always secure levers before operating chair. If the chair fails to operate and the battery charge indicator flashes 9 bars, check both quick-connect levers for securement. Then cycle the on/off switch.

B. BATTERY REMOVAL

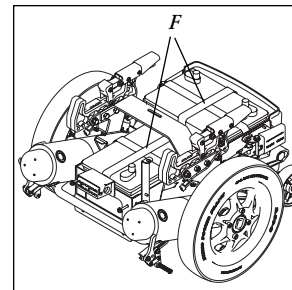
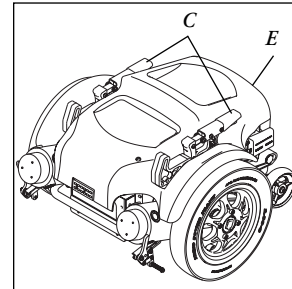
To remove the batteries for transporting or servicing:

Remove batteries

- Remove seat and main frame from power drive unit using quick-release levers shown above (C).
- Remove battery compartment lid by releasing the two (2) quarter turn fasteners.
- Prevent the power drive unit from tipping forward by placing a solid object underneath the front of the power drive unit.
- Lift battery compartment (E) off the power drive unit.
- Disconnect battery leads from power harness.
- Lift rear-most battery out using the battery straps (F).
- Pull the front battery backwards, disconnect it, and remove it using the battery straps.

⚠ WARNING

Batteries may weigh up to 25 kg. Care must be taken to avoid injury when lifting.

C. FOLDING BACKREST

Quickie/Sopur P-220 wheelchairs are shipped with backrest in folded position.

1. Upright positioning

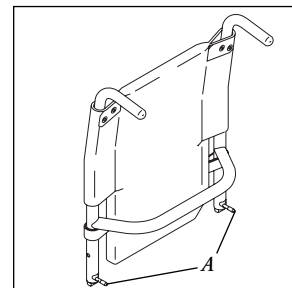
- Lift backrest to upright position so that latch handles (A) lock into place.
- Backrest is ready for use.

2. Folding backrest

- Lift and turn backrest latch handles (A) inward or outward.
- Fold backrest forward.

⚠ WARNING

CAUTION: Backrest latches must be locked into place before operating wheelchair.

D. SWING-AWAY FOOTRESTS

1. Installation

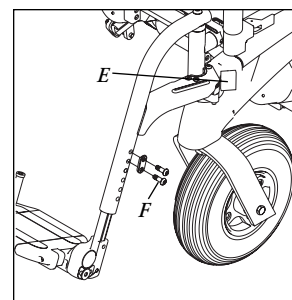
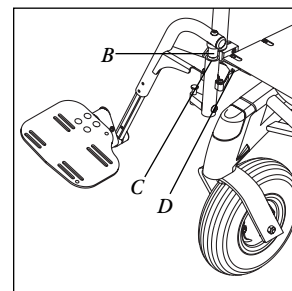
- Place swing-away pivot pin (B) into the locating hole (C) on top of the hanger mount with the footrest facing outward from frame.
- Rotate the footrest inward until latch plate locks (D) into place on locking bolt.

2. Removal

- To remove footrest, press quick-release lever (E).
- Rotate footrest outward and lift.

3. Height adjustment

- Loosen all four (4) set screws (F) in the hanger frame tube.
- Reposition footrest height.
- Retighten all set screws (F).


E. ELEVATING LEGREST (OPTIONAL)

1. Installation

To install or remove Elevating Legrest (ELR) see instructions for swing-away footrest installation.

2. Adjustment

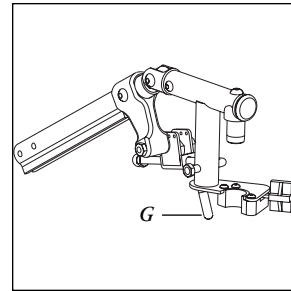- To raise legrest, lift to desired position. Legrest will automatically lock in place.
- To lower legrest, while seated in chair, first lift slightly then press release lever (G) down and lower legrest to desired position.


F. JOYSTICK INSTALLATION

1. To connect

- Line up small cylindrical connector with its receptacle on the rear of the joystick.
- Push in firmly.


2. To disconnect

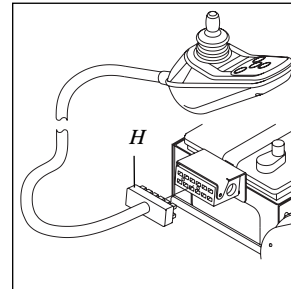Pull back on the retaining ring (H) of the cylindrical connector until it disconnects from the joystick box.

G. TO ADJUST THE HEIGHT OF THE REMOTE JOYSTICK

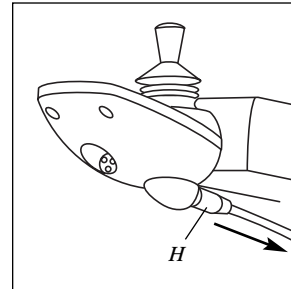- Remove the two pan-head screws (A), securing the angle bracket mount (B).
- Remove the joystick from the mount by unfastening the two screws (C) on the underside of the mount.
- Position angle bracket mount such that you achieve the desired joystick height. Four positions are available by relocating the mount either on the top or bottom of the mounting bar (D) and/or by rotation of the angle bracket mount.


H. REMOTE JOYSTICK SWING-AWAY RETRACTABLE MOUNT (OPTIONAL)

The remote joystick is mounted with a mechanism which allows the control to be locked in a forward or retracted position.

Adjustment

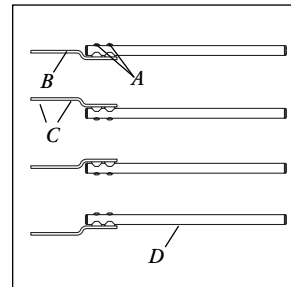- To retract joystick, depress lever (E) located on the side of the armrest.
- Push joystick away from front of armrest until it locks into retracted position.
- To return to forward position, depress lever and push joystick forward until it locks into place.

I. DUAL-POST HEIGHT-ADJUSTABLE ARMRESTS

The dual-post flip-back armrest can be used as either a flip-back armrest or a removable armrest.

1. Installing armrest

- Insert front and rear posts into armrest receivers.
- Tighten clamp levers (A) and (B) to secure armrest.

2. Set-up for flip-back operation

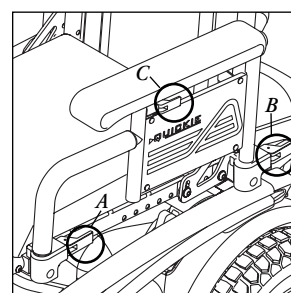
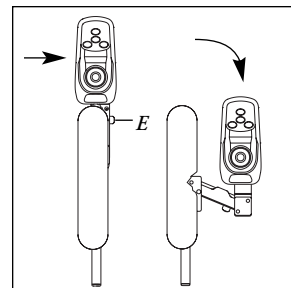
- Loosen clamp lever (A) so front post is able to come free.
- Armrest now has the ability to be flipped back without removal.

3. Set-up for removal

- Insert front and rear posts into armrest receivers.
- Loosen clamp lever (A) and (B) so front and rear post are able to come free.
- Armrest can now be easily removed for transferring.

4. Height-Adjustment (option)

- Release the upper securing lever (C).
- Set at desired height.
- Return securing lever to locked position.
- Move armrest up or down to allow armrest to snap into place.

J. SINGLE-POST HEIGHT-ADJUSTABLE ARMRESTS (OPTIONAL)**1. Installation**

- Slide the outer armpost into the receiver mounted to the wheelchair frame.
- The armrest will automatically lock into place.

2. Height adjustment

- Rotate release lever to stop.
- Slide armrest pad up or down to desired height.
- Return lever to locked position.
- Push arm pad until upper armpost locks firmly into place.

3. Removing armrest

- Rotate release lever and remove the armrest.

4. Replacing armrest

- Slide armrest back into receiver.
- Return release lever to locked position against armpost.

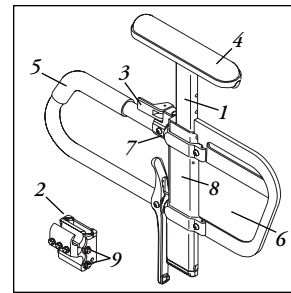
5. Adjusting armrest receiver fit

To tighten or loosen the fit of the outer armpost in the receiver:



- Loosen the four bolts on the sides of the receiver.
- With the armrest in the receiver, squeeze the receiver to achieve the desired fit.
- Tighten the four bolts.

6. Adjusting inner armpost fit

- Two set screws are installed in the outer armpost.
- Turn the set screws in or out until the desired fit is achieved.

**Height-Adjustable Armrest Key**

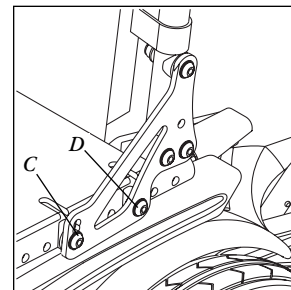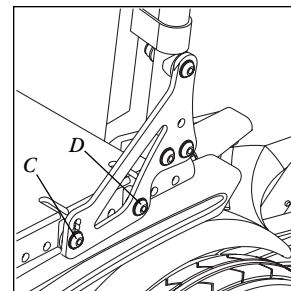1. Inner Arm Post
2. Receiver
3. Release Lever, Locked Position
4. Armrest Pad
5. Transfer Bar
6. Side Panel
7. Tension Adjustment Set Screw
8. Outer Arm Post
9. Receiver Adjustment Bolts

**K. SEAT DEPTH**

The seat depth can be adjusted in one inch increments.

1. Seat frame depth adjustment

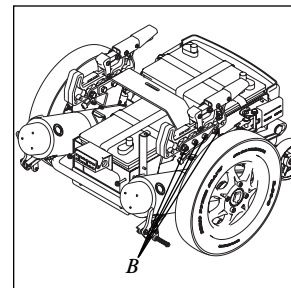
- To adjust, remove the two bolts (C and D) from each side of the backrest pivot plate.
- Reposition the backrest to the desired position.
- Replace and retighten bolts on each side of the backrest pivot plate.

2. Back angle adjustment on seat frame

- Remove the front securing bolt (C) on the side of the backrest hinge plate.
- Loosen the lower rear bolt (D).
- Set at desired angle. There are five holes (in 4° increments) to choose from.
- Reinstall the front bolt and tighten both bolts securely

L. 12" (300mm) DRIVE WHEEL**Drive wheel position adjustment**


- Remove the seat and main frame and the batteries.
- Remove the four (4) bolts (B) using a 1/2" wrench from each quick-connect socket.
- Move the socket forward or backwards to the desired position.
- Reinstall the four (4) bolts in the quick-connect socket. Use a torque setting of 240 in.-lbs. (27 Nm) to tighten bolts.

**M. DYNAM1C STABILISER****4" (100 mm) rear wheel adjustment**

- Remove rear batteries.
- Remove the upper pivot bolt (A) on the rear wheel spring.
- Set at desired height.
- Tighten nut. Use a torque setting of 240 in.-lbs. (27 Nm) to tighten nut.
- Height adjustment may also be made by using extra set of mounting holes on rear wheel mount bracket.

NOTE- The lowest point of the rear wheels should be 1/4" to 1/2" (6 to 13 mm) from the ground to allow for proper clearance (with the tyres properly inflated and user sitting in the chair). A smaller clearance produces less rocking motion, and a larger clearance allows easier movement on deep carpets.

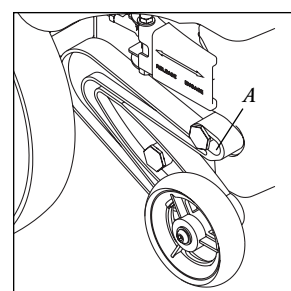
NOTE- In addition to the user's weight, the seat depth and back angle adjustment will affect this choice. Choose the position that provides the most comfortable transition onto and off the dynamic stabiliser wheels.

N. LIGHT UNIT

If your wheelchair is equipped with a light unit, when you are disassembling the wheelchair, you must also undo the plug connection to the light module, in order to disconnect the frame from the drive unit. The light circuit is protected with blowout fuses, which are located under the seat, along the light cable harness. Should a fuse (1A T) blow, then you can replace it with a standard fuse of the same rating.

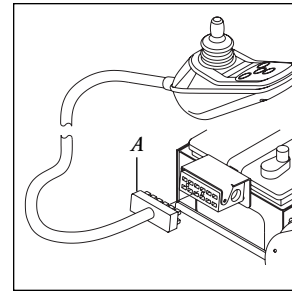When in doubt, please contact your Sunrise Medical supplier.


MANUAL WHEELCHAIR CONVERSION KIT

A. JOYSTICK

Removal

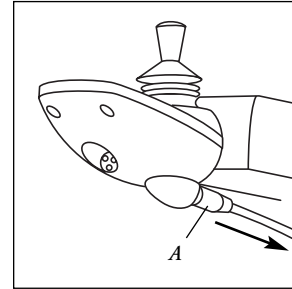
- Disconnect the joystick by pulling back on the retaining collar (A).
- Remove joystick power cable from armrest.
- Remove joystick assembly.

B. POWER DRIVE UNIT

Detachment

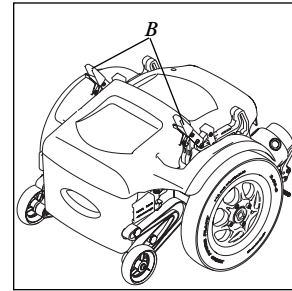- Release both quick-connect levers (B) until completely disengaged.
- Lift seat and main frame off power unit.
- Set power unit aside.


C. WHEEL LOCKS

Wheel locks are installed by Sunrise Medical when requested on the Quickie/Sopur P-220.

Adjustment

The wheel locks mount on a split clamp system. Use a torque of 100 in./lbs. (11.3 Nm) when adjusting the position of the wheel locks.

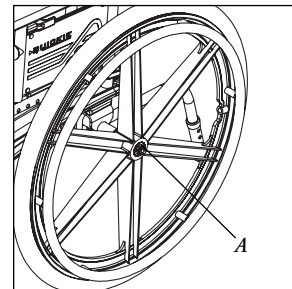
- Loosen the screws on the side of each clamp (C) using a 3/16" Allen wrench. Do not attempt to remove one screw at a time.
- Slide assembly toward rear wheel until clamp embeds into tyre to prevent wheel movement when in the locked position.
- Adjust angle position. Tighten screws.

D. 24" (600mm) REAR WHEELS

Installation

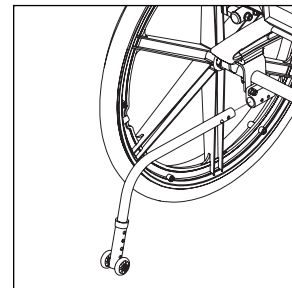- Depress the quick-release button (A) on the axle.
- Slide it into the axle sleeve until it locks.


E. ANTI-TIP TUBES

Sunrise Medical recommends anti-tip tubes for all wheelchairs.

NOTE– Use a torque setting of 100 in.-lbs (11.3 Nm) when setting-up anti-tip receivers.

1. Inserting anti-tip tubes into receiver

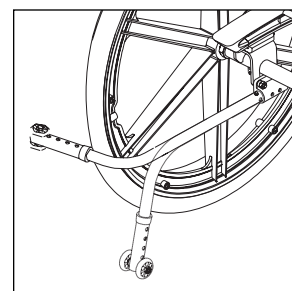- Press the rear anti-tip release lever on the anti-tip tube so that both release pins are drawn inside.
- Insert into the anti-tip tube receiver on the axle tube assembly.
- Turn the anti-tip tube down until release pin is through the receiver.
- Insert second anti-tip tube the same way.


2. Turning anti-tip tubes up

Turn anti-tip tubes up when being pushed by attendant, overcoming obstacles or climbing kerbs.

- Press the rear anti-tip tube release lever.
- Hold lever in and turn anti-tip tube up.
- Release lever.
- Repeat with second anti-tip tube.
- Remember to return anti-tip tubes to down position after completing manoeuvre.

F. INSTALLATION OF AXLE TUBE ASSEMBLY

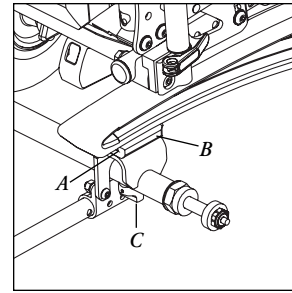1. Align and slide the receivers on the axle tube assembly (A) with the guides (B) located on the underside of the rear frame.
2. Rotate spring knobs (C) to disengage position.
3. Slide the axle tube assembly onto the guides.
4. Engage locking pins by rotating spring knob.
5. Ensure spring knob is in correct position and locking pin is engaged.
6. Ensure 24" (600 mm) manual wheels and anti-tip tubes are installed on the axle tube assembly.

**G. CHECK-OUT**

Once the wheelchair is assembled and adjusted, it should roll smoothly and easily. All accessories should also perform smoothly.

After the wheelchair has been set up, be sure the chair performs to your specified operational settings (see *X. Operating Guide*). If the chair does not perform to specifications, turn the wheelchair OFF immediately and reprogram operational specifications.

Repeat this procedure until the wheelchair performs to specifications before attempting active use of the wheelchair.

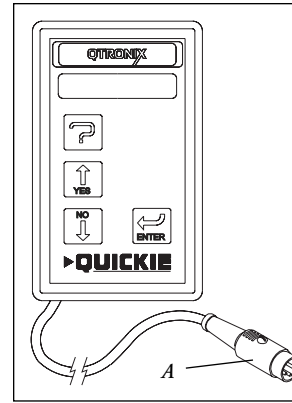
If you have any problems, follow these procedures:

1. Review the set-up and check-out section and operating guide to make sure chair was properly prepared.
2. If your problem persists, contact your authorised supplier. If you still have a problem after contacting your authorised supplier, contact Sunrise Medical customer service. See the introduction page for details on how to contact your authorised supplier or Sunrise customer service.

X. OPERATING GUIDE

A. PERFORMANCE CONTROL SETTINGS

1. It is vital to match control settings to your level of function and ability.
2. Consult your health care professional and your supplier to select the best control settings for you.
3. Check and adjust the settings every six to twelve months (or more often, if needed).
4. Adjust the control settings immediately if you notice any change in your ability to:
 - Control the joystick.
 - Hold your torso erect.
 - Avoid running into objects.
5. Control Settings are adjusted through the use of the Quickie/Sopur P&G PILOT+ Programmer . See your supplier to adjust the programming of the wheelchair.

B. P&G PILOT+/VSI PROGRAMMING PAD (OPTIONAL)

⚠ WARNING

Program settings beyond the ability of the user can result in serious injury. Consult your health care advisor before you alter settings.

1. Notes -

The P&G Pilot+/VSI Programmer lets you adjust the control settings of the P-220. You can customise a program for yourself or select the standard program.

- | | |
|-------------------------------|-------------------------------|
| • acceleration rate | • deceleration rate |
| • turn acceleration | • turn deceleration |
| • forward speed (max and min) | • reverse speed (max and min) |
| • turn speed (max and min) | • steer correct |
| • 4-axis joystick throw | • sleep timer |
| • read timer | |

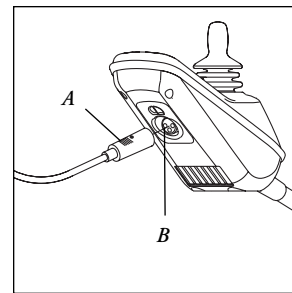
2. To program your chair:

- Turn off power to the chair.
- Insert P&G Pilot+/VSI Programmer plug (A) into the Programmer/Charger socket (B) on the front of the joystick.
- Turn on power to the chair; program as needed.
- Remove plug and cycle ON/OFF switch to drive chair.


NOTE- Review the P&G Pilot+/VSI Programmer Owner's Manual for more details on how to program your chair.

NOTE- Program settings that are not matched for the capabilities of the user can result in serious injury. Do not alter settings without the advice of your health care professional.

NOTE- Programming the controller of the wheelchair is only allowed through authorised Sunrise Medical Agents.

VSI

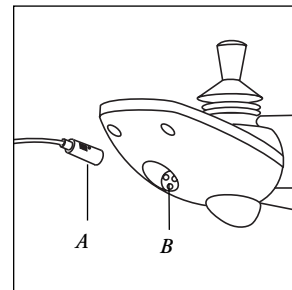Pilot+

WRONG CONTROLLER SETTINGS COULD CAUSE DRIVING OUTSIDE THE SAFE LIMIT AND COULD CAUSE DAMAGES AND INJURYC. THERMAL ROLL-BACK

Your chair has a thermal roll-back circuit. This protects the controller from damage due to over-heating. In extreme conditions (such as repetitive hill climbing) the circuit will decrease the power to your motors. This allows the chair to operate at a reduced speed. When the controller cools, the chair will return to normal speed.

D. CIRCUIT BREAKERS

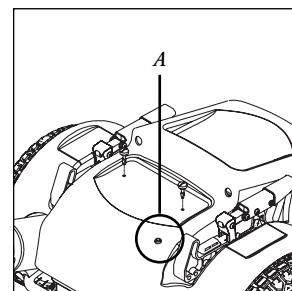
1. Notes -

Your Quickie/Sopur P-220 has a single circuit breaker (A).

- In the unlikely event of a short circuit or heavy overload, all power to your chair will be shut off.
- To reset your chair, depress the circuit breaker button (A). A few minutes' wait is required before the circuit breaker will reset.

2. Repeated Shutdown:

If the chair continues to shut down after resetting, have it serviced by a supplier.

E. JOYSTICK ASSEMBLY

1. INTEGRAL JOYSTICK ASSEMBLY VSI

A– On/off switch

This key switches your wheelchair on and off. Do not use this switch to stop the wheelchair: Except in an emergency.

B– Battery charge indicator

This is a row of 10 LED and indicates to you that the wheelchair is switched on and shows you the current charge status of the batteries. In addition this display is used for fault diagnosis. This is described in the user instructions for your wheelchair.

C–Speed preset

This shows the selected speed level of your wheelchair.

D– Reducing the maximum speed: Pressing this key reduces the maximum attainable speed, corresponding to the LED's on the display.

E– Increasing the maximum speed: Pressing this key increases the maximum attainable speed, corresponding to the LED's on the display.

F– Horn

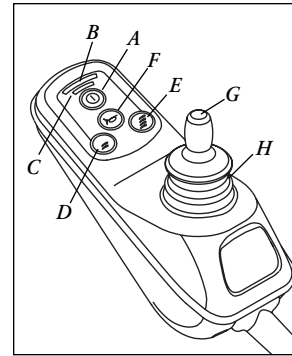
G– Joystick

H– Joystick Boot

Make sure the boot is not torn or cracked (this could allow debris, water or moisture to enter). If the boot is torn or cracked, replace it as soon as you can.

Light unit

This is only applicable if your joystick has keys for lights, indicators (right, left) and the hazard warning lights. Pressing the key switches the function on, pressing it again switches the function off.

VSI Joystick

2. PILOT+ WHEELCHAIR CONTROL WITH SEPARATE POWER MODULE

The operating pod is connected to a controller, which controls the chair. The operating pod unit is made up of the following components:

Operating pod

Operating pod displays

The current design may vary depending on the specification of the wheelchair. Most operating pods are as follows:

A– On/off switch

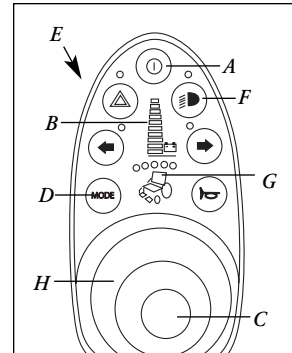
This key switches your wheelchair on and off. Do not use this key to stop the wheelchair: Except in an emergency.

B– Charge status indicator

This is a row of 10 LED and indicates to you that the wheelchair is switched on and shows you the current charge status of the batteries. In addition this display is used for fault diagnosis. This is described in the user instructions for your wheelchair.

C– Joystick

This is used to control the direction and speed of your wheelchair. The further you push the joystick, the faster your wheelchair will go. As soon as you let go of the joystick, the wheelchair will stop and the automatic brakes will take effect.

Pilot+ Joystick

D– Speed preset

You use this to select the maximum speed of your wheelchair. Press the MODE key. The 5 LED for the speed preset will flash. By pushing the joystick to the left, you will reduce the mobile speed, by moving the joystick to the right, you will increase this to the maximum permissible speed (6 or 10 kmh). Moving the joystick forwards or backwards will accept the current value and the electronics will change to drive mode.

E– Charge connector

The batteries in your wheelchair are recharged using a 3-pin charge connector at the front of your operating pod. The same connector is also used for programming your drive electronics with the PP1 programming device.

IMPORTANT– Modules for electronic adjustments and/or lights are only used on wheelchairs which are fitted with a light unit and/or electronic adjusters.

F– Light unit

Your operating pod has keys for lights, indicators (right, left) and hazard warning lights. Pressing the key switches the function on, pressing it again switches the function off.

G– Options for electronic adjustments

If your wheelchair has electronically adjustable options (leg rests, backrest, etc.), you can adjust these as follows: Press the MODE key, the speed display flashes. By pressing the MODE key again, the appropriate wheelchair symbol LED lights up, between the joystick and the battery displays. Moving the joystick forwards adjusts the selected electronic option, and by moving the joystick backwards, the adjustment is made in the reverse direction. If you have several electronic options on your wheelchair, then selection is made by moving the joystick to the left and/or right, and adjustment is made by pushing the joystick forwards or backwards. Pressing the MODE key takes you back to drive mode.

H–Joystick Boot

Make sure the boot is not torn or cracked (this could allow debris, water or moisture to enter). If the boot is torn or cracked, replace it as soon as you can.

Using the immobiliser with the Penny&Giles drive electronics

Connection:

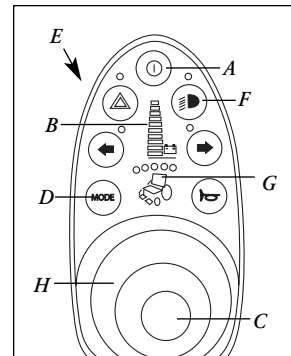
Connection is made via the recessed socket under the housing, with the plug of the joystick cable. When removing the console, please pull the plug out from under the operating console (only applicable for Pilot+).

Switching on the immobiliser:


With the wheelchair switched on, plug the 3-pin connector plug into the charge socket and then pull it out again. The operating display is extinguished. Your wheelchair is then protected against unauthorised use.

Switching the immobiliser off

Press the green on/off switch. The speed display will begin to flash. Now plug the 3-pin connector plug into the charge socket and then pull it out again. The normal operating display will appear on the operating console. Your wheelchair is then enabled.

Pilot+ Joystick

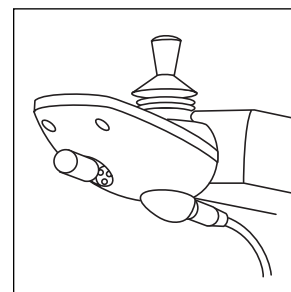Pilot+ Joystick

3. Safety when travelling

⚠ WARNING

If the speed is difficult to control, let go of the joystick, and the chair will stop.

- When you release the joystick, it returns on its own to the neutral position, the chair stops and the motor is blocked.
- We recommend that you switch the chair off, if you are stopping for a long time. This way you will save power.

⚠ WARNING

Switching off the chair after stopping does not affect the magnetic brakes.

4. Remote Switch Jacks (optional)

Two 1/8" switch jacks are optional to provide remote switch control of the on/off power function and the mode select function. Any of the single switches offered by Sunrise Medical can be used to connect to either or both of these jacks (except the single-zero touch switch).

⚠ WARNING

1. **Never use the ON/OFF switch to stop your chair except in an emergency. This will result in an abrupt stop, and may cause you to fall.**
2. **To slow or stop your chair, return the joystick to neutral.**

F. FREE WHEEL MECHANISM

Disconnect the free wheel mechanism when you need to manually push the chair. (For example, in an emergency, or if batteries fail).

⚠ WARNING

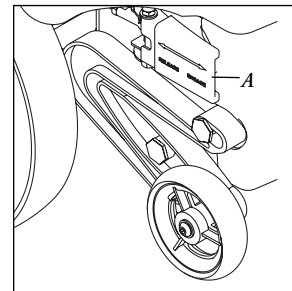
1. **Do not disengage the free wheel mechanism unless power to the chair is off.**
2. **With the brakes released, the chair will not operate and the battery charge indicator will flash 9 bars rapidly (if power to the chair is on).**
3. **Be aware that the chair will not have brakes in the free-wheel position.**
4. **Make sure that the person pushing the chair has full control when the free wheel mechanism is used.**

1. To disengage free wheel mechanism:

Lift both levers (A) outwards, to the free-wheel position.

2. To engage free wheel mechanism:

Push both levers (A) inwards, into the drive position.

XI. BATTERIES

A. INTRODUCTION

1. Notes –

- Batteries supply the power for your chair. They contain a finite amount of energy and have limits on how long they can store and supply energy.
- You can charge batteries only a certain number of times before they will fail and no longer hold a charge.
- For answers to questions about batteries, consult your supplier.

2. Use proper batteries:

Your chair operates on two 12 volt batteries.

- Size of the batteries LxWxH (L = 260 mm W = 170 mm, H = 240 mm)
- They should have a minimum of 65 ampere hour rating. Only deep cycle sealed case construction batteries should be used in this device.
- Connecting configuration. They should have a post and clamp style connection.
- When you buy a replacement, insist on a deep cycle sealed case type. Do not use a car starter battery.

3. Breaking in:

- A battery requires “breaking-in” for the first 6 to 12 charges. It will not accept a full charge for this period.
- It is best to limit the length of your trips until you break the batteries in and you know the range of your chair.

4. Discharged batteries:

- Never allow a battery to completely discharge. If you operate your wheelchair until it has almost stopped, you will greatly reduce the life of your batteries.
- Never let a battery sit in a discharged condition. Give unused or stored batteries a full charge once per month.
- Always fully charge the batteries. Avoid “topping off” with frequent charges.

WARNING

Never connect a life support or auxiliary device to a wheelchair battery. The electrical system may fail, and result in severe injury to or death of user.

B. BATTERY CHARGER

A battery charger produces a direct current (DC). When applied to a discharged battery, this reverses the chemical reaction that led to its discharge.

1. Charge rate. How fast a battery will charge depends on:

- Its electrical capacity; state of charge; electrolyte temperature, and internal condition of the battery.
- The maximum current of the charger.

2. Caution – to avoid damage to the charger:

- Never place the charger on top of a battery during charging. (Gases from the battery can damage the charger and may lead to an explosion or fire).
- Never place a battery on top of the charger.
- Never expose charger to damp, rain or snow.
- Never unplug charger by pulling on the electrical cord. This may damage the cord.
- Never open a charger or attempt to repair it yourself. Return charger to Sunrise Medical or have repairs made only by a qualified person.
- Keep cord out of the way, where it will not be stepped on, tripped over or damaged.

3. Caution – to avoid damage to the battery:

- Make sure you use the correct setting for sealed (gel) batteries (located on the off-board charger).
- Never charge a frozen battery. A fully charged battery will rarely freeze, but the fluid in a discharged battery can freeze at minus 9 degrees Centigrade. If you suspect a battery is frozen, thaw it before charging.

⚠ WARNING

Lead acid batteries generate explosive gas while charging. Completely read and follow all warnings about the batteries in this user instruction manual and any labels applied to the product. Failure to do so could result in fire, explosion, injury and/or death.

C. BATTERY ACID

The acid inside the batteries is corrosive. It can cause serious burns to the eyes and skin and can damage floors, furniture, clothing and your wheelchair.

⚠ WARNING

1. Do not puncture or attempt to open the batteries.
2. Do not use the batteries that are cracked or damaged.
3. If acid leaks from the battery and contacts your skin or clothing, wash immediately with soap and water.
4. If acid leaks from the battery and contacts your eyes, immediately flood eyes with cold running water for at least 15 minutes. Seek medical attention immediately.

D. CONNECTING BATTERIES IN BATTERY COMPARTMENT**⚠ WARNING**

1. Batteries weigh up to 25 kg. Take care to avoid injury when lifting.
 2. Keep batteries upright.
 3. Always wear rubber gloves and safety glasses when you handle batteries.
 4. Before working around batteries, remove all metal personal effects, such as necklaces, rings, watches, pins and other metal jewellery that might contact battery terminal and cause a short.
1. Lift batteries into battery compartment.

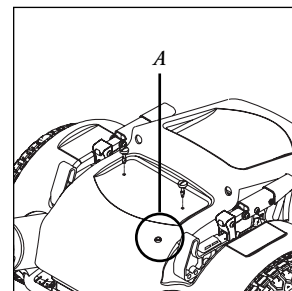
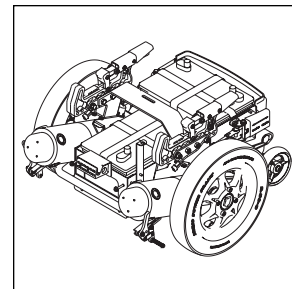
⚠ WARNING

Batteries weigh up to 55 pounds (25 kg). Lifting batteries may cause back strain. If you fail to heed these warnings severe injury may occur to you.

2. Connect and disconnect battery cables with caution. Connect the red lead to the positive terminal and the black lead to the negative terminal. Do this for each battery.
3. Tighten all terminal fasteners using two 1/2" (13 mm) box end wrenches to 60 in-lbs (6.8 Nm).
4. Place the battery compartment lid on power base and secure using the two (2) quarter turn fasteners (push down on fasteners).

⚠ WARNING

Never make direct contact across both battery terminals, as an explosion may occur. A spark may result in an explosion and/or fire, resulting in severe injury or death.

E. CHARGING BATTERIES

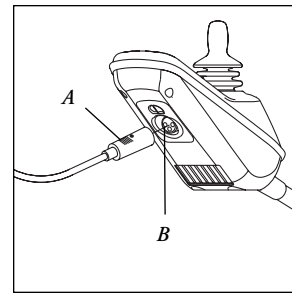
⚠ WARNING

To avoid the risk of severe injury or death from electrical shock, fire or explosion while charging:


1. Never connect charger to a source greater than 230 volts.
2. Connect charger to the batteries **BEFORE** connecting the charger to the power supply.
- Never connect or disconnect charger from battery while there is power to the charger. **CHECK CAREFULLY**
3. Never use an extension cord. Use of an improper cord could damage the charger, or cause a fire or electrical shock.
4. A battery emits hydrogen gas during charging. To reduce the risk of fire:
 - Make sure area is well-vented. Never charge battery in a closed-in area.
 - Never smoke or allow a spark, flame or high heat near battery during charging.
 - Never allow metal tools or chair parts to make direct contact across both battery terminals.

Always:

1. Use the charger that comes with your wheelchair. Read and follow all instructions and warnings.
2. Make sure room is well-ventilated.
3. Turn off all power to your chair.
4. Connect and disconnect battery cables with caution. Only connect the charger to the power supply after the charger has been connected to the chair.
5. Make sure to allow enough time to fully charge batteries.
Note: Batteries should never be left for long periods in the discharged state. Unused or stored batteries should be given a charge once per month.
6. Charge batteries by engaging charger connector (A) to the charger/programmer port (B) located on the joystick.

VSI

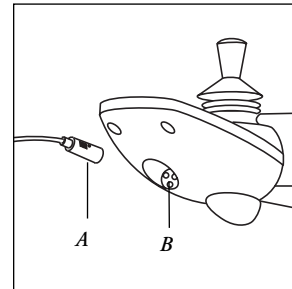Pilot+

F. DISPOSING OF BATTERIES

⚠ WARNING

1. All batteries, once they have reached the end of their useful life, are deemed to be hazardous waste.
2. For further information on handling and recycling, contact your local recycling authority.
3. Always dispose of product through a recognised agent.

XII. MAINTENANCE**A. NOTES**

1. Proper maintenance will improve performance and extend the useful life of your chair.
2. Clean your chair regularly. This will help you find loose or worn parts and will make your chair easier to use.
3. To protect your investment, have all major service and repair work done by your supplier.

⚠ WARNING

1. **Your chair needs regular maintenance for peak performance and to avoid injury from chair failure, damage or premature wear.**
2. **Inspect and maintain this chair strictly per the "Maintenance Chart" (See H).**
3. **If you detect a problem, make sure to service or repair the chair before use.**
4. **At least once a year, have a complete safety check and service of your chair performed by a supplier.**

B. CLEANING**1. Paint finish:**

- Clean the paint finish with mild soap water monthly.
- Protect the paint with a coat of non-abrasive auto wax every three months.

2. Motors:

- Clean around motor area weekly with a slightly damp (not wet) cloth.
- Wipe off or blow away any fluff, dust, or dirt on or around the motors.

NOTE– You do not need to grease or oil the chair.

3. Upholstery:

- Hand-wash only as needed. Machine washing may damage fabric.
- Drip-dry only. Heat from a dryer may damage fabric.

NOTE– Washing the fabric may decrease fire retardant properties.

C. STORAGE TIPS

1. Store your chair in a clean, dry area. If you fail to do so, parts may rust or corrode.
2. Before using your chair, make sure it is in proper working order. Inspect and service all items on the "Maintenance Chart" (See H).
3. If you store this chair for more than three months, have it inspected by a supplier before use.

D. BATTERY MAINTENANCE**⚠ WARNING**

1. Always keep batteries upright.
2. Never smoke or hold an open flame near batteries.
3. Always wear rubber gloves and safety glasses when you handle batteries.

1. Maintenance Schedule:

This varies for different types of batteries. Always follow the instructions supplied with your batteries.

NOTE– Always wear rubber gloves and safety glasses when you handle or service batteries.

2. Corrosion:

Check battery terminals often for corrosion.

- a. If corrosion is present, use a wire brush to clean the terminals.

NOTE– Always wear safety glasses and rubber gloves.

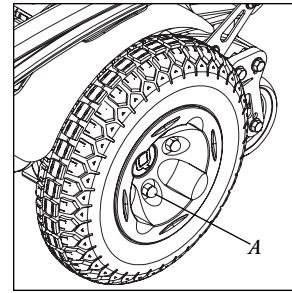
- b. Use baking soda to neutralise acid leaks.
- c. Use petroleum jelly to re-grease the terminals after connecting cable to battery. (Completely cover the terminal nut and bolt, cable clip and any exposed cable with jelly).

3. For answers to questions: Contact your supplier.

E. PNEUMATIC TYRES

⚠ WARNING

1. Do not use this chair if any of the tyres are under or over-inflated.
2. Low pressure in a tyre may cause the chair to veer to one side and result in loss of control.
3. An over-inflated tyre may burst.
4. Never use a petrol station pump to inflate a tyre. Such pumps provide air at high volume, and may cause tyre to burst.
1. Check for signs of wear and correct air pressure in pneumatic tyres weekly.
2. For best performance, inflate tyres to the same pressure level, as shown on the tyre sidewall:
 - 8" and 9" (200 and 250 mm) pneumatic casters – 30 to 40 psi (207 to 275 kpa)
 - 12" (300 mm) pneumatic drive wheels – 35 to 45 psi (240 to 310 kpa)
3. **CAUTION**– To prevent tyre damage:
 - Use a hand pump (or a low volume air pump) to inflate tyres.
 - Use a tyre gauge to check pressure.


F. TO REPAIR OR REPLACE A TYRE

⚠ WARNING

Residual air pressure in tyres can cause severe injury. Make sure you release all air in tyre before attempting to service or repair tyres.

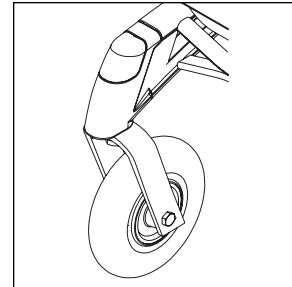1. **8" and 9" (200 and 250 mm) caster wheels:**
 - a. Release all air from tyre.
 - b. Remove caster wheel from fork using a 1/2" wrench.
 - c. Make sure there is no residual air in tyre before you remove bolts that hold the two rim halves together.
 - d. Remove four bolts from rim using a #3 Phillips screwdriver. Separate two halves of rim.
 - e. Repair or replace inner tube.
 - f. Reassemble the caster wheel; make sure the bearing spacer is in place and all four bolts are tightened to a torque of 60 in.-lbs. (6.8 Nm).
 - g. Mount caster wheel onto fork. Install washers correctly, tighten bolt to 240 in.-lbs. (27 Nm).
 - h. Inflate tyre to proper pressure level.
2. **12" (310 mm) drive wheels**
 - a. Elevate and securely support chair so that wheel is off the ground. Release all air from tyre.
 - b. Remove four (4) screws (A) from rim using a 9/16" socket wrench.
 - c. Make sure all air has been removed from tyre. Remove the four bolts from the rim. Separate two halves of rim.
 - d. Repair or replace inner tube. Reassemble the wheel. Use a torque setting of 120 in.-lbs. (13.5 Nm) to re-tighten bolts.
 - e. Inflate tyre to proper pressure level (listed on tyre sidewall).

G. ORDERING PARTS

When you order parts, provide the following:

1. Model of chair.
2. Serial number of chair.
3. Left hand or right hand control.
4. Part number, description and quantity of parts you need.
5. State reason for replacement.

NOTE– Replacement units are available on request by the dealer for repairing or changing. Please refer to the P220 parts manual.

H. MAINTENANCE CHART

You should check the items on this chart at the indicated intervals. If any of the items are loose, worn, bent or distorted, immediately have them checked and/or repaired by your authorised Sunrise Medical supplier. Frequent maintenance and servicing will improve performance, extend wheelchair life, and help prevent injuries.

CHECK...	Daily	Weekly	Monthly	Quarterly	Annually
Charge Batteries	✓				
Check tires for proper inflation level			✓		
Check batteries for proper electrolyte level (wet only)			✓		
Check plugs and connectors for proper connections				✓	
Check all moving parts for wear					✓
Inspect all nuts, bolts and fasteners for looseness or wear					✓
Inspect upholstery for wear					✓
Service by authorized supplier					✓


XIII. TECHNICAL SPECIFICATIONS

ISO 7176-15	Min	Max	Comments
Overall length (with legrest)	-	1160mm	
Overall width	-	665mm	Depends on seat width
Folded length	n.a.	n.a.	Not foldable
Folded height	n.a.	n.a.	Not foldable
Total mass	-	115 kg	
Mass of the heaviest part	-	23,5 kg	One Battery
Static stability downhill	-	25°	
Static stability uphill	-	12°	
Static stability sideways	-	20°	
Energy consumption (Max. range)	-	50 km	
Dynamic stability uphill	-	18% (10°)	
Obstacle climbing	-	100 mm	
Maximum speed forward	6 km/h	10 km/h	Two different versions
Minimum braking distance from max. speed	-	2000 mm	
Seat plane angle	-	-	adjustable
Effective seat depth	360 mm	51 mm	
Effective seat width	360 mm	510 mm	
Seat surface height at front edge	460 mm	510 mm	
Backrest angle	-4°	12°	adjustable
Backrest height	410mm	510 mm	
Footrest to seat distance	-	420 mm	adjustable
Leg to seat surface angle	-	115°	
Armrest to seat distance	180 mm	300 mm	adjustable
Front location of armrest structure	-	-	
Handrim diameter	n.a.	n.a.	Not a manual chair
Horizontal location of axle	n.a.	n.a.	Not a manual chair
Minimum turning radius	500 mm	-	
Mass of the test dummy		140 kg	
EN 12184	Min	Max	Comments
Maximum kerb height	-	100 mm	
Maximum height of obstacle		100 mm	
Turning space		1250 mm	
Speed control operation force	-	1,5 N	
Direction control operation force	-	1,5 N	
Pressure switches operation force (puff)	n.a.	n.a.	No special controls
Pressure switches operation force (sip)	n.a.	n.a.	No special controls


NOTE– For further Specifications to select the wheelchair a separate specificationsheet is available on request.

NOTE– Apart from the maintenance section XII in the manual, there is no service manual available.

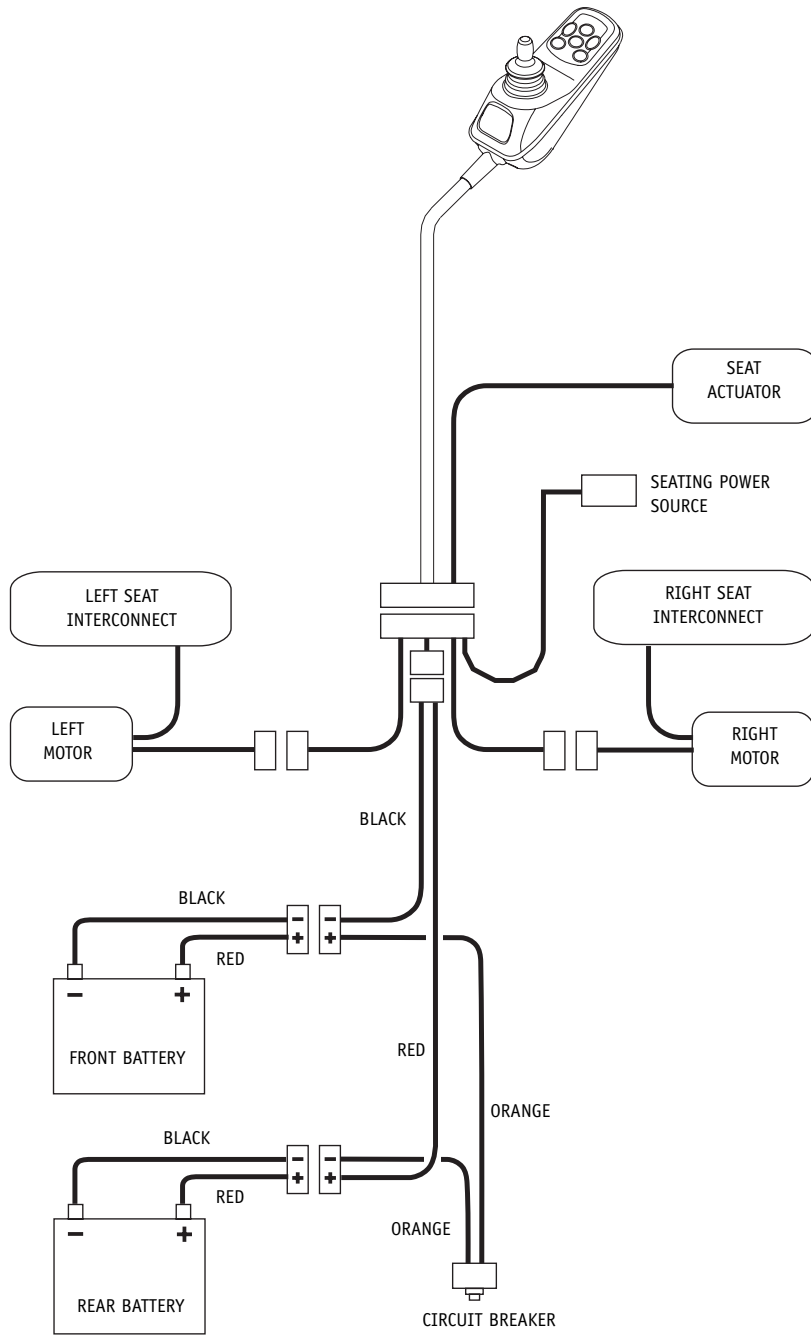XIV. WIRING DIAGRAMS

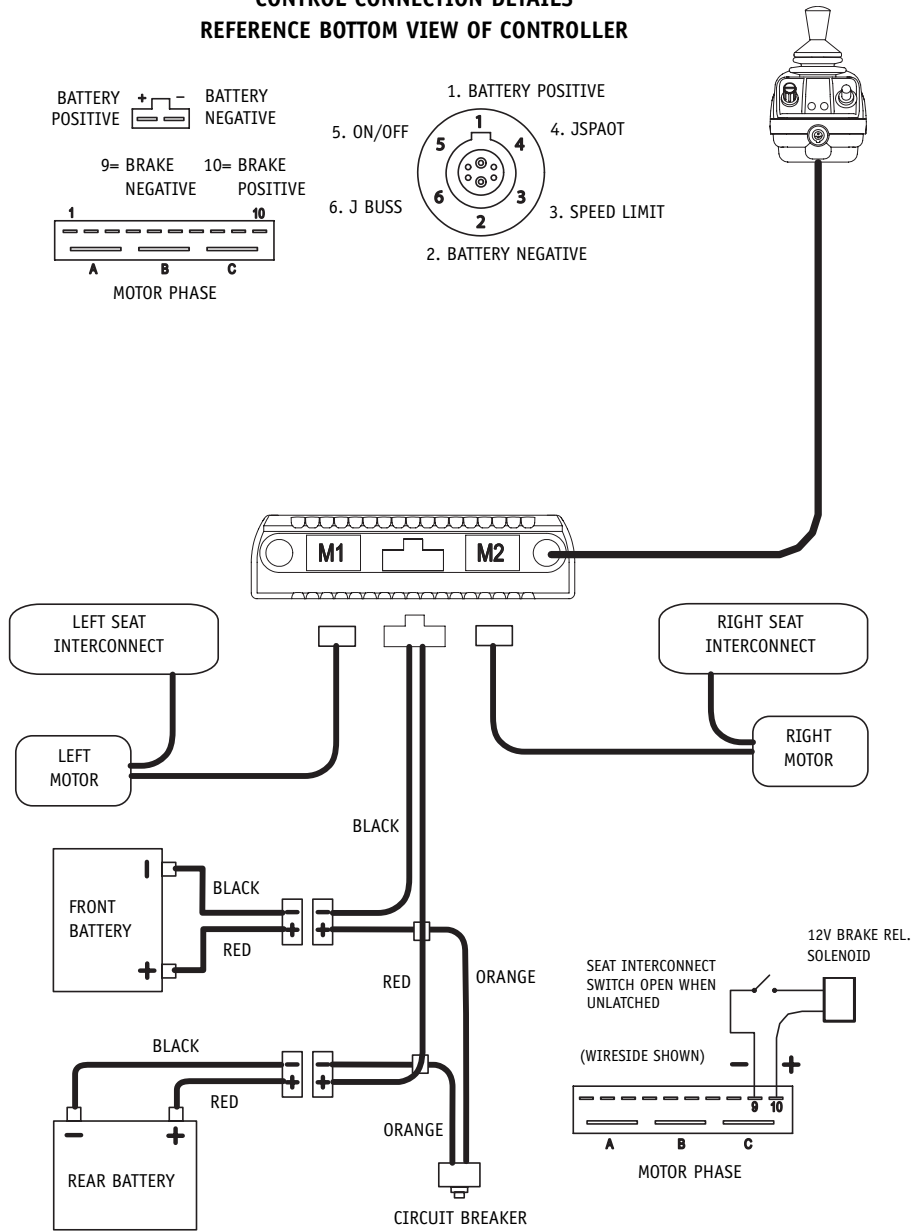CONTROL CONNECTION DETAILS REFERENCE BOTTOM VIEW OF CONTROLLER

XV. SETTINGS OF THE CONTROLLER

P220 SETTINGS OF THE CONTROLLER FOR EN 12184 TESTING AT TNO

Settings of the controller,

trademark: Penny+Giles, type: Pilot Plus, programme: PP1B V3.0

		MAX	MIN
DRIVE PROFILE			
Acceleration	acceler'n	25	
Deceleration	deceler'n	60	
Turn Acceleration	turn accel	35	
Turn Deceleration	turn decel	30	
Forward Speed	fwd sp	100	20
Reverse Speed	rev sp	40	15
Turning Speed	trn sp	35	15
Preset	preset prof 1	no	
Sleep Timer	sleep time	0m	
Joystick Throw	forward throw	100	
	reverse throw	100	
	right throw	100	
	left throw	100	
Steer Correct	steer correct	0	
Read Timer	elapsed	55h	
Fead Fault Log	1:code	3C00,#1	
Preset Unit	preset	no	
ENGINEER MENU			
Soft Reverse	soft rev	70%	
Current Limit	current	80A	70A
Temp Foldback	foldback	55°C	
Timed Foldback	stall time	5s	
	cl percent	50%	
Compensation	motor cmp.	65m_	
BATTERY MENU			
Battery Cable	cable res.	40m_	
Gauge Cal.	calibration	100	
Park Brake Trip	brake trip	yes	
Front Drive	front drive	yes	
Fast Brake Rate	fast brake	80	
Soft Stop	soft stop	yes	
Set Inhibit	active short	yes	
	latched stop	no	
Profiles	profiles	1	
Swap Motors	swap motors	no	
Brake Voltage	brake volts	12V	
Min Accel'n %	min accel	70%	
Min Decel'n %	min decel	65%	
Min Turn Acc %	min accel	75%	
Min Turn Dec %	min decel	80%	
Clear Timer	clear timer	no	
Erase Fault Log	clear log	no	
Preset Eng	preset eng	no	
BACK TO ROOT			

XVI. SUNRISE MEDICAL WARRANTY

1.

We warrant the main frame of this wheelchair against defects in materials and workmanship for the lifetime use of the first consumer purchaser.

2.

We warrant Sunrise-made components of this wheelchair against defects in materials and workmanship as follows:

- a. Main electronic controller – For two (2) years from the date of first consumer purchase.
- b. Motors, gear boxes and remote joystick – For one (1) year from the date of first consumer purchase.
- c. All other original components (such as wheels, upholstery, plastic, rubber parts and painted surfaces) – For three (3) months from the date of first consumer purchase.

3. LIMITATIONS

- a. We do not warrant damage due to:
 - Neglect, misuse or improper installation or repair.
 - Use of parts or changes not authorised by Sunrise Medical.
 - Exceeding the weight limit of 140 kg.
- b. This warranty is void if the original chair serial number tag is removed or altered.
- c. This warranty applies in the USA only. Check with your supplier to find out if international warranties apply.

4. WHAT WE WILL DO

Our sole liability is to repair or replace covered parts. This is your only remedy for consequential damages.

5. WHAT YOU MUST DO

- a. Return the warranty card.
- b. Obtain from us, while this warranty is in effect, prior approval for return or repair of covered parts.
- c. Return the wheelchair or part(s), freight pre-paid, to Sunrise Mobility Products Division at: 2842 Business Park Avenue, Fresno, CA 93727-1328.
- d. Pay the cost of labour to repair, remove or install parts.

6. NOTICE TO CONSUMER

There are no other express warranties. To the extent permitted by law, any implied warranty (including a warranty of merchantability or fitness for a particular purpose) is limited to:

- a. One (1) year from the first consumer purchase, and
- b. Repair or replacement of the defective part only.

This warranty gives you certain legal rights. You may also have other rights that vary from state to state.

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Sunrise Medical Srl
Sunrise Medical GmbH
Sunrise Medical B.V.