



BULLETIN DETAILS

Product Group:	Breezy	
Model Affected:	BasiX2, RubiX2	
Effective Date: See Below	Serial No. Effective From:	N/A
Bulletin Subject:	2021 BasiX2 & RubiX2 design updates	

At Sunrise Medical we are committed to our continuous improvement programme. Our new 2021 updates are focused on product performance which has been generated from customer feedback.

The BasiX2 and RubiX2 meet a broad range of clinical requirements, however we are introducing new updates to complement existing options. This benefits the client as it provides a wider choice to meet more clinical needs, resulting in an improved user experience. Therefore, we are pleased to announce that the following updates will be available from the **1st November**.

- NEW Half Folding Backrest Mechanism
- NEW Half Folding Backrest Angle
- NEW Seat to Floor Height Settings
- NEW Tall Backrest Option
- JAY SHAPE Backrest and JAY SHAPE Laterals
- Quickie Life Single Post Armrests
- NEW Anti-Tip Option
- NEW Anti-Tip Setting

Half Folding Backrest Mechanism – Available on BasiX2 and RubiX2

.....



From customer feedback we have found that folding down the backrest isn't always efficient, as the fixing strap gets caught in the mechanism not allowing it to be locked back up. We have removed the fixing straps to reduce the difficulty.

Benefits

- Removed fixing straps on the backrest improves folding and unfolding - this also increases the durability of the backrest.

Half Folding Backrest Angle – Available on BasiX2 and RubiX2



Customers have reported that the current backrest angle doesn't allow for full immersion, meaning clients cannot experience the full benefit of the tension adjustable straps. We have now changed this from an 8 degree angle to 13 degrees

Benefits

- Backrest angle has been changed to 13 degrees which allows for further immersion into the upholstery. Improving comfort and reducing forward sliding.

New Seat to Floor Height 48cm – Available on BasiX2 and RubiX2



We are introducing a 48cm seat height when using 12" transit option.

Benefits

- Higher seat height allows taller users to utilise the wheelchair.
- No compromise on lower leg length.
- Will benefit taller standing attendants.

Tall Backrest Option (50cm) – Available on RubiX2



A taller backrest option of 50cm will be available alongside the standard half folding back.

Benefits

- Tall backrest option is suitable for taller users and users with higher backrest cushions.
- More support and increased comfort for taller users.

JAY Shape Backrest and Jay Shape Laterals – Available on RubiX2



Our popular JAY Shape backrest and JAY Shape lateral supports will be available on the RubiX2.

Benefits

- More lateral options to cater for a wider range of client needs.
- JAY shape laterals additionally support the user's posture as it promotes positive back alignment.
- Improved comfort from the JAY Shape backrest as it allows for higher immersion.
- JAY shape backrest and laterals provides support to the shoulders when self-propelling.

Single Post Armrests – Available on RubiX2



RubiX2 will receive the Quickie Life single post armrest. The lightweight height adjustable single post provides good arm support with less weight added to the wheelchair.

Benefits

- The single post armrest is stronger and more durable for clients who transfer vs the previous version

New Anti-Tip– Available on RubiX2



A new anti-tip option provides an integrated tip assist and anti-tip all in one; removing the compromise of having to choose one or the other

Benefits

- Easier for carers and attendants
- Access to the benefits of both options

New Anti-Tip Setting – Available on BasiX2 & RubiX2



The standard setting for the anti-tip has always been in the second receiver. From customer feedback we found there was difficulty getting over kerbs and thresholds. Therefore we have adjusted the standard setting so it is now one hole higher.

Benefits

- Improves the clearance over obstacles which supports an easier user experience.
- No need to manually change the setting.
- No longer need to manually flip away the anti-tips.

Next Steps

All of our Area Sales Managers have demonstration units containing these new options, so please contact them for any assessment support or if you would like to evaluate this product.

For any queries on any of these updates please contact your local Area Sales Manager.

Customer Services	Customer Support	Technical Service Centre	E-mail
Telephone:	+44 (0)845 605 66 88 Option 1-1	+44 (0)845 605 66 88 Option 1-2	enquiries@sunmed.co.uk
Fax:	+44 (0)845 605 66 89	+44 (0)845 605 66 89	help.technical@sunmed.co.uk