



ZIPPIE Q300M Mini Kids / ZIPPIE Q300M Mini Teens

Tilbehør- og
reservedelskatalog
(En)



This product is manufactured to comply with the Medical Device Directive 93/42/EEC.

Use of Illustrations & Text:

The illustrations and accompanying text set out in this document are the property of Sunrise Medical Limited and must not be copied or used for manufacturing, tendering or any other purpose without written consent from Sunrise Medical.

Ordering:

Item numbers shown as N/A are for reference purposes only and should not be used when ordering replacement parts. Always quote the correct serial number and part number.

Quality Management Systems:

Sunrise Medical Limited is accredited to ISO 9001: 2008, which is the internationally recognised standard for quality management systems, as well as ISO 13485: 2012, which specially governs Medical Devices. These approvals ensure that we provide quality in all areas of our business from development through to final delivery.

Parts Disclaimer:

It is potentially hazardous to fit or use any parts other than genuine Sunrise Medical parts. The company disclaims all liability for the consequences of such use, which in addition voids the product warranty.

This product is for the carriage of one person and must not under any circumstances be used for any other purpose.

Queries:

Should you have any queries then please contact Sunrise Medical Ltd on

Tel +44(0) 845 605 6688
Fax +44(0) 845 605 6689

Table of contents

ZIPPIE Q300M Mini Kids / ZIPPIE Q300M Mini Teens	6
Armrest and sideguards	7
Armrest flip back, ZIPPIE	8
Armrest flip back, TEENS	9
Armpads	10
Sideguards	12
Legrest	13
Centre mount legrest, fixed	14
Life legrest	15
Life legrest, high mount	17
ZIPPIE 90° legrest	19
ZIPPIE manual elevating legrest	21
TEENS manual elevating legrest	23
TEENS power elevating legrest	25
TEENS power articulating legrest	27
UNI 80° legrest	29
Headrest	31
Headrest pads	32
Headrest mount	33
Product and safety labels	35
Safety labels	36
Q300 M - Product labels	38
Backrest	41
Backcushion, Holder	42

Table of contents

Backrest standard	43
Backrest power, adjustable	45
Tension back upholstery	46
Back cushion, deep contour	47
Lateral thoracic supports	48
Seat	49
Seat module, lift and tilt 30°	50
Seat plate	52
Seat module, fixed	53
Seat module, tilt	54
Seat cushion	55
Q300 M - Seat interface	56
Control mount	57
Mount, 4-actuator box VR2	58
Mount, joystick VR2/R-Net	59
VR2 / R-Net attendant control and mounting	60
Control input device	62
Joystick knobs	63
VR2 joysticks and 4-actuator box	65
R-Net joysticks	66
OMNI 2	68
Accessories	70
Charger	71
Q300 M - Batteries - (only for Q300 M Mini)	73

Table of contents

Lap belts	74
Rear view mirror	75
Tray table, swing away	76
Shrouds	78
Q300 M - Battery cover	79
Coloured shroud elements	81
Drive wheel / castor wheel	82
Q 200 R - 9 Inch Castor Wheel	83
Q300 M - 13 inch drive wheel, knobbly	85
14 inch drive wheel, knobbly	86
12 inch drive wheel, knobbly	88
Q300 M - 7 inch castor wheel	89
Chassis	90
Q300 M - Chassis	91
Looms	92
Looms and electronic	93
Battery looms	94
R-Net bus cable	95
Actuator looms	97
Inhibit looms	99
Light looms	100
Lights	101
Q300 M - LED lights and indicators (StVZO)	102

Table of contents

Motor	103
Motor 2 - pole standard	104
Q300 M - Motor 4 - pole	105
Q300 M - Motor 2 - pole performance	107
Control	109
R-Net electronic components	110
R-Net control modules	111
VR2 control modules	112
Dongle	113

ZIPPIE Q300M Mini Kids / ZIPPIE Q300M Mini Teens

Spare parts


Description
Armrest and sideguards
Legrest
Headrest
Product and safety labels
Backrest
Seat
Control mount
Control input device
Accessories
Shrouds
Drive wheel / castor wheel
Chassis
Looms
Lights
Motor
Control

Armrest and sideguards

Spare parts

Description
Armrest flip back, ZIPPIE
Armrest flip back, TEENS
Armpads
Sideguards


Armrest flip back, ZIPPIE

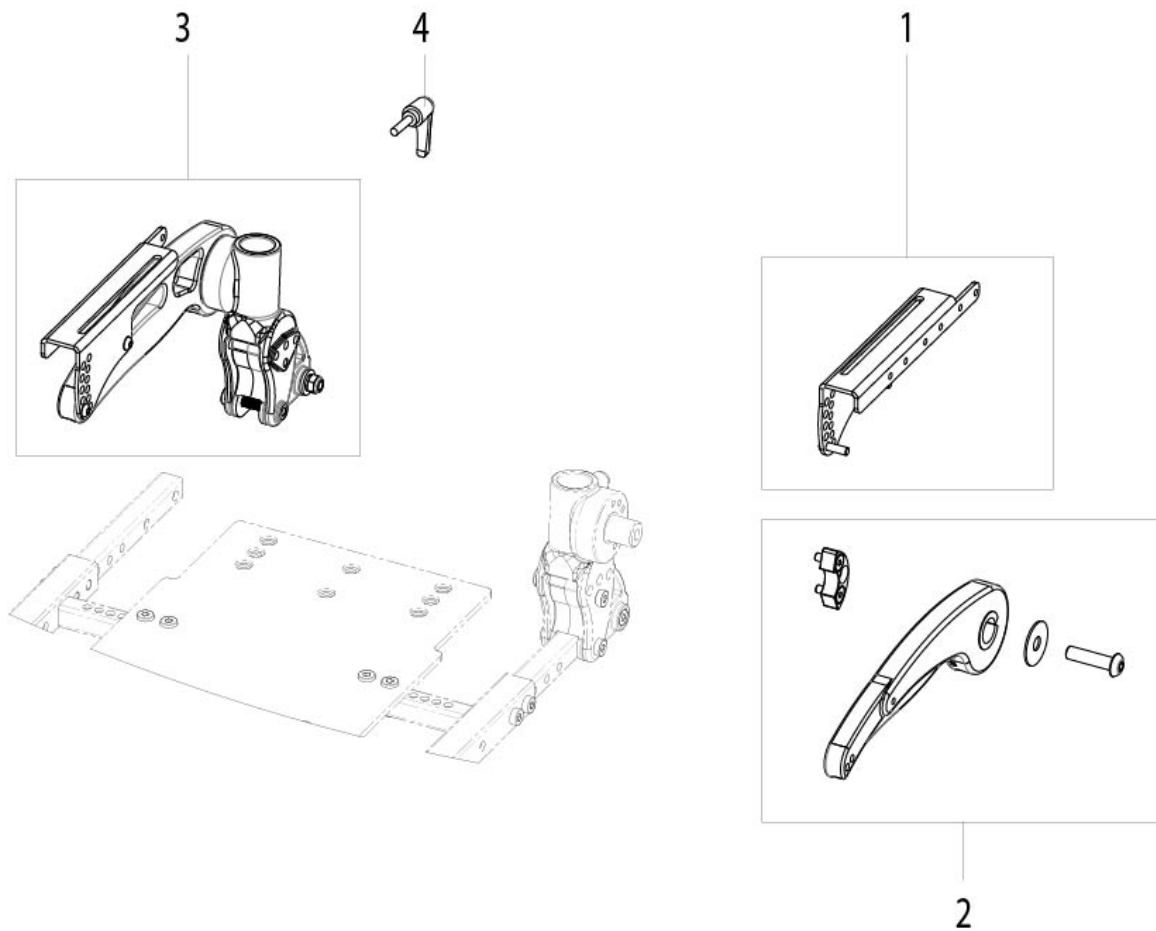

Spare parts

Item no.	Article no.	Description	Remark
4	000102021-ET	Clamping lever M6 x 25 GN503	2 pcs./pack
	035104	Armrest, cpl.	left
3	035105	Armrest assembly, cpl.	right
1	21504040	Flip back armpad bracket	right
	21504041	Flip back armpad bracket	left
2	21504042	Flip back armrest cast	right
	21504043	Flip back armrest cast	left


Armrest flip back, TEENS

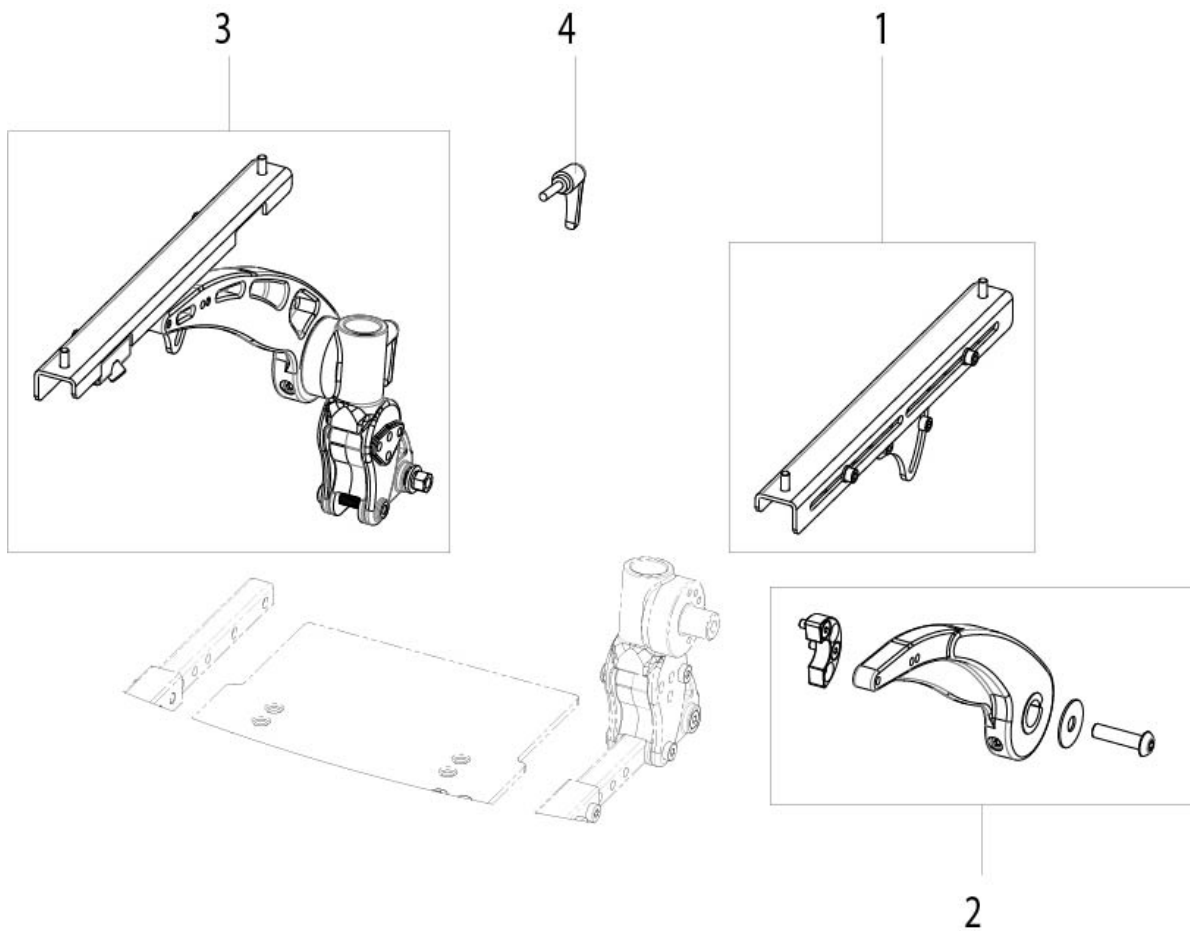

Spare parts

Item no.	Article no.	Description	Remark
4	000102021-ET	Clamping lever M6 x 25 GN503	2 pcs./pack
	060844	Armrest assembly	left
3	060845	Armrest assembly	right
1	20104045	Flip back arm bracket	right
	20104046	Flip back arm bracket	left
2	20104047	Side Part for flip back armrest	right, note: without bottom pivot bracket
	20104048	Flip back arm	left, Note: without bottom pivot bracket

Armpads

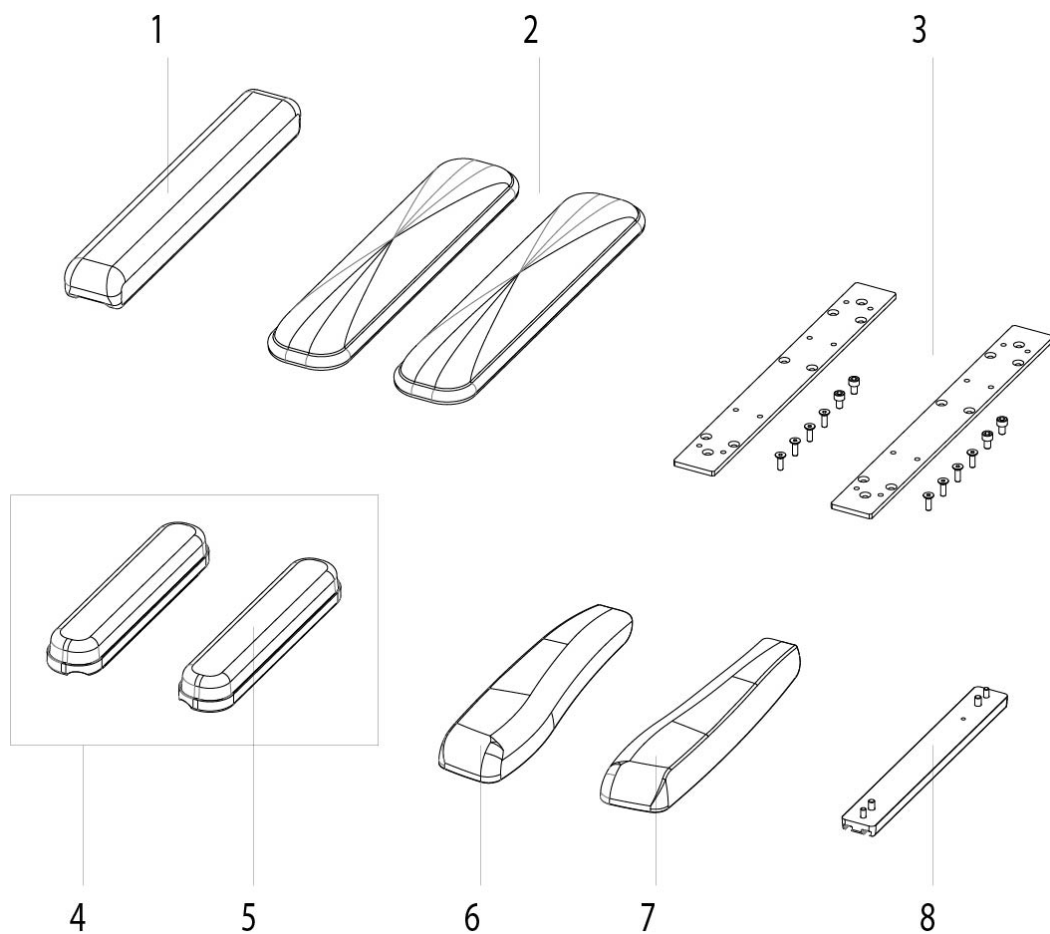


Spare parts

Item no.	Article no.	Description	Remark
5	014475	Armrest pad short, 250 mm incl. mounting accessories, cpl.	left/right useable
1	015071	Standard armpad	1 pc., left/right usable
4	035043	Armrest pad short, 250 mm, incl. receiver and mounting accessories, cpl.	pair
3	20104050	Interface plate for all armpads, inclusive mounting material	Pair
7	22504218-003	Armpad assembly 300 mm, inclusive mounting accessories	left
6	22504218-004	Armpad assembly 300 mm, inclusive mounting accessories	right
8	22804051	Arm pad profile, incl. mounting accessories	used with arm pad 300,400 und 450 mm and arm pad UMO

Armpads

Item no.	Article no.	Description	Remark
2	511.100G	Standard skai armpad, length 400 x 80 mm / 15.7 x 3.1 inch	pair

Sideguards

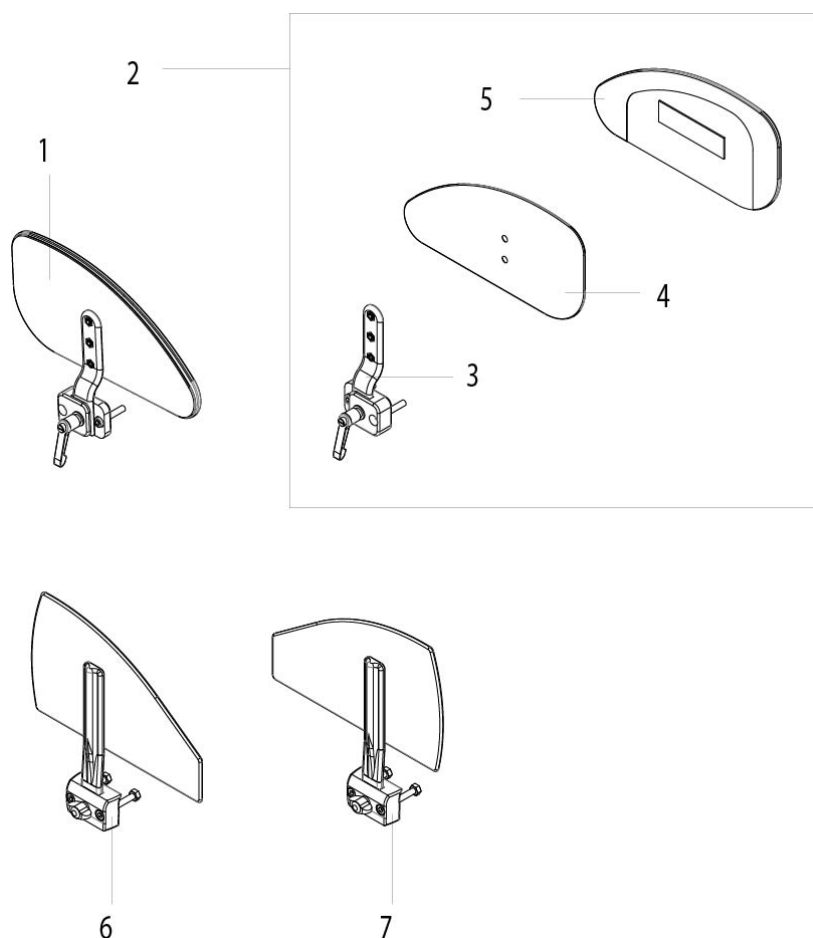

Spare parts


Item no.	Article no.	Description	Remark
4	010980	Skirt guard cover - small	right
	010981	Skirt guard cover - small	left
	014656	Side guard pad - small	right
5	014657	Side guard pad - small	left
3	035106	Mounting accessories used with skirt guard	
1,2	035107-000	Skirt guard cpl. - small	pair
1	035107-001	Skirt guard cpl. - small	left
2	035107-002	Skirt guard cpl. - small	right
6	21504045-001	Side guard tall, incl. mounting material	left
7	21504045-002	Side guard tall, incl. mounting material	right

Legrest

Spare parts

Description
Centre mount legrest, fixed
Life legrest
Life legrest, high mount
ZIPPIE 90° legrest
ZIPPIE manual elevating legrest
TEENS manual elevating legrest
TEENS power elevating legrest
TEENS power articulating legrest
UNI 80° legrest


Centre mount legrest, fixed

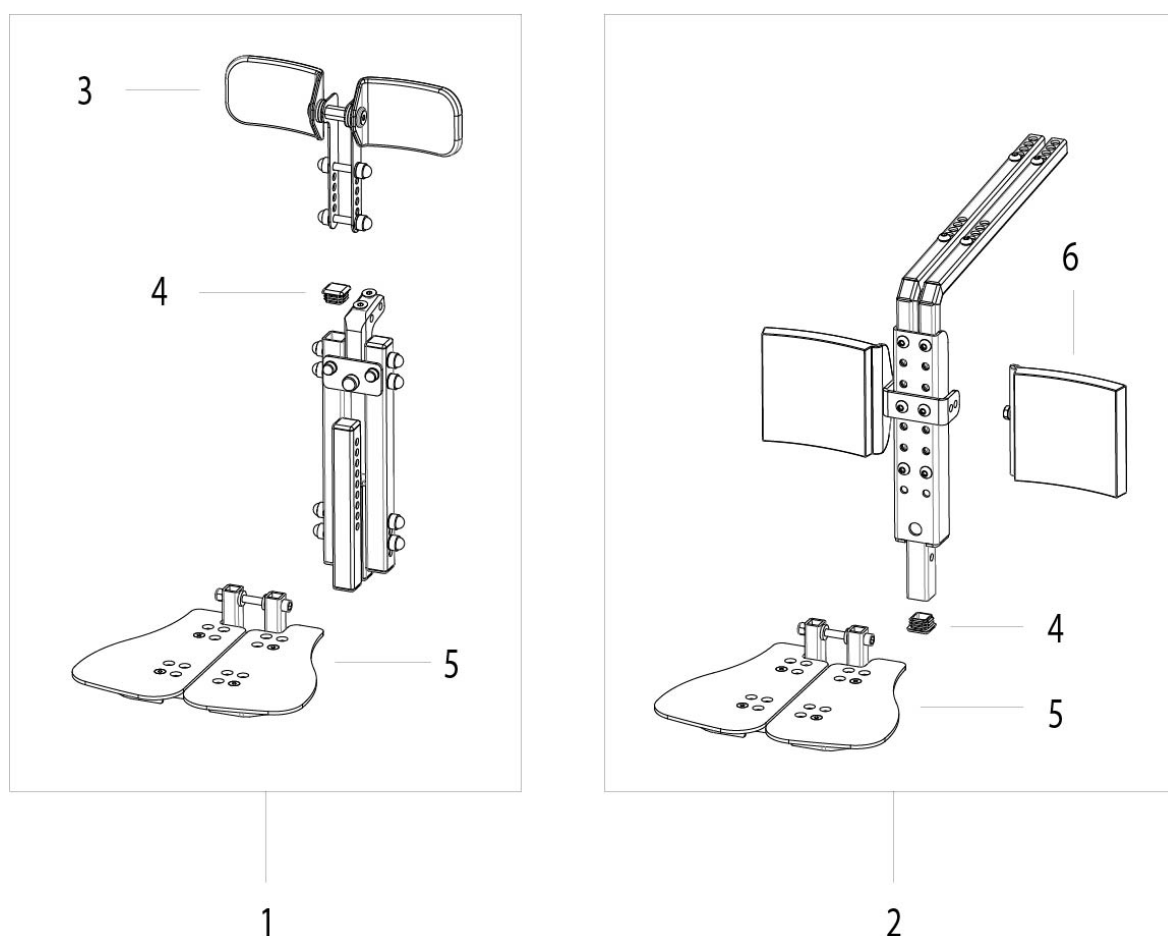

Spare parts

Item no.	Article no.	Description	Remark
4	010615	End cap, square	
3	035140	Calf pad centre mount legrest	
6	103118	Calf pad	
5	21505608	QUICKIE footplate	
2	21505609-001	Center mount hanger assembly, medium	370 - 470 mm (14.5 - 18.5 inch) lower leg length
2	21505609-002	Center mount hanger assembly, short	290 - 370 mm (11.5 - 14.5 inch) lower leg length
1	21505610-000	Centre mount legrest assembly	

Life legrest

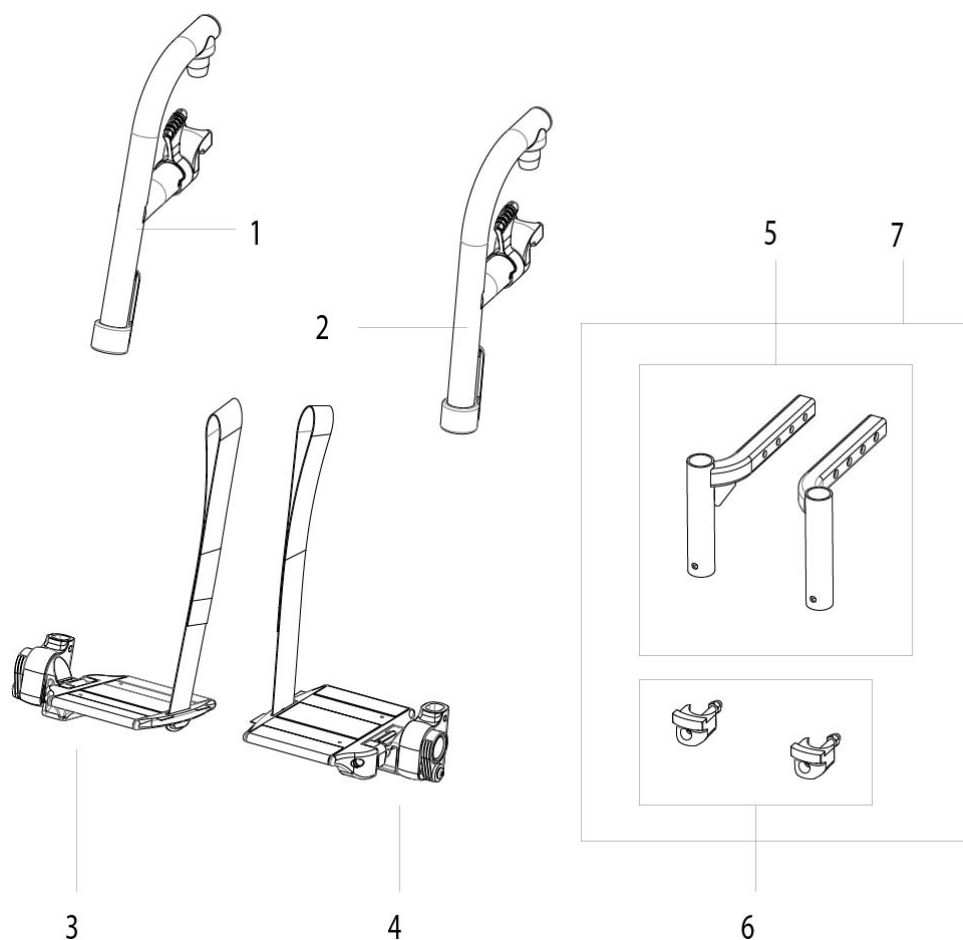


Spare parts

Item no.	Article no.	Description	Remark
6	064925	Hanger boss complete	2 pcs./pack, Note: used with PariX ² manufactured starting August 2019.
1	072244-001	Hanger Alu 70 Degree cpl. Black (9005) (1 Side)	
2	072244-002	Hanger Alu 80 Degree cpl. Black (9005) (1 Side)	
4	21505695-001	Footplate divided, 86 mm, for SW left	300-320 mm
4	21505695-002	Footplate divided, 106 mm, for SW left	340-360 mm
4	21505695-003	Footplate divided, 116 mm, for SW left	380-400 mm
3	21505696-001	Footplate divided, 86 mm, for SW right	

Life legrest

Item no.	Article no.	Description	Remark
		300-320 mm	
3	21505696-002	Footplate divided, 106 mm, for SW 340-360 mm	right
3	21505696-003	Footplate divided, 116 mm, for SW 380-400 mm	right
5	22605061	Receiver Basix Footrest	
7	22605063	Footrest receiver incl. hanger boss	pair

Life legrest, high mount

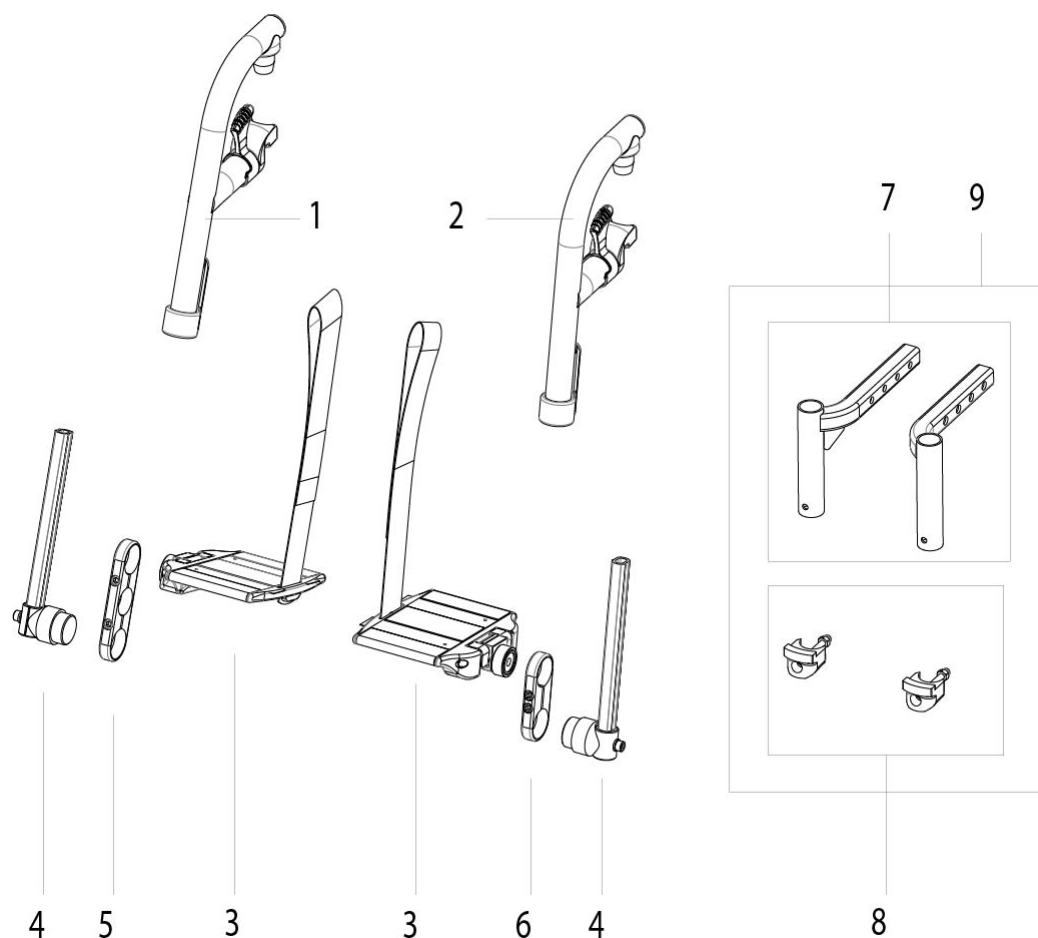


Spare parts

Item no.	Article no.	Description	Remark
8	064925	Hanger boss complete	2 pcs./pack, Note: used with PariX ² manufactured starting August 2019.
1	072244-001	Hanger Alu 70 Degree cpl. Black (9005) (1 Side)	
2	072244-002	Hanger Alu 80 Degree cpl. Black (9005) (1 Side)	
4	21505697	Extension tube legrest high mount	left/right useable
3	21505698-001	Footplate divided, high mount 76 mm, for SW 300-320 mm	left/right useable
3	21505698-002	Footplate divided, high mount 86 mm, for SW 340-360 mm	left/right useable
3	21505698-003	Footplate divided, high mount	left/right useable

Life legrest, high mount

Item no.	Article no.	Description	Remark
		106 mm, for SW 360-400 mm	
7	22605061	Receiver Basix Footrest	
9	22605063	Footrest receiver incl. hanger boss	pair
5	840825002	Clamp tube with fittings, complete, long variant (121 mm)	Long, 121 mm
6	840825005	Clamp tube with fittings, complete, short variant (90 mm)	Short, 90 mm

ZIPPIE 90° legrest

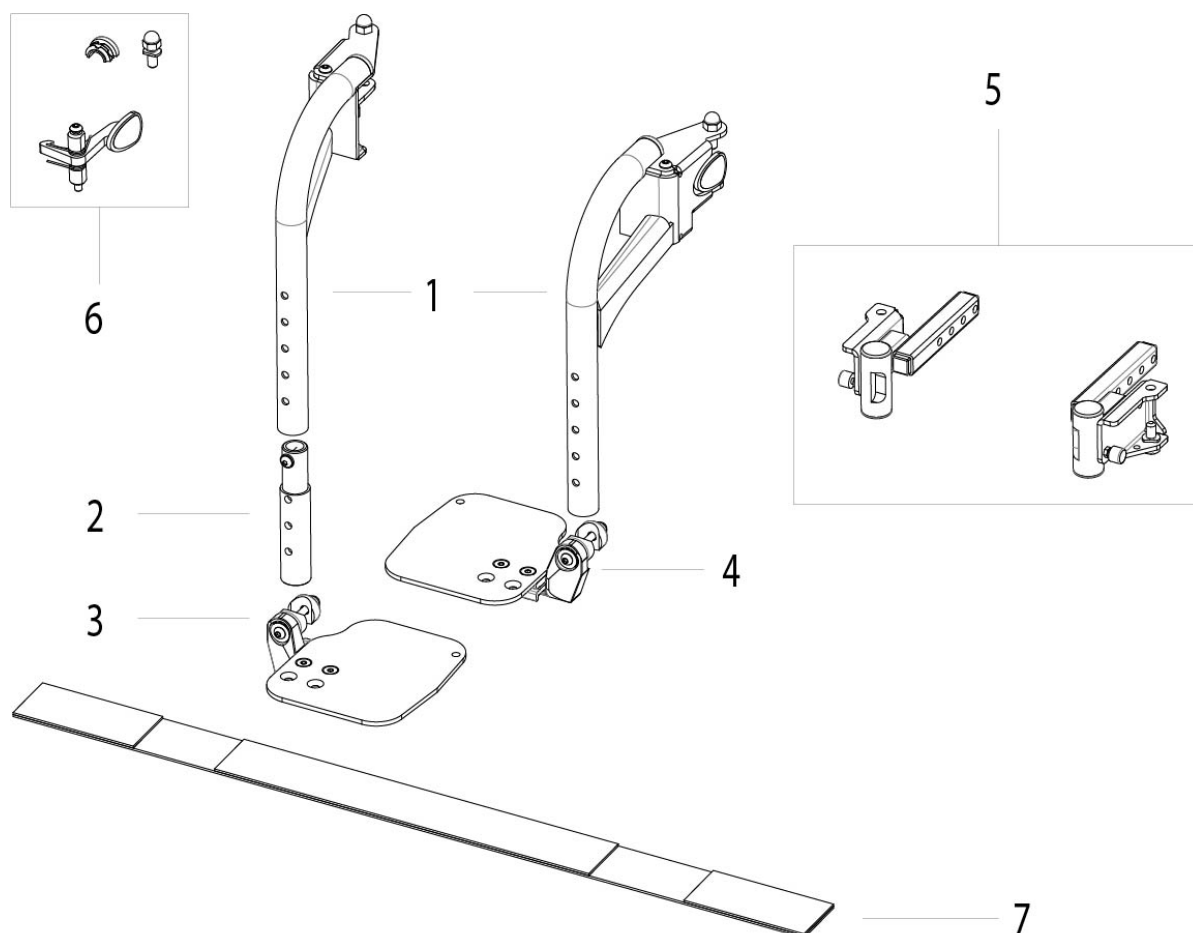


Spare parts

Item no.	Article no.	Description	Remark
6	013451	Latch kit, swing away	black
5	035024-ET	Legrest swing away retainer assembly	pair
1	035090	90° swing away hanger matt black 9105	pair
2	035134	90° swing away hanger extension tube	
3	22005010-001	Adjustable flip up footplate - 90 deg legrest	right, SW 300-340 mm
3	22005010-002	Adjustable flip up footplate - 90 deg legrest	right, SW 360-400 mm
4	22005010-011	Adjustable flip up footplate - 90 deg legrest	left, SW 300-340 mm
4	22005010-012	Adjustable flip up footplate - 90	left, SW 360-400 mm

ZIPPIE 90° legrest

Item no.	Article no.	Description	Remark
		deg legrest	
7	740130	Leg strap SW 300 - 360 mm / 10 - 14 inch	
7	740131	Legstrap standard 380 - 430 mm (15 - 17 inch)	

ZIPPIE manual elevating legrest

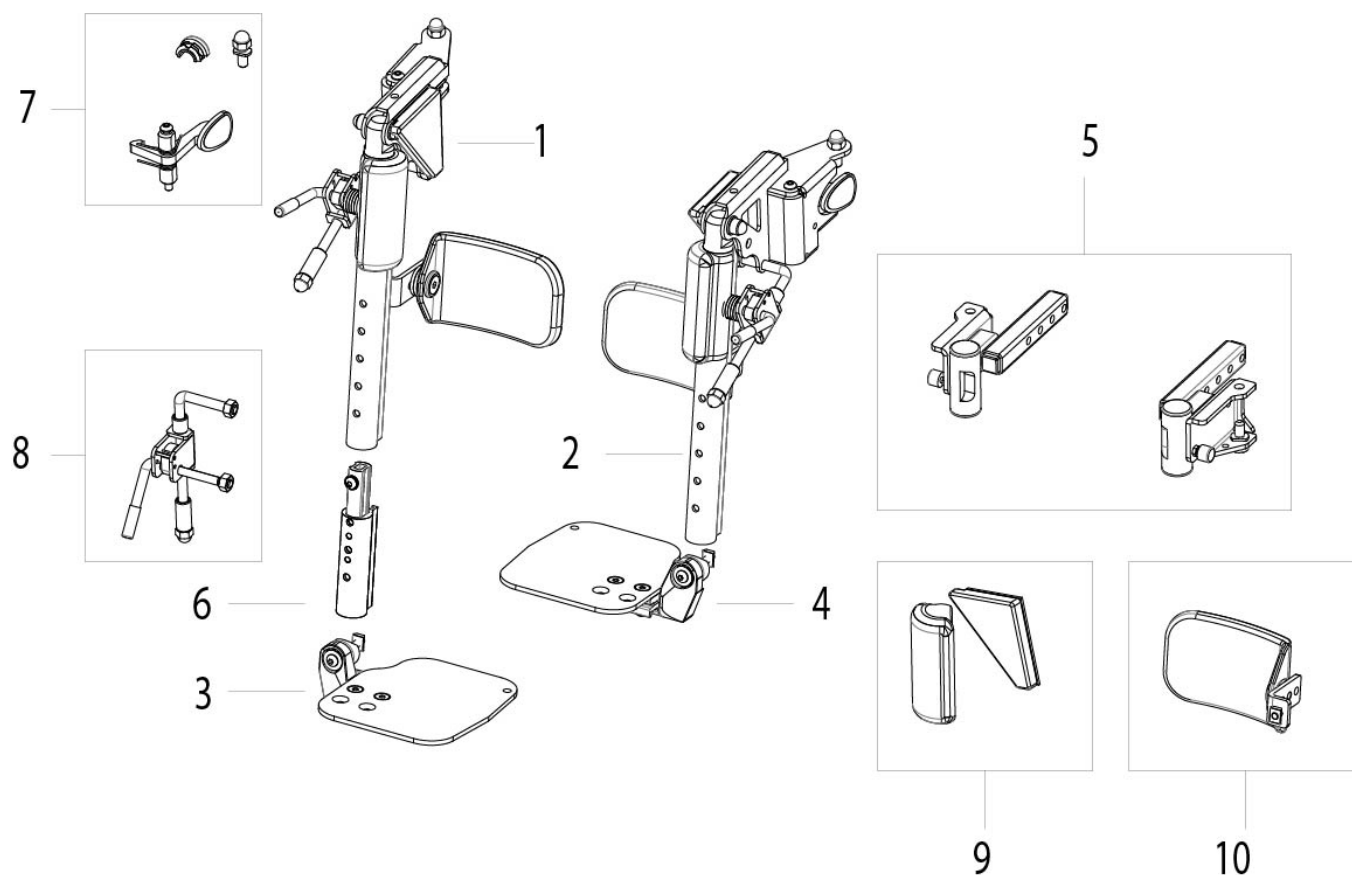


Spare parts

Item no.	Article no.	Description	Remark
7	013451	Latch kit, swing away	black
5	035024-ET	Legrest swing away retainer assembly	pair
1	035124-001	ELR Kids assembly	right
2	035124-002	ELR Kids assembly	left
6	035132-ET	Elevating legrest extension tube	2 pcs./pack
10	035141	ELR calf pad	left/right useable
9	035143	ELR pads for kids ELR	left
9	035144	ELR pads for kids ELR	right
8	118040	ELR lock rod assembly	right
8	118041	ELR lock rod assembly	left
3	22005010-101	Adjustable flip up footplate - ELR	right, SW 300-340 mm
3	22005010-102	Adjustable flip up footplate - ELR	right, SW 360-400 mm

ZIPPIE manual elevating legrest

Item no.	Article no.	Description	Remark
4	22005010-111	Adjustable flip up footplate - ELR	left, SW 300-340 mm
4	22005010-112	Adjustable flip up footplate - ELR	left, SW 360-400 mm

TEENS manual elevating legrest

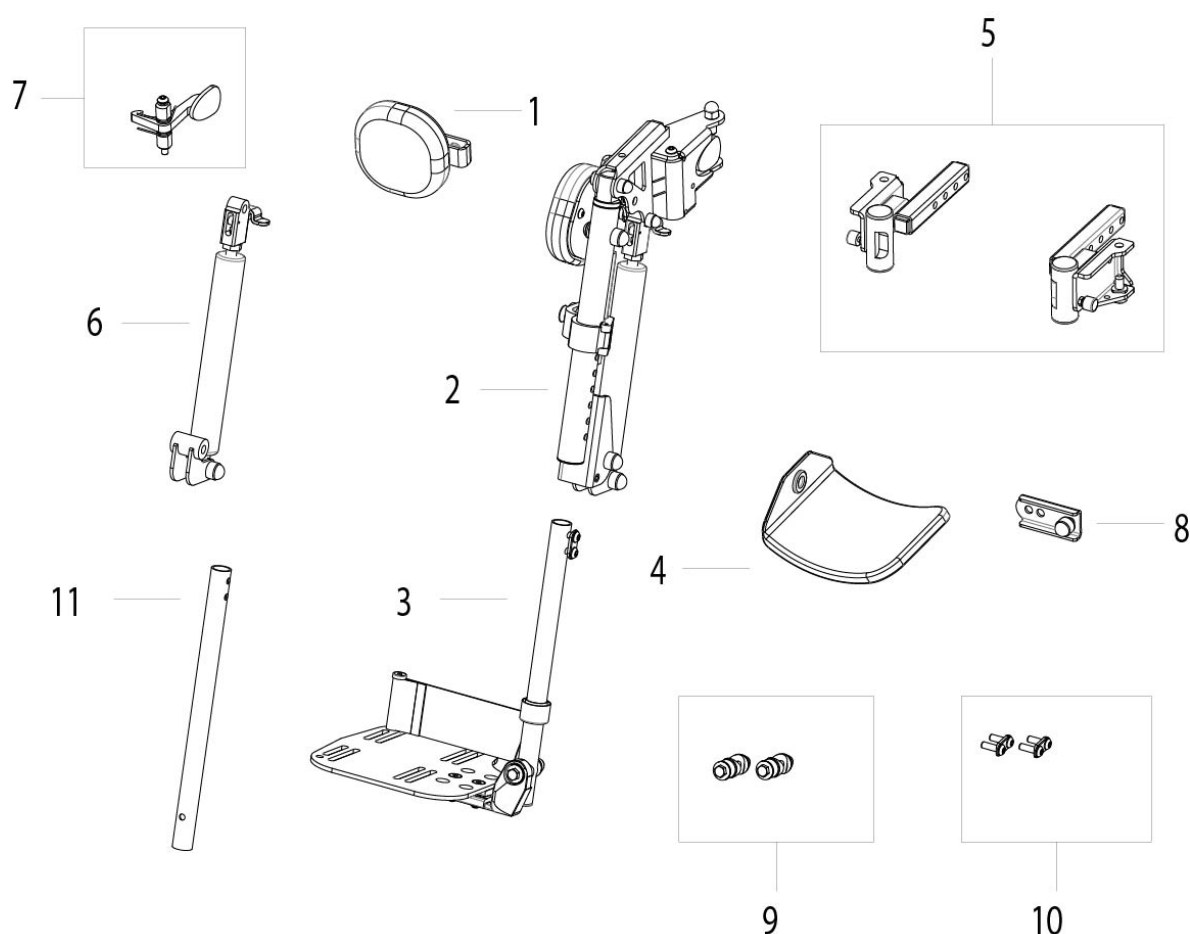


Spare parts

Item no.	Article no.	Description	Remark
7	013451	Latch kit, swing away	black
2	015271	Manual elevating legrest, cpl.	left
2	015279	Manual elevating legrest, cpl.	right
6	015338	Inline ELR manual gas strut assembly	cpl.
8	015727	Extension kit calf pad	Note: for ALR and ELR
5	035024-ET	Legrest swing away retainer assembly	pair
11	060209	Extension tube	
3	060522	Angle adjustable footplate assembly, inclusive heel loop and extension tube	right
3	060523	Angle adjustable footplate	left

TEENS manual elevating legrest

Item no.	Article no.	Description	Remark
		assembly, inclusive heel loop and extension tube	
9	21005130	Footplate washer kit	2 pcs./pack
10	21005132	Fixing kit footrest including 2 saddle washers and 4 screws	
3	21505700-001	Footplate, angle adjustable, for SW 300-320 mm	left
3	21505700-002	Footplate, angle adjustable, for SW 340-360 mm	left
3	21505701-001	Footplate, angle adjustable, for SW 300-320 mm	right
3	21505701-002	Footplate, angle adjustable, for SW 340-360 mm	right
1	524001.32100	Knee pad assembly	
4	755013	Calf pad cpl. seat widths 12 - 13 inch	
	755014	Calf pad cpl. 14 - 16 inch	

TEENS power elevating legrest

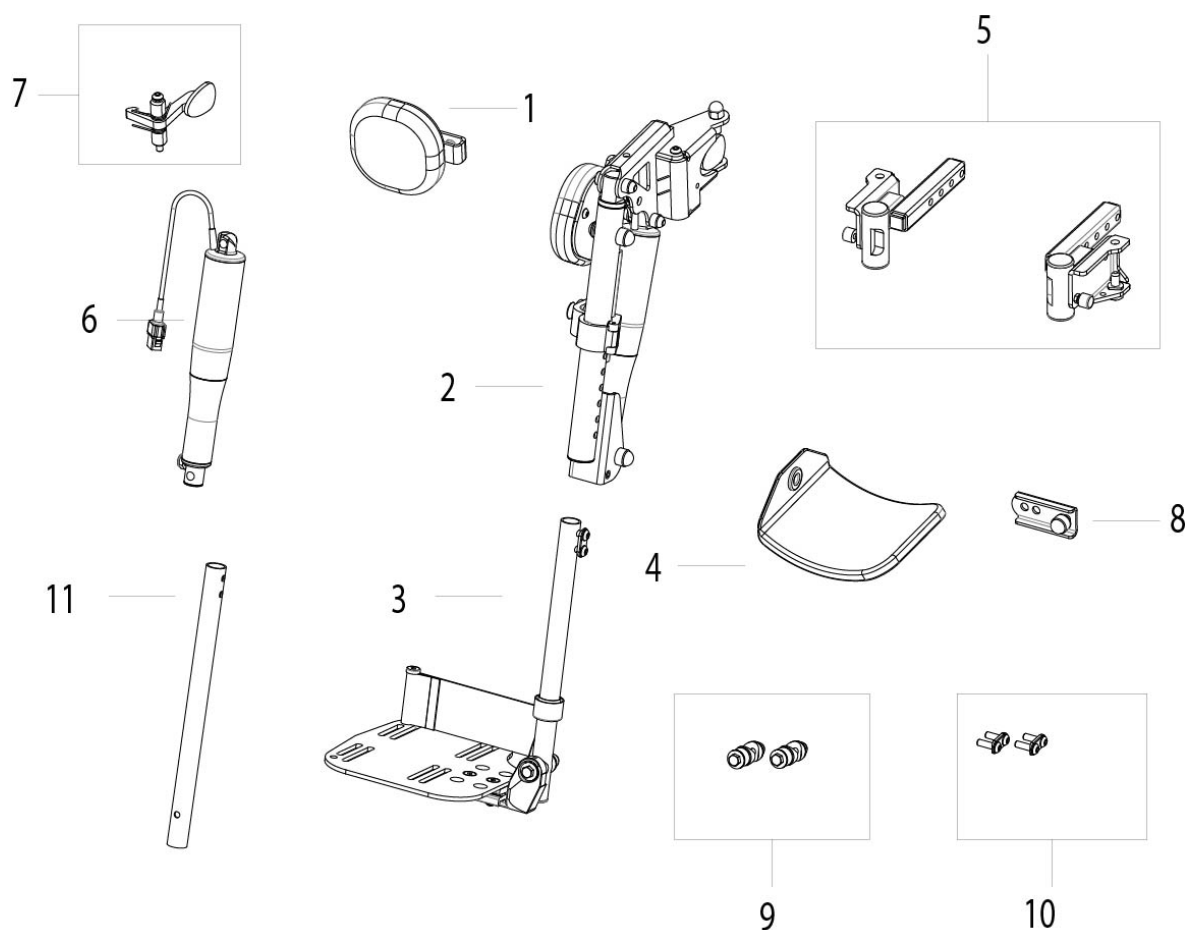


Spare parts

Item no.	Article no.	Description	Remark
7	013451	Latch kit, swing away	black
2	015255	Complete assembly	left
2	015256	Complete assembly	right
8	015727	Extension kit calf pad	Note: for ALR and ELR
5	035024-ET	Legrest swing away retainer assembly	pair
11	060209	Extension tube	
3	060522	Angle adjustable footplate assembly, inclusive heel loop and extension tube	right
3	060523	Angle adjustable footplate assembly, inclusive heel loop and extension tube	left

TEENS power elevating legrest

Item no.	Article no.	Description	Remark
6	20010011	Actuator	
9	21005130	Footplate washer kit	2 pcs./pack
10	21005132	Fixing kit footrest including 2 saddle washers and 4 screws	
3	21505700-001	Footplate, angle adjustable, for SW 300-320 mm	left
3	21505700-002	Footplate, angle adjustable, for SW 340-360 mm	left
3	21505701-001	Footplate, angle adjustable, for SW 300-320 mm	right
3	21505701-002	Footplate, angle adjustable, for SW 340-360 mm	right
1	524001.32100	Knee pad assembly	
4	755013	Calf pad cpl. seat widths 12 - 13 inch	
4	755014	Calf pad cpl. 14 - 16 inch	

TEENS power articulating legrest

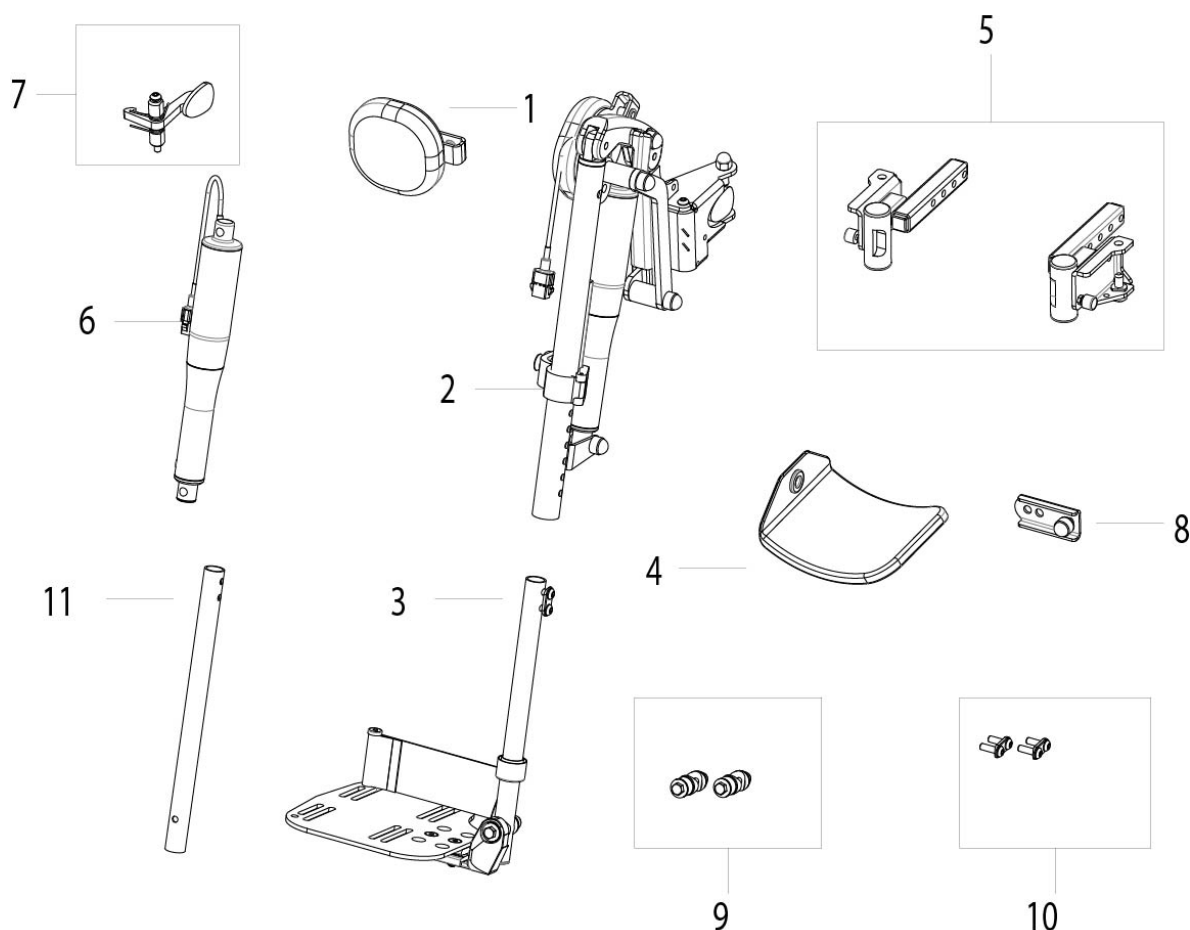


Spare parts

Item no.	Article no.	Description	Remark
7	013451	Latch kit, swing away	black
2	015171	Inline articulating legrest complete, inclusive knee pads	left
2	015172	Inline articulating legrest complete, inclusive knee pads	right
8	015727	Extension kit calf pad	Note: for ALR and ELR
5	035024-ET	Legrest swing away retainer assembly	pair
11	060209	Extension tube	
3	060522	Angle adjustable footplate assembly, inclusive heel loop and extension tube	right
3	060523	Angle adjustable footplate assembly, inclusive heel loop and	left

TEENS power articulating legrest

Item no.	Article no.	Description	Remark
		extension tube	
6	20010015	Actuator	Note: for ALR
9	21005130	Footplate washer kit	2 pcs./pack
10	21005132	Fixing kit footrest including 2 saddle washers and 4 screws	
3	21505700-001	Footplate, angle adjustable, for SW 300-320 mm	left
3	21505700-002	Footplate, angle adjustable, for SW 340-360 mm	left
3	21505701-001	Footplate, angle adjustable, for SW 300-320 mm	right
3	21505701-002	Footplate, angle adjustable, for SW 340-360 mm	right
1	524001.32100	Knee pad assembly	
4	755013	Calf pad cpl. seat widths 12 - 13 inch	
4	755014	Calf pad cpl. 14 - 16 inch	

UNI 80° legrest

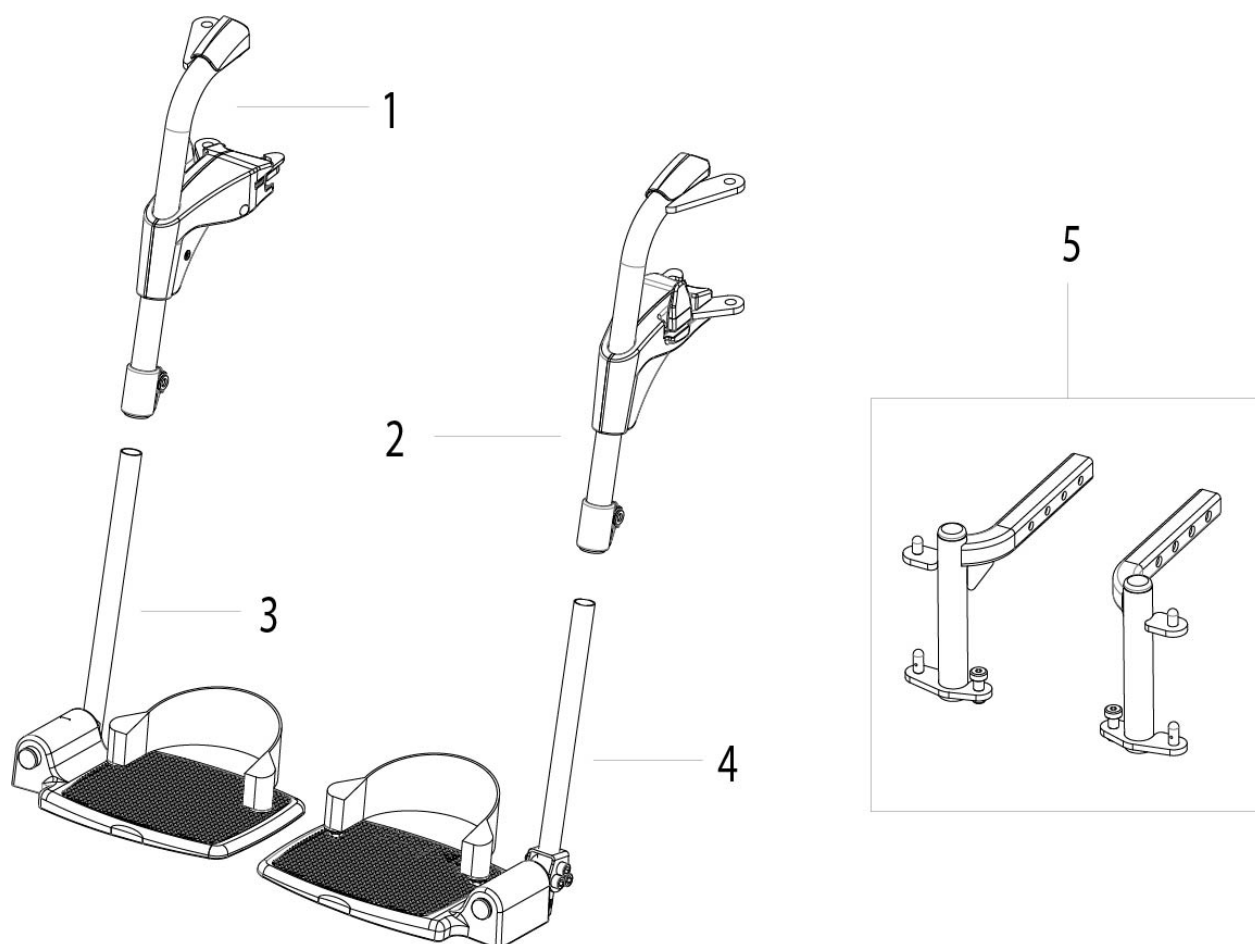

Spare parts

Item no.	Article no.	Description	Remark
2	22605041	Footrest UNI hanger	left
1	22605042	Footrest UNI hanger	right
5	22605060	Receiver Uni Footrest	Pair
3	25066	Footplate, stem assembly UNI aluminium 7.5 inch / 190 mm, for SW 380-400 mm	right, used on 15 and 16 inch wheelchairs
4	25067	Footplate, stem assembly UNI aluminium 7.5 inch / 190 mm, for SW 380-400 mm	left, used on 15 and 16 inch wheelchairs
3	25068	Footplate, stem assembly UNI aluminium 6 inch / 150 mm, for SW 300-360 mm	right
4	25069	Footplate, stem assembly UNI aluminium 6 inch / 150 mm, for	left

UNI 80° legrest


Item no.	Article no.	Description	Remark
		SW 300-360 mm	
3	25371	Footplate angle adjustable assembly 190 mm, for SW 380-400 mm	right
4	25372	Footplate angle adjustable assembly 190 mm, for SW 380-400 mm	left
3	25373	Footplate angle adjustable assembly 150 mm, for SW 300-360 mm	right
4	25374	Footplate angle adjustable assembly 150 mm, for SW 300-360 mm	left

Headrest

Spare parts

Description
Headrest pads
Headrest mount


Headrest pads

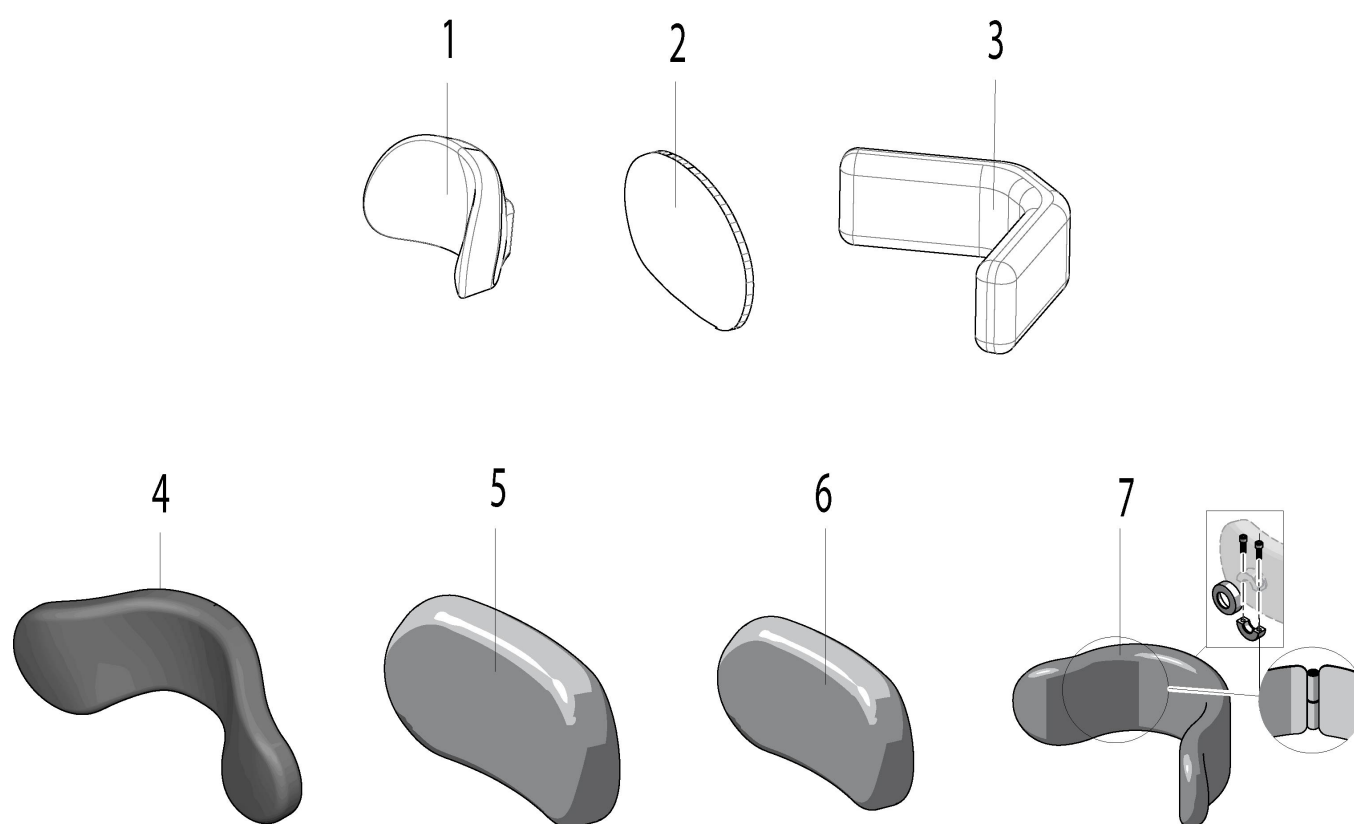

Spare parts

Item no.	Article no.	Description
2	20009103	Headrest cover, cushion standard
1	22509013-001	SEDEO pad small with mounting kit
3	22509013-002	SEDEO pad flexible with mounting kit
5	P1000L	WHITMYER pad, plush, 10 inch / 254 mm
4	P1400L	WHITMYER pad, plush, 14 inch / 356 mm
6	P800L	WHITMYER pad, plush, 8 inch / 203 mm
	P8044L	WHITMYER pad adjustable-A-plush, narrow, LYCRA
7	P8088L	WHITMYER pad adjustable-A-plush, standard, LYCRA

Headrest mount

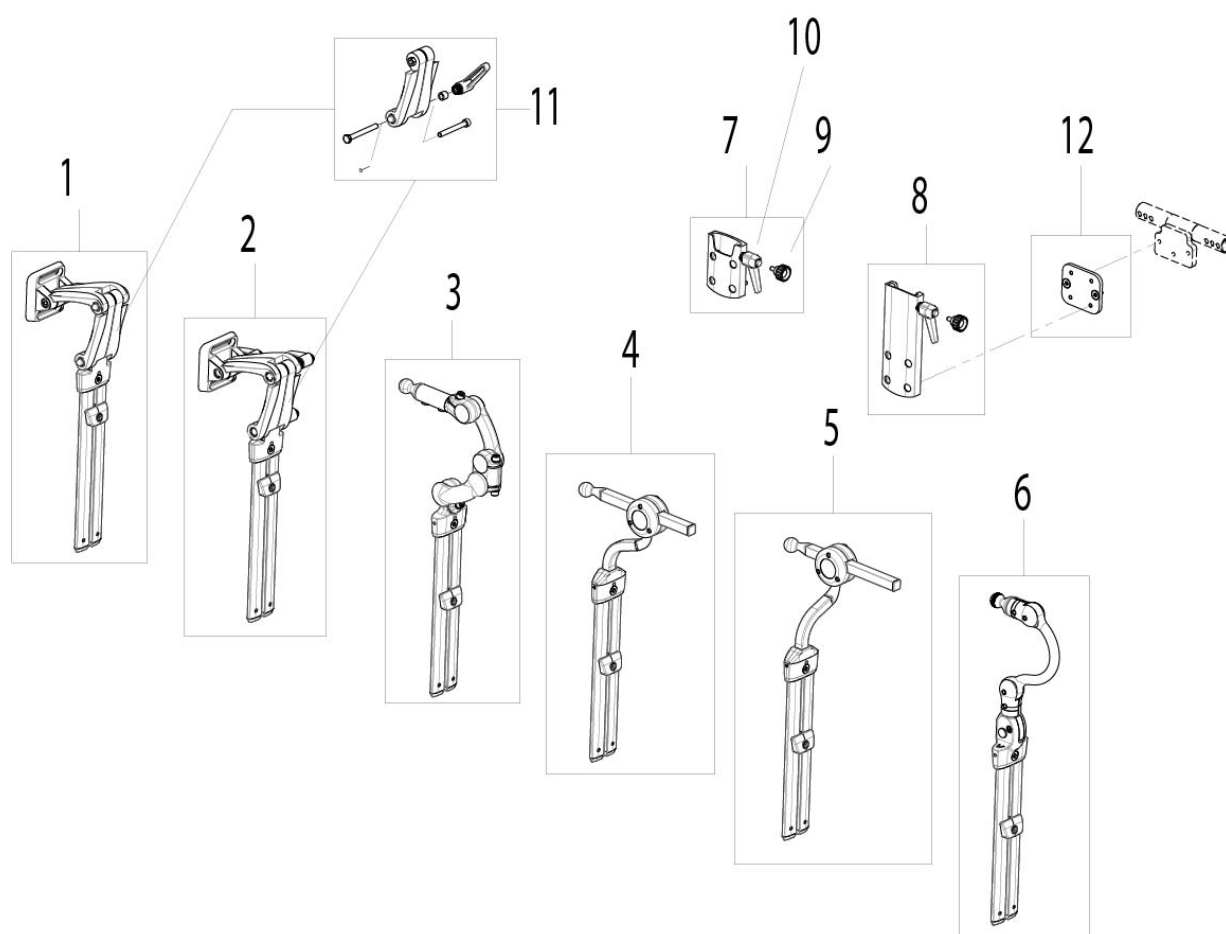

Spare parts

Item no.	Article no.	Description	Remark
10	000233033	Quick release lever	
9	000233061	Knurled screw	1 piece
3	20009005-002	WHITMYER headrest mount, Linx2	
4	20009005-003	WHITMYER headrest mount, Axys	
5	20009005-004	WHITMYER headrest mount, Axys offset	
6	20009005-005	WHITMYER headrest mount, Cobra xtra flip back	
11	20009278	Hinge, Sedeo Headrest mount	
8	21509010	Headrest mount, backframe mount	
12	21509011	Headrest adaptor kit - SEDEO to ZIPPIE	

Headrest mount


Item no.	Article no.	Description	Remark
1	21809210	SEDEO headrest assembly	
7	22509006-002	Headrest mount, back cushion, short	
2	22509012	Headrest mount, quick release	

Product and safety labels

Spare parts

Description
Safety labels
Q300 M - Product labels

Safety labels

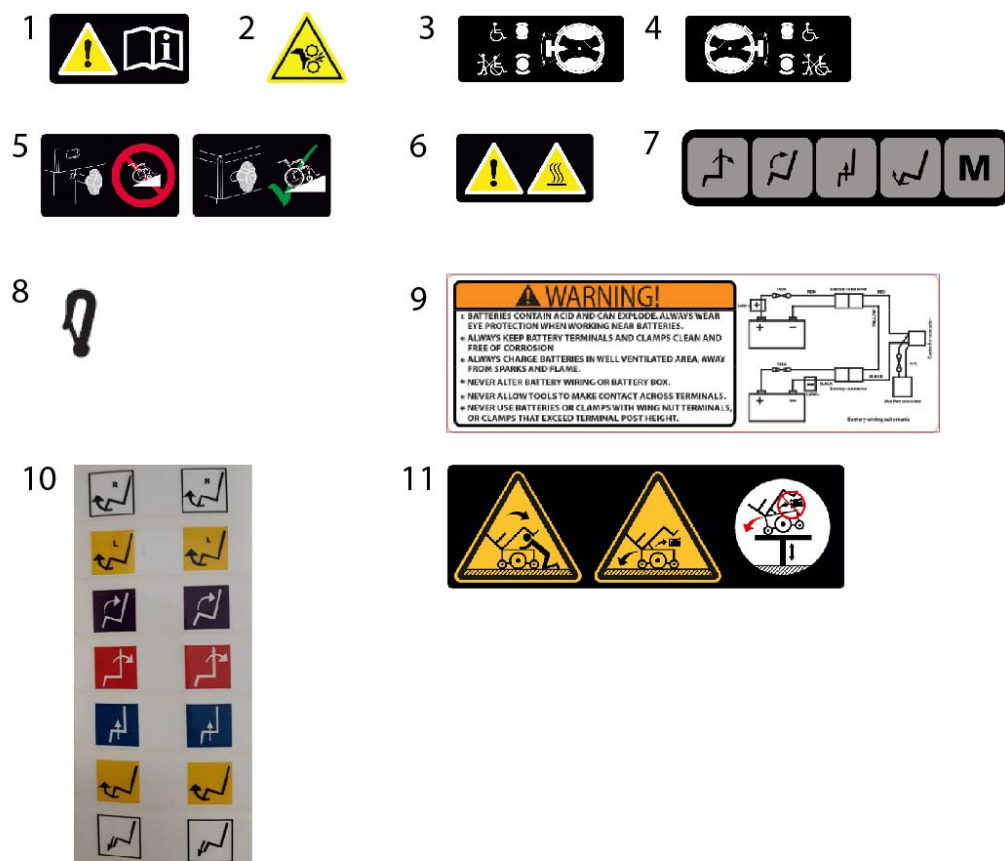

Spare parts

Item no.	Article no.	Description	Remark
3	000280100-001	Label free wheel warning A	MWD / FWD = left, RWD = right
4	000280100-002	Label free wheel warning B	MWD / FWD = right, RWD = left
8	000280221-ET	Sticker - Fixation point - white for rough surface	4 pcs./pack
10	013684	Loom labels	
6	014599	Label - hot surface (motor) -	2 pcs. per wheelchair
2	014603	Label - common finger trap warning -	
1	060589	Label - refer to user manual -	2 pcs. per wheelchair
9	065128	Label - Battery warning -	2 pcs. per wheelchair
7	20017010-001	Pictogram 5 - switch box for standard seating system FOR STANDARD SEATING SYSTEM	

Safety labels

Item no.	Article no.	Description	Remark
11	20017011	Warning label - seat stay	Q700 M / Q500 M / Q400 M
5	21017002	Label - seat interface quick release	

Q300 M - Product labels



Spare parts

Item no.	Article no.	Description	Remark
2	21517604-001	Front and back decal -QUICKIE- red	pair
2	21517604-002	Front and back decal -QUICKIE- blue	pair
2	21517604-003	Front and back decal -QUICKIE- white	pair
2	21517604-004	Front and back decal -QUICKIE- black	pair
4	21517607-001	Front and back decal -ZIPPIE- red	pair
4	21517607-002	Front and back decal -ZIPPIE- blue	pair
4	21517607-003	Front and back decal -ZIPPIE- white	pair
4	21517607-004	Front and back decal -ZIPPIE- black	pair

Q300 M - Product labels

Item no.	Article no.	Description	Remark
1	21517655-001	Product label -Q300M Mini- white	pair
1	21517655-002	Product label -Q300M Mini- black	pair
1	21517655-003	Product label -Q300M Mini- silver	pair
3	21517660-001	Front or back decal -QUICKIE- grey burst	
3	21517660-002	Front or back decal -QUICKIE- red burst	
3	21517660-003	Front or back decal -QUICKIE- blue burst	
3	21517660-004	Front or back decal -QUICKIE- pure white burst	
3	21517660-005	Front or back decal -QUICKIE- flames asteroid grey	
3	21517660-006	Front or back decal -QUICKIE- flames hot red	
3	21517660-007	Front or back decal -QUICKIE- flames sharp blue	
3	21517660-008	Front or back decal -QUICKIE- flames pure white	
3	21517660-009	Front or back decal -QUICKIE- floral white	
3	21517660-010	Front or back decal -QUICKIE- carbon black	
3	21517660-011	Front and back decal -QUICKIE- purple burst	
3	21517660-012	Front and back decal -QUICKIE- pink burst	
5	21517670-001	Front or back decal -ZIPPIE- grey burst	
5	21517670-002	Front or back decal -ZIPPIE- red burst	
5	21517670-003	Front or back decal -ZIPPIE- blue burst	
5	21517670-004	Front or back decal -ZIPPIE- pure white burst	
5	21517670-005	Front or back decal -ZIPPIE- flames asteroid grey	
5	21517670-006	Front or back decal -ZIPPIE- flames hot red	

Q300 M - Product labels

Item no.	Article no.	Description	Remark
5	21517670-007	Front or back decal -ZIPPIE- flames sharp blue	
5	21517670-008	Front or back decal -ZIPPIE- flames pure white	
5	21517670-009	Front or back decal -ZIPPIE- floral white	
5	21517670-010	Front or back decal -ZIPPIE- carbon black	
5	21517670-011	Front and back decal -ZIPPIE- purple burst	
5	21517670-012	Front and back decal -ZIPPIE- pink burst	


Backrest

Spare parts

Description
Backcushion, Holder
Backrest standard
Backrest power, adjustable
Tension back upholstery
Back cushion, deep contour
Lateral thoracic supports

Backcushion, Holder


1

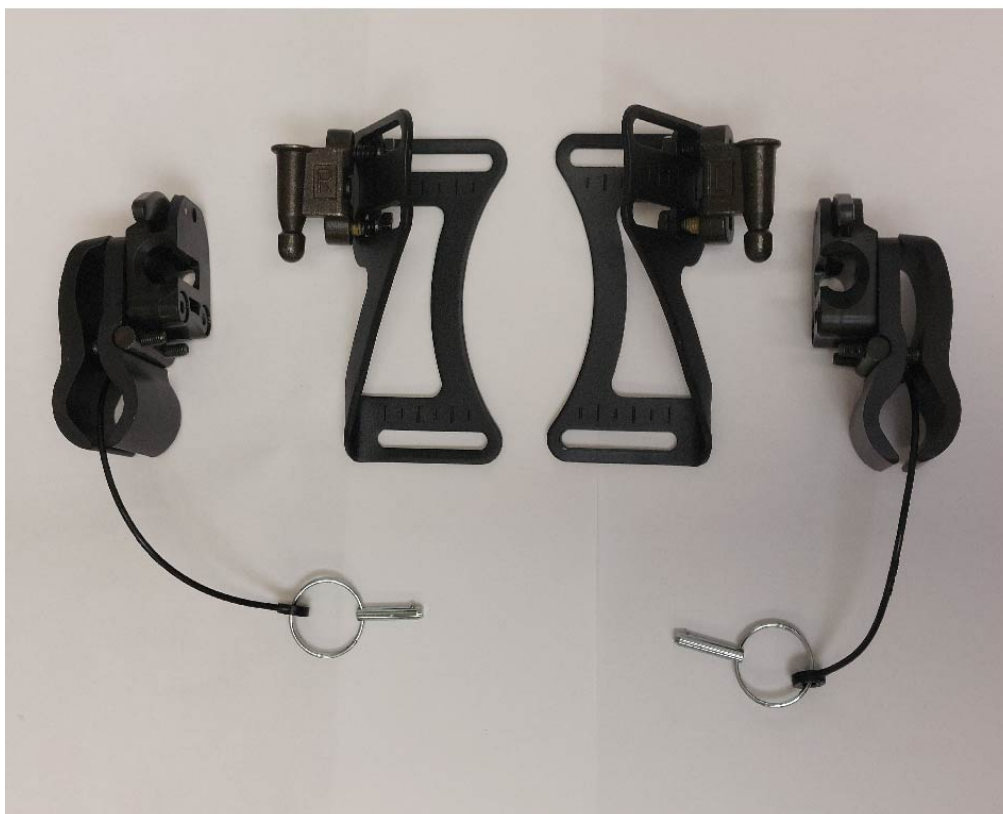

Spare parts

Item no.	Article no.	Description	Remark
	20003021	Screw Kit for back bracket (J3BCKG)	
1	J3BCKG	Fixing bracket, complete	Pair

Backrest standard

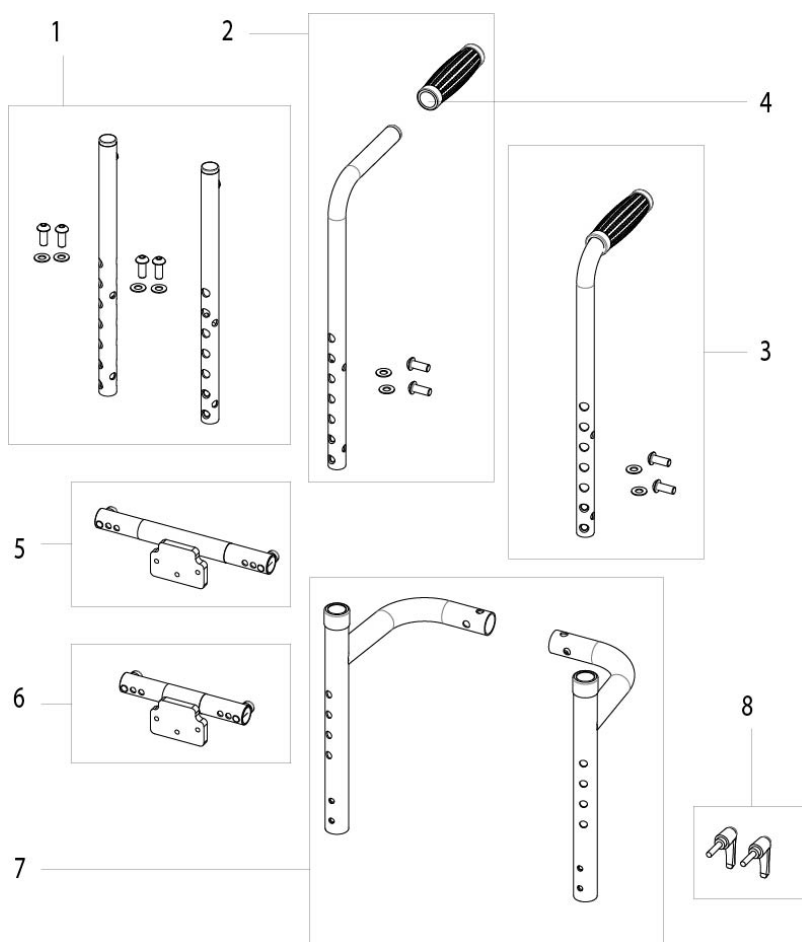


Spare parts

Item no.	Article no.	Description	Remark
8	000102021-ET	Clamping lever M6 x 25 GN503	2 pcs./pack
6	035071-001-ET	Central back brace SW 300-320-340 mm	
5	035071-002-ET	Central back brace SW 360-380-400 mm	
4	104428-ET	Handgrip	2 pcs./pack
3	20103001	Push handle, standard, BH 380-480 mm	left
2	20103002	Push handle, standard, BH 380-480 mm	right
	20103009	Push handle, tall, BH 430-530 mm	left
	20103010	Push handle, tall, BH 430-530 mm	right
1	21503613	Backpost straight, standard, BH	pair

Backrest standard

Item no.	Article no.	Description	Remark
		380-480 mm	
	21503614	Backpost straight, tall, BH 430-530 mm	pair
7	21503615	Backpost, assembly	pair


Backrest power, adjustable

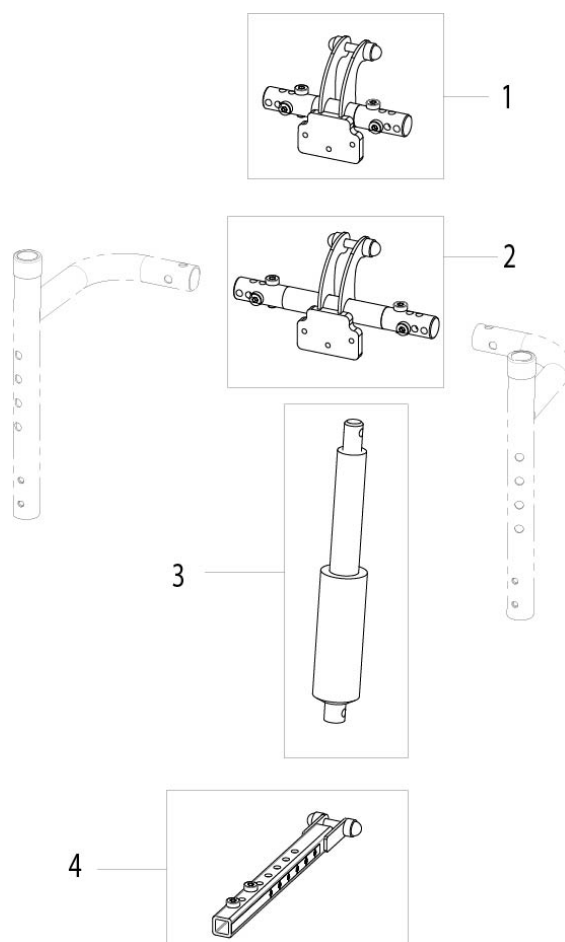

Spare parts

Item no.	Article no.	Description
3	015277	MBL actuator assembly LL1001 special
1	035058-001-ET	Actuator fixing centre SW 300-320-340 mm
2	035058-002-ET	Actuator fixing centre SW 360-380-400 mm
4	035061-ET	Actuator bracket cpl.
1,3,4	035080	Powered adj. back cpl. SW 300 - 320 - 340 mm
2,3,4	035081	Powered adj. back cpl. SW 360 - 380 - 400 mm


Tension back upholstery

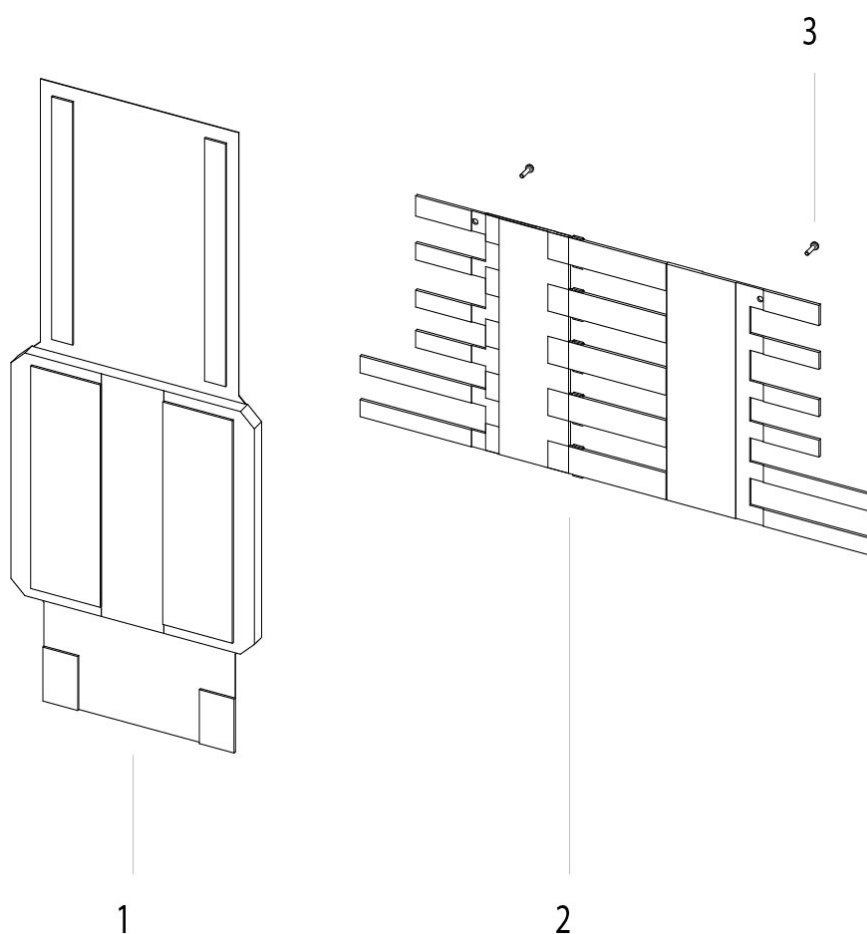

Spare parts

Item no.	Article no.	Description	Remark
3	000102066-ET	Socket button head cap screw M5 x 20 DIN7985	12 pcs./pack
1	035064	Back upholstery	Kids seat
2,3	035065-ET	Zippie back straps and fixing cpl.	1 pc.

Back cushion, deep contour

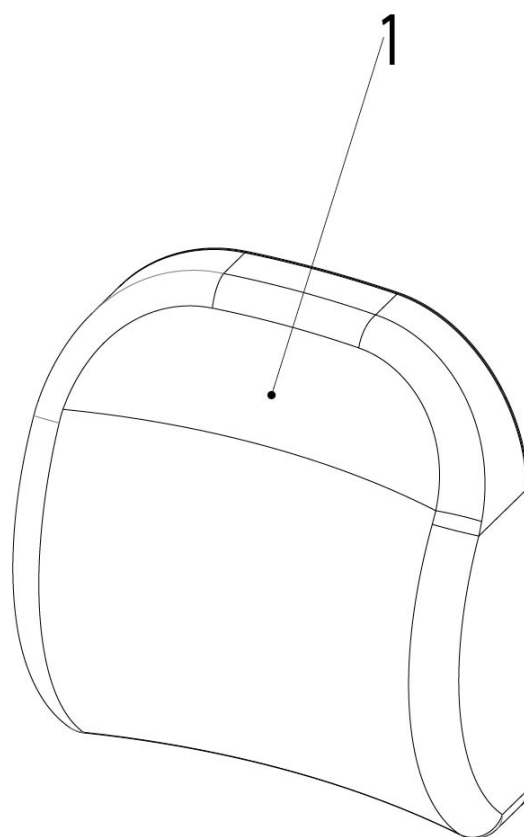


Spare parts

Item no.	Article no.	Description	Remark
1	96503200-001	Back cushion, deep contour, active, without straps, incontinent cover	SW 320 x BH 350 mm
1	96503200-002	Back cushion, deep contour, active, with straps, incontinent cover	SW 370 x BH 400 mm
1	96503200-101	Back cushion, deep contour, active, without straps, comfort cover	SW 320 x BH 350 mm
1	96503200-102	Back cushion, deep contour active with straps, comfort cover	SW 370 x BH 400 mm

Lateral thoracic supports

1

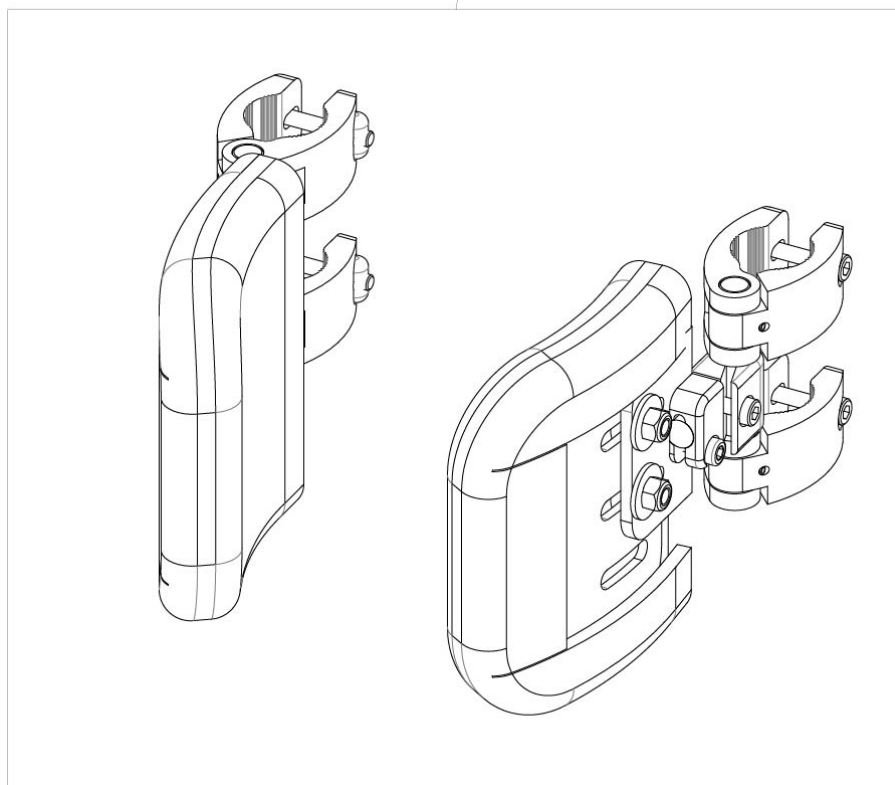

Spare parts


Item no.	Article no.	Description	Remark
1	JSBLS1012P	Fixed lateral support 100 x 120 mm complete	Pair

Seat

Spare parts

Description
Seat module, lift and tilt 30°
Seat plate
Seat module, fixed
Seat module, tilt
Seat cushion
Q300 M - Seat interface

Seat module, lift and tilt 30°

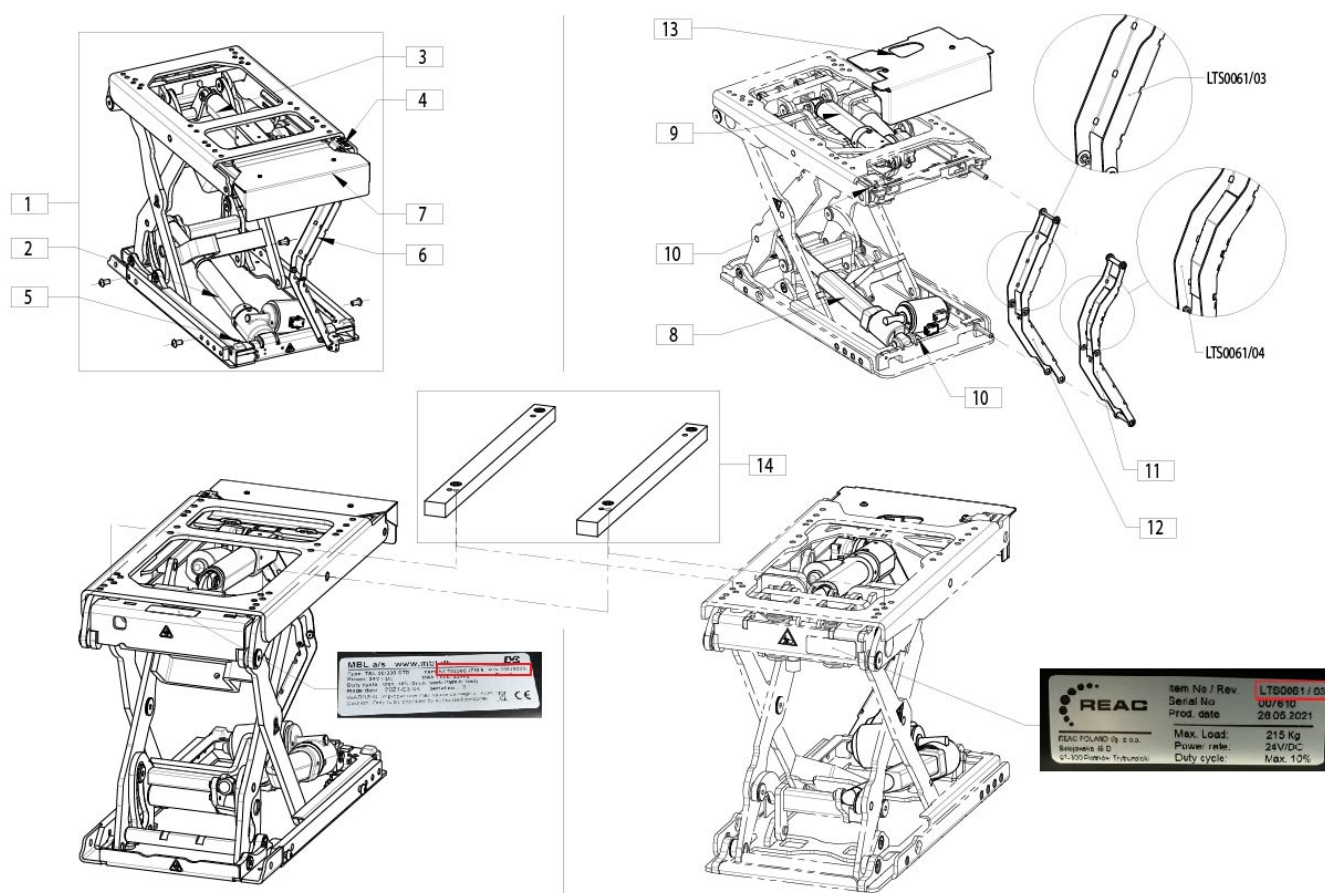


Spare parts

Item no.	Article no.	Description	Remark
8	20010500-101	Lift actuator kit	only for LTS0061 resp. 20010103
2	20010500-201	Lift actuator kit	only for NLT0250C resp. 20010203
9	20010501-101	Tilt actuator 30° kit	only for LTS0061 resp. 20010103
3	20010501-201	Tilt actuator 30° kit	only for NLT0250C resp. 20010203
10	20010502-101	Micro Switch, lift resp. tilt	only for LTS0061 resp. 20010103
4	20010502-201	Micro Switch kit, lift	only for NLT0250C resp. 20010203
5	20010502-202	Micro Switch kit, tilt	only for NLT0250C resp. 20010203
11	20010503-101	Cable track,incl. mounting material	only for LTS0061/03 resp. 20010103
12	20010503-102	Cable track,incl. mounting	only for LTS0061/04 resp.

Seat module, lift and tilt 30°

Item no.	Article no.	Description	Remark
		material	20010103
6	20010503-201	Cable track, incl. mounting material	only for NLT0250C resp. 20010203
13	20010504-101	Cover, cable track, incl. mounting material	only for LTS0061 resp. 20010103
7	20010504-201	Cover, cable track, incl. mounting material	only for NLT0250C resp. 20010203
1	20012011-004	Lift / tilt module, inclusive mounting accessories	NLT0250C resp. 20010203
14	22812000	Adapter plate lift / tilt module, cpl.	Pair


Seat plate

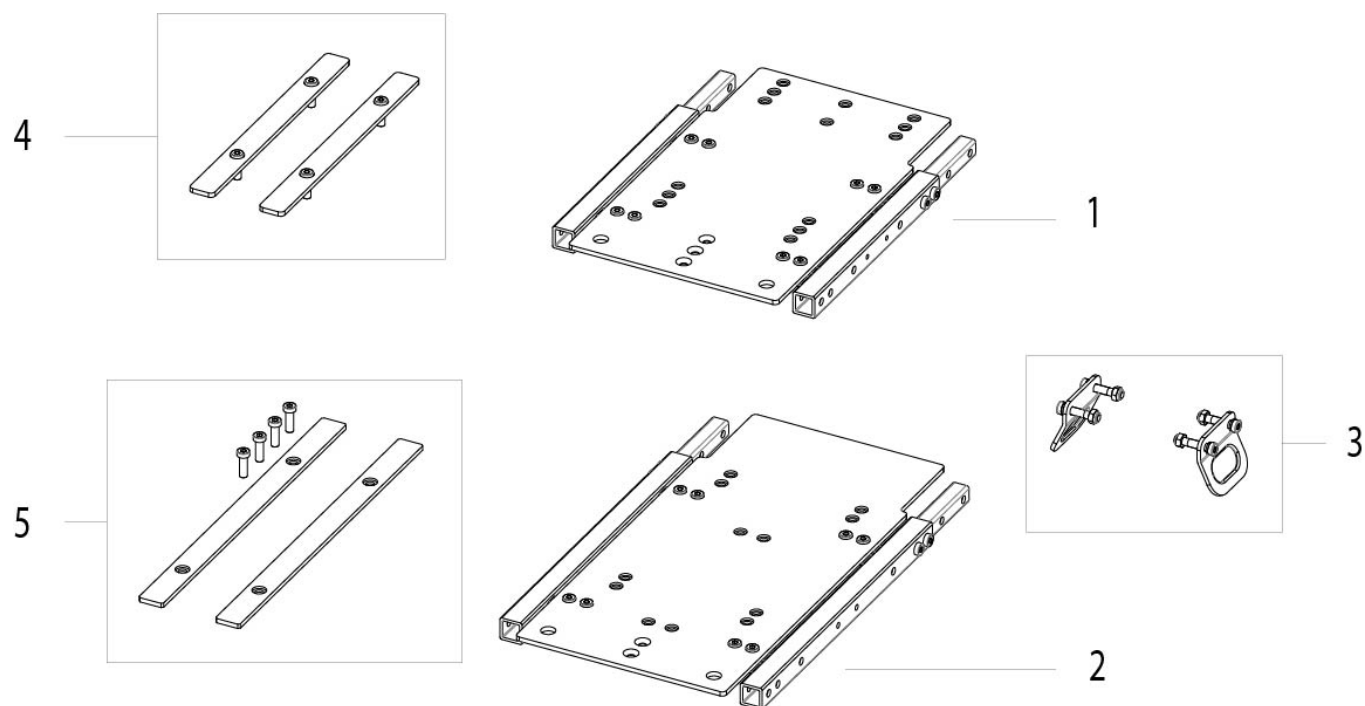

Spare parts

Item no.	Article no.	Description	Remark
4	035037-ET	Seat cover SW 300 - 320 - 340 mm, cpl.	
3	060878	Transit kit, front	pair, Q300M Mini / SALSA / SALSA M, SALSA R2 / M2, HULA
5	21502611	Cover plate kit, SW 360-400 mm	pair
1	21502675	Seat plate assembly ZIPPIE kids	
2	21502676	Seat plate assembly ZIPPIE teens	


Seat module, fixed

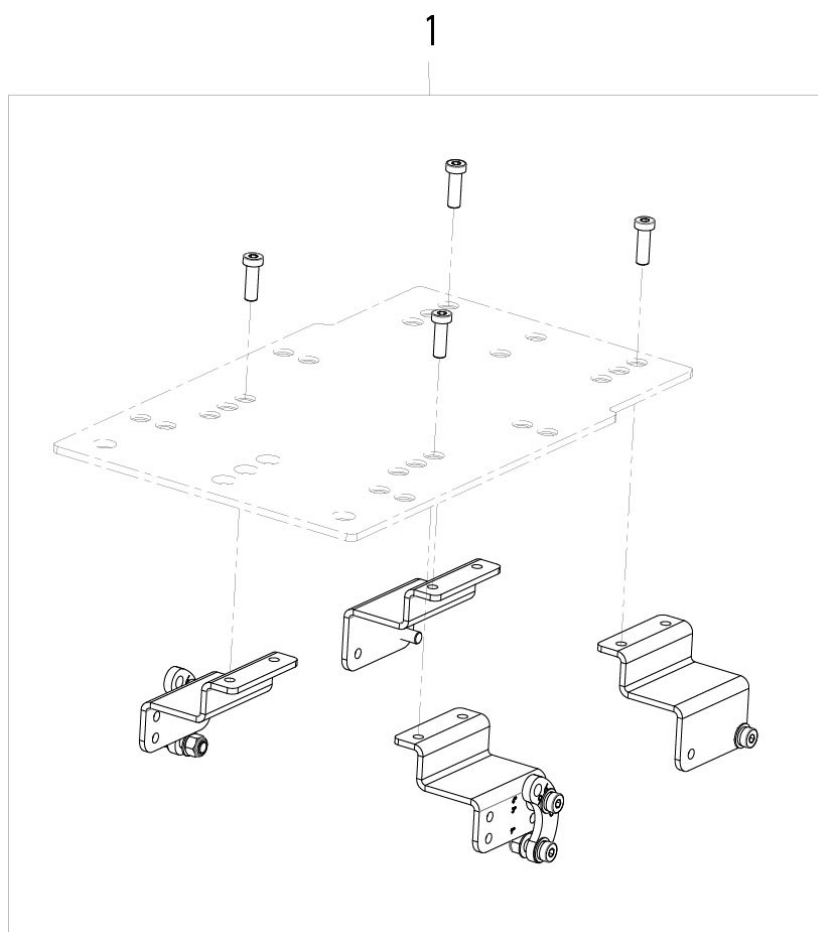

Spare parts

Item no.	Article no.	Description
1	035028	Adjustable seat mounting kit, incl. fixings


Seat module, tilt

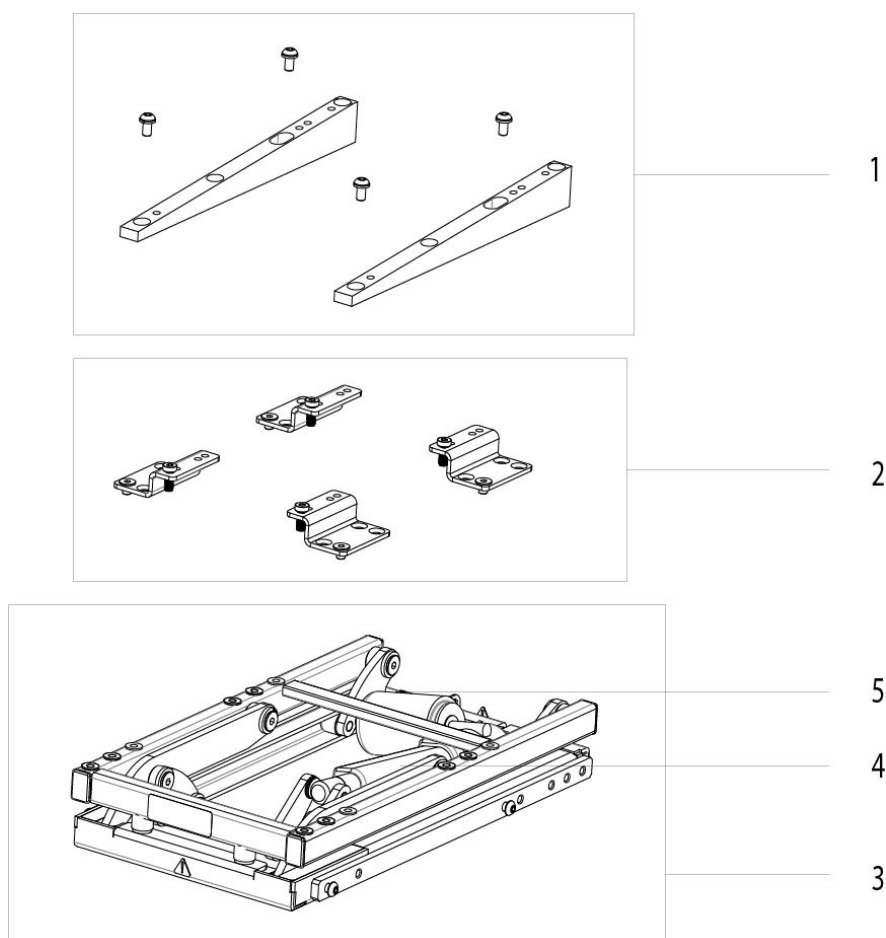

Spare parts

Item no.	Article no.	Description	Remark
2	035029	Adjustable seat mounting kit, incl. fixings	
4	20010131	Actuator tilt module 30°	Note: must be used only for # 20010100, 30 degree Tilt Module and 20010101, 50 degree Tilt Module, Act.Load: 6000N, Item Nr. 94Q0000M06, NAQ002-C
5	20010132	Micro switch and connecting loom	
3	20012010-001	Tilt module 30°, inclusive mounting accessories	
3	20012010-002	Tilt module 50°, inclusive mounting accessories	
1	21012015	Spacer - 5 to 25 or -5 to 45 degree kit	


Seat cushion

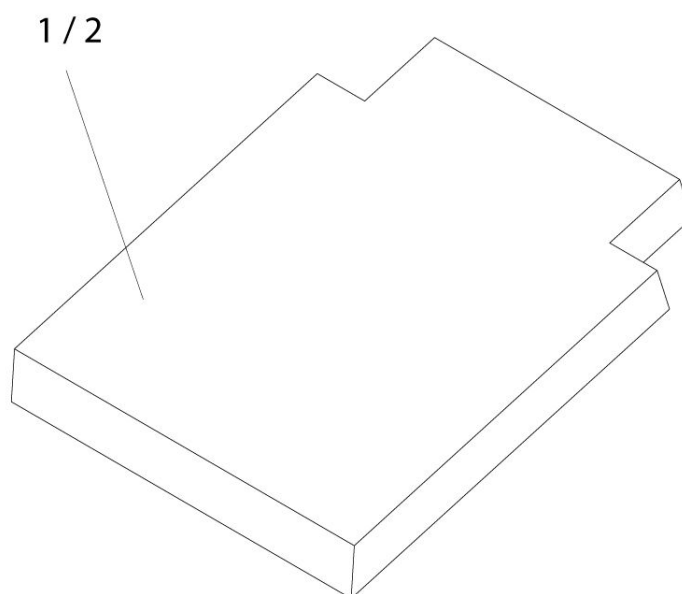

Spare parts

Item no.	Article no.	Description	Remark
1	035034-001	Seat cushion - kids	SW 300 - 340 mm
2	035034-002	Seat cushion - kids	SW 360 - 400 mm
1	21502607-001	Seat cushion - teens	SW 300 - 340 mm
2	21502607-002	Seat cushion - teens	SW 360 - 400 mm

Q300 M - Seat interface

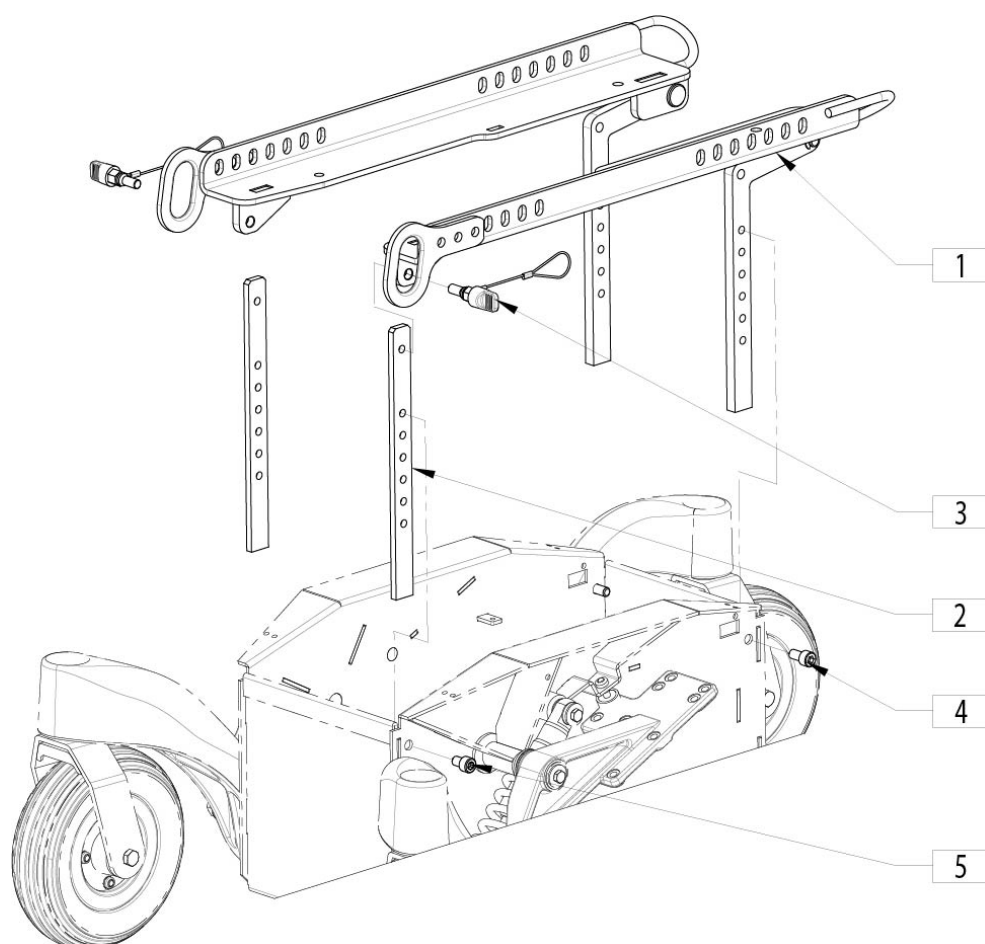

Spare parts


Item no.	Article no.	Description	Remark
3	162343	Seat interface, fixing kit	Pair
1,2,4,5	21512200	Seat interface cpl.	
4,5	21512606	Screw kit	

Control mount

Spare parts

Description
Mount, 4-actuator box VR2
Mount, joystick VR2/R-Net
VR2 / R-Net attendant control and mounting


Mount, 4-actuator box VR2

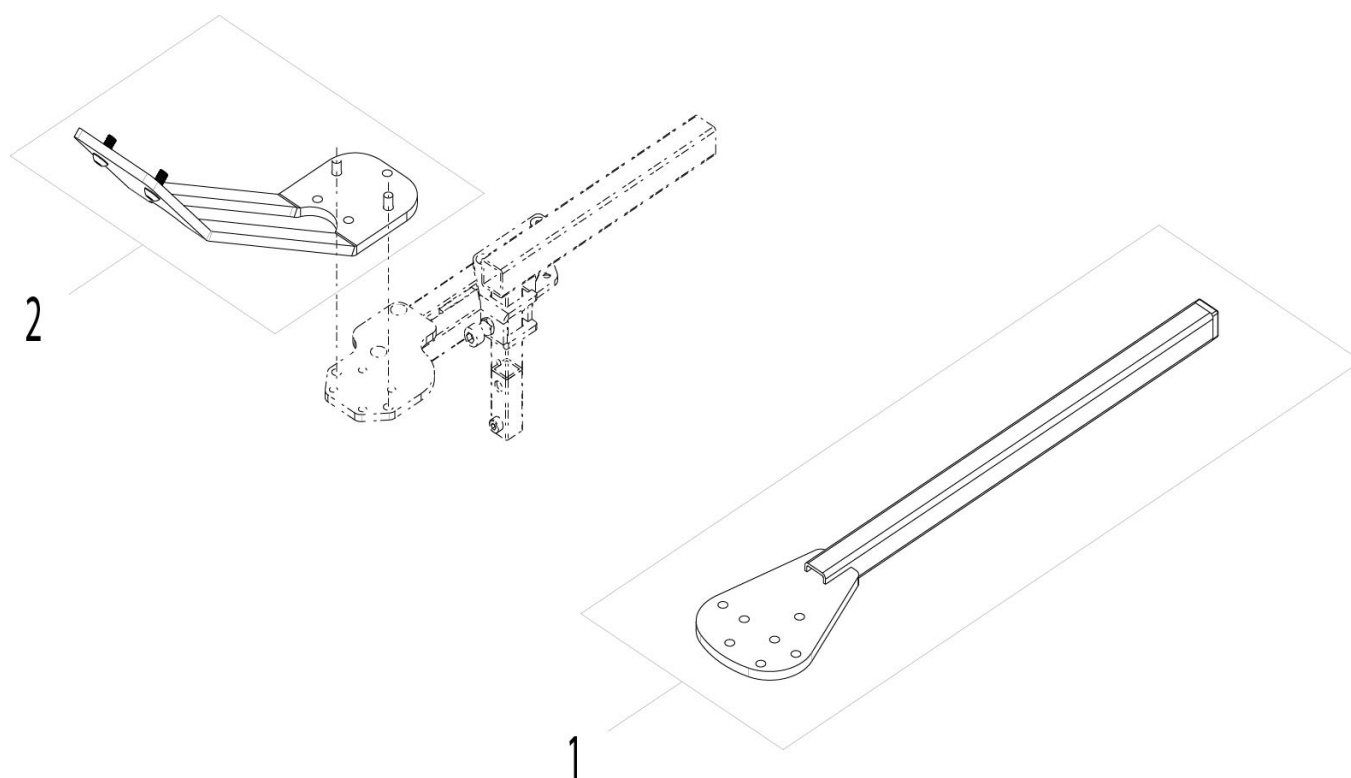

Spare parts

Item no.	Article no.	Description	Remark
1	060600	4 - actuator box mounting	Note: armrest fitting
2	060685	4 - actuator box mounting	Note: VR2 controller fitting


Mount, joystick VR2/R-Net

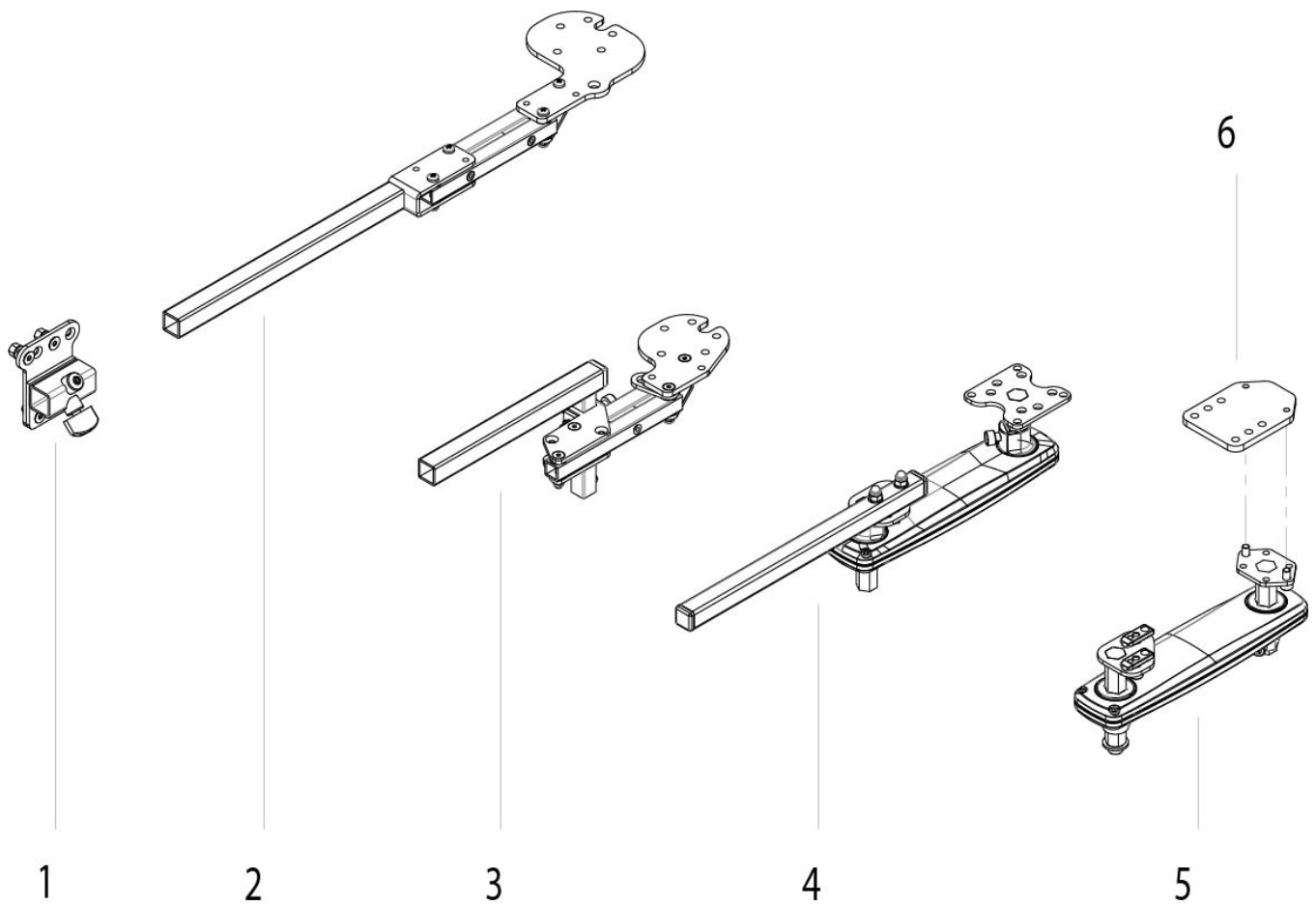

Spare parts

Item no.	Article no.	Description	Remark
2	060248-001	Swing away control mount	right
2	060248-002	Swing away control mount	left
1	060527	Control mount with vertical adjustment assembly (to fit on armrest)	
6	21504030	Mounting plate	
3	21504605	New swing away mount	
5	22509070-002	Joystick mount swing away assembly	left, for joystick / compact joystick / 5-switch box
3	22604060	Swing away control mount	left
4	6003392	Joystick mount swing away	right
4	6003507	Joystick mount swing away	left

VR2 / R-Net attendant control and mounting

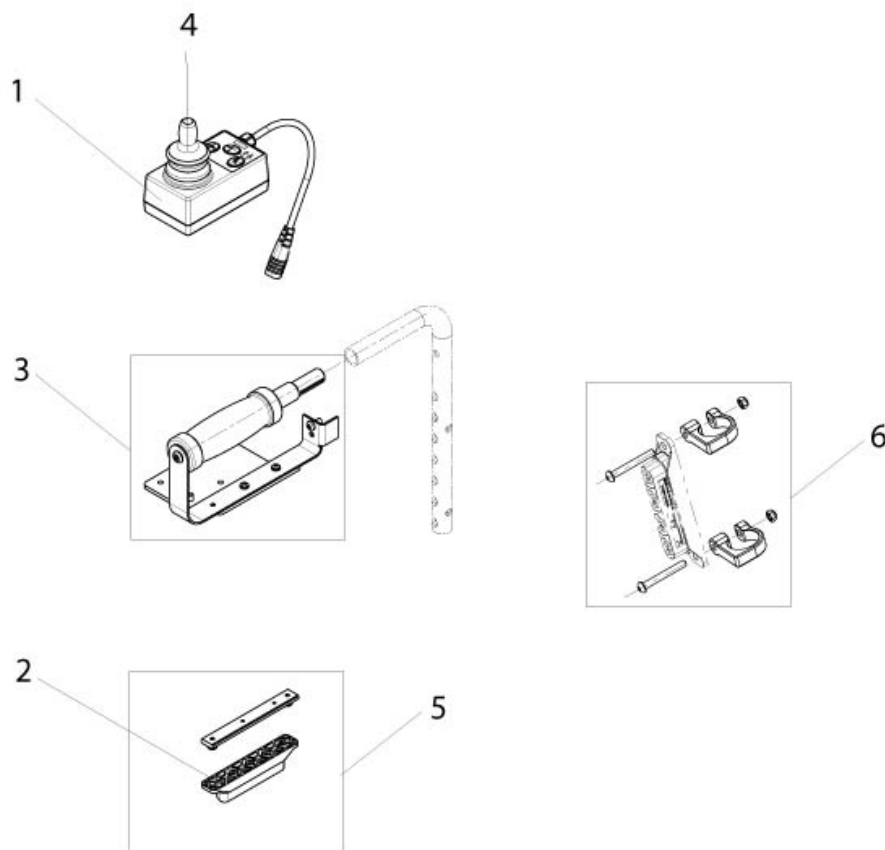

Spare parts

Item no.	Article no.	Description	Remark
	060648	VR2 loom for dual control	
1, 3, 5	060723	Retrofit kit R-Net remote dual inclusive bus cable	Note: without headrest
3	060724	Attendant control mount	VR2 / R-Net, note: without headrest
5	060747	Fixing for distribution block	R-Net
1, 3	060778	VR2 remote dual inclusive cable	Note: without headrest
4	1010114	Knob, standard joystick knob	not allowed for allround joystick light version and versaguide joystick light version
2	108312	R-Net distribution block	
1,2	21011025	R-Net attendant control joystick, incl. fixing material	

VR2 / R-Net attendant control and mounting

Item no.	Article no.	Description	Remark
1	21011026	VR2 dual control joystick for attendant control	
6	21109035	Fixation kit destributation block	

Control input device

Spare parts

Description
Joystick knobs
VR2 joysticks and 4-actuator box
R-Net joysticks
OMNI 2

Joystick knobs




Spare parts

Item no.	Article no.	Description	Remark
6	000934.0139	Knob, chin cup, D = 37 mm	not allowed for allround joystick light version and versaguide joystick light version
2	014860	Knob, golf ball, D = 42 mm	not allowed for allround joystick light version and versaguide joystick light version
9	1010114	Knob, standard joystick knob	not allowed for allround joystick light version and versaguide joystick light version
4	172384.3800	Knob, soft ball, D = 70 mm	not allowed for allround joystick light version and versaguide joystick light version
3	172384.3801	Knob, t-bar, 82 mm width	not allowed for allround joystick light version and versaguide joystick light version

Joystick knobs

Item no.	Article no.	Description	Remark
8	172384.3802	Knob, mushroom, D = 45 mm	not allowed for allround joystick light version and versaguide joystick light version
7	172384.3803	Knob, stick, 80 x 15 mm	not allowed for allround joystick light version and versaguide joystick light version
5	172384.3804	Knob, ball, D = 35 mm	not allowed for allround joystick light version and versaguide joystick light version
1	6003447	Knob, Allround ball, D = 48 mm	

VR2 joysticks and 4-actuator box

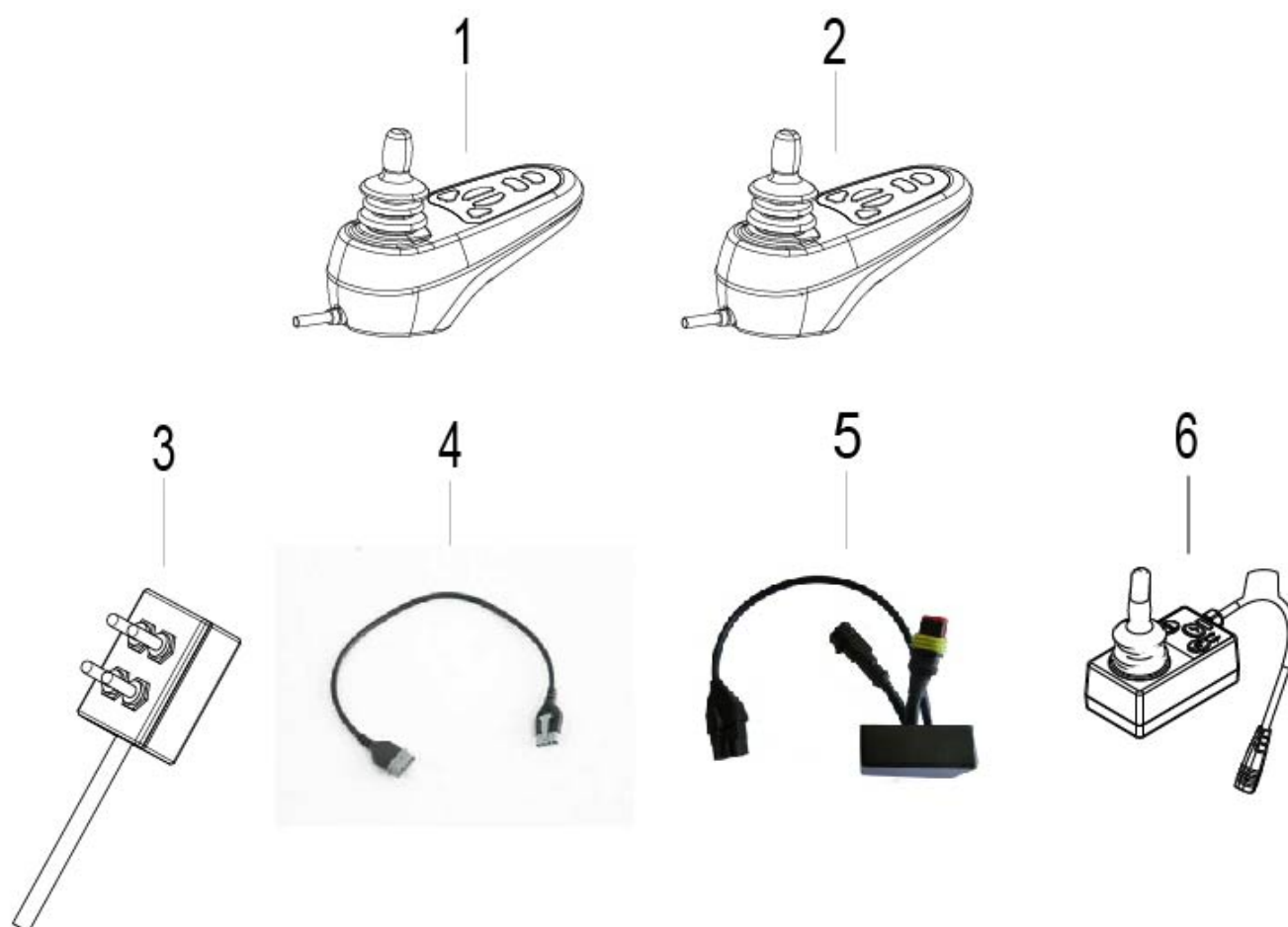

Spare parts

Item no.	Article no.	Description	Remark
5	060786	Circuit breaker used with black box (TÜV)	
4	172001.32117	VR2 extension cable 300 mm	
2	20011014	VR2 joystick	drive only
1	21011015	VR2 joystick 2 actuator control	
1	21011016	VR2 joystick 2 actuator control lights and indicators + actuators, inclusive mounting material	with lights and indicators
6	21011026	VR2 dual control joystick for attendant control	
3	21011029	4 actuator control box and loom	

R-Net joysticks




Spare parts

Item no.	Article no.	Description	Remark
3	20011015	R-Net joystick, coloured LCD, lights and indicators inclusive 2 jacks, incl. fixing material	
5	20011016-001	R-Net joystick, coloured LCD HiRes, IR function, incl. fixing material	For R-Net connection loom 20011032-120 or 20011035 required
6	20011016-002	R-Net joystick, coloured LCD HiRes, IR function and Bluetooth, incl. fixing material	For R-Net connection loom 20011032-120 or 20011035 required
1	21011017	R-Net joystick LED JSM	
2	21011018	R-Net joystick LED JSM light	
4	21011025	R-Net attendant control joystick, incl. fixing material	
7	6002050	R-Net joystick, table top control	
8	6003875	R-Net joystick, coloured LCD, lights and indicators inclusive 2 jacks, heavy duty, incl.	

R-Net joysticks

Item no.	Article no.	Description	Remark
		fixing material	

OMNI 2

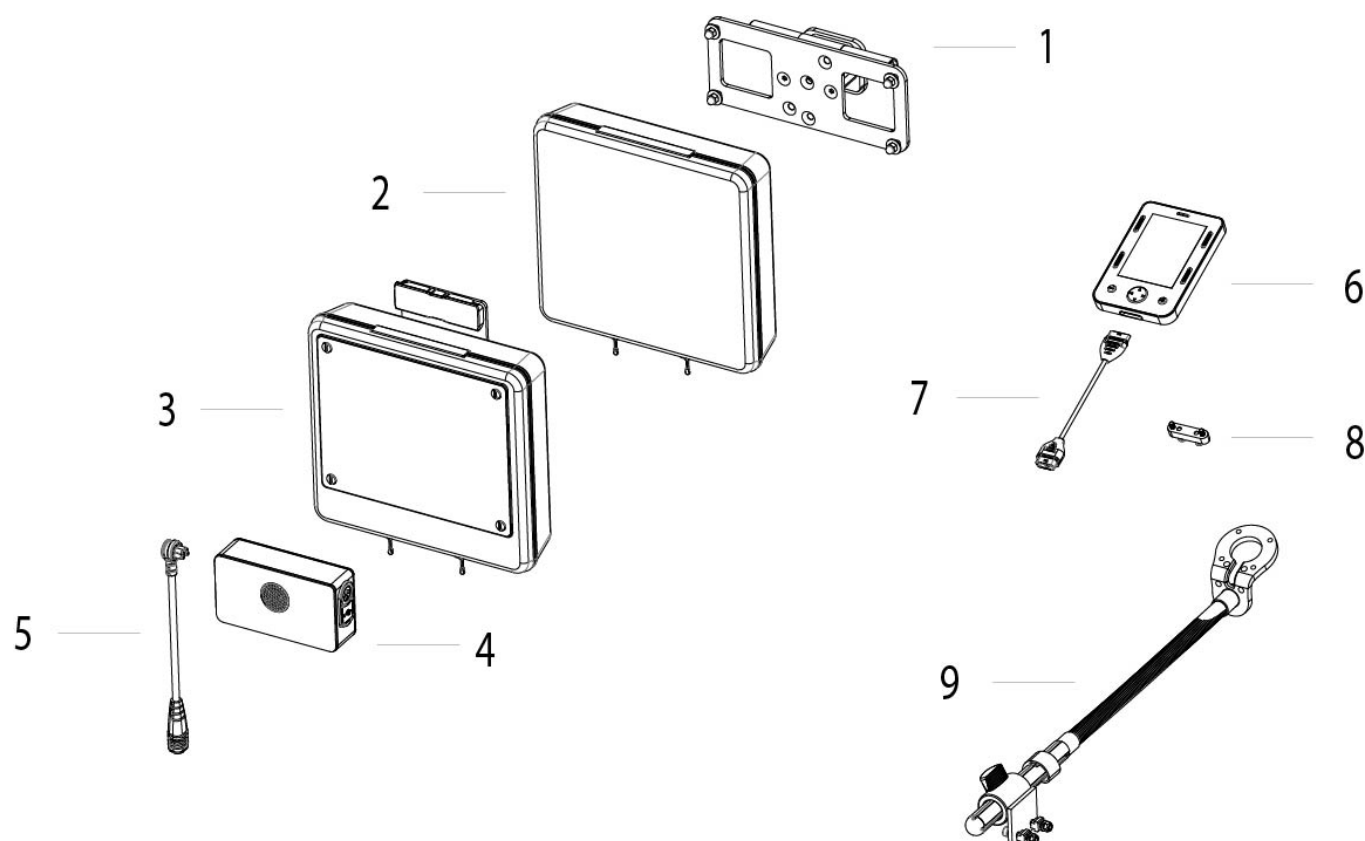

Spare parts

Item no.	Article no.	Description	Remark
9	035350	Omni control panel flex mounting complete	
1	20009164-001	OMNI 2 mounting, no headrest	fixed recline
1	20009164-003	OMNI 2 mounting, no headrest	powered recline
1	20009164-005	OMNI 2 mounting, SEDEO headrest	fixed recline
1	20009164-006	OMNI 2 mounting, SEDEO headrest	powered recline
8	20009174	OMNI 2 adapter plate, incl. fixings	
2	20009190-010	Bag for OMNI 2 control device	
3	20009190-030	Bag for OMNI 2 control device incl. Reflector kit	
5	20011035	R-Net loom with angle plug, 300 mm	
4	20011230-010	OMNI 2 Control device	

OMNI 2

Item no.	Article no.	Description	Remark
6	20011231-010	Omni 2 display	
7	20011236-200	Omni 2 Display connection cable, 2000 mm	

Accessories

Spare parts

Description
Charger
Q300 M - Batteries - (only for Q300 M Mini)
Lap belts
Rear view mirror
Tray table, swing away

Charger




Spare parts

Item no.	Article no.	Description	Remark
3	060500	EU power cord for charger	Note: includes EU mains lead
2	060501	Charger powercord UK 6FT	
5	133001.10010	Charger classic	220V / 8A, IP54
6+2	133001.10012	Charger 220V 10 A, with UK 3 pin power cord	
7+3	133001.10013	Charger 220V 10 A - with EU 2 pin power cord	
1,3	20016102	8 A IEB charger with EU 2 pin power cord	
1,2	20016103	8 A IEB charger with UK 3 pin power cord	
1,4	20016104	8 A IEB charger with CH 3 pin power cord	


Charger

Item no.	Article no.	Description	Remark
4	20016110	CH 3 pin power cord for charger	

Q300 M - Batteries - (only for Q300 M Mini)

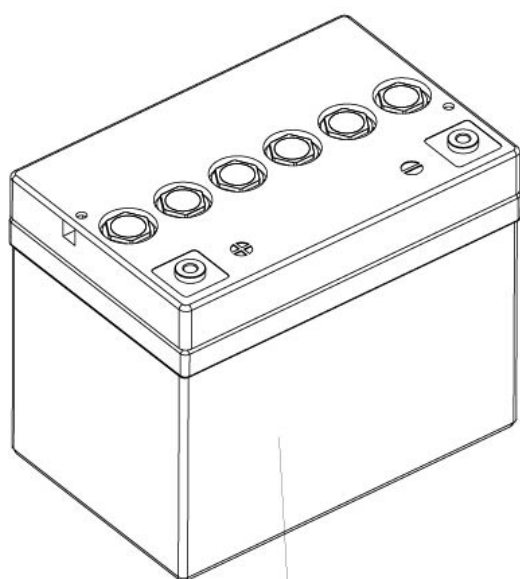

1

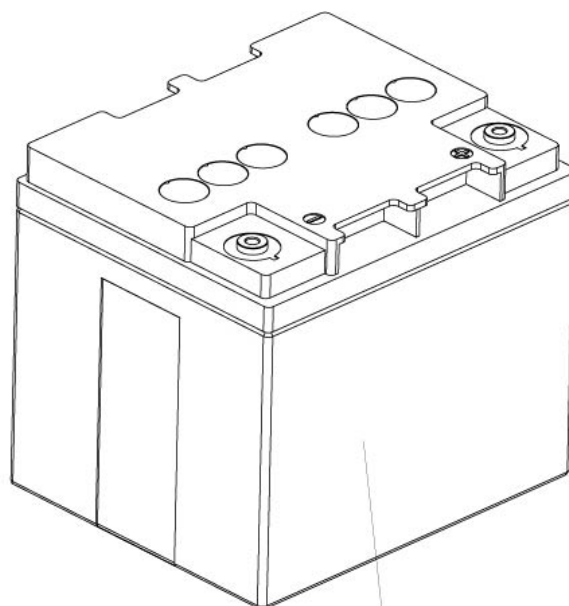


2

Spare parts

Item no.	Article no.	Description
1	000172.20006	Battery 41 Ah
2	21516650	Battery 56 Ah


Lap belts

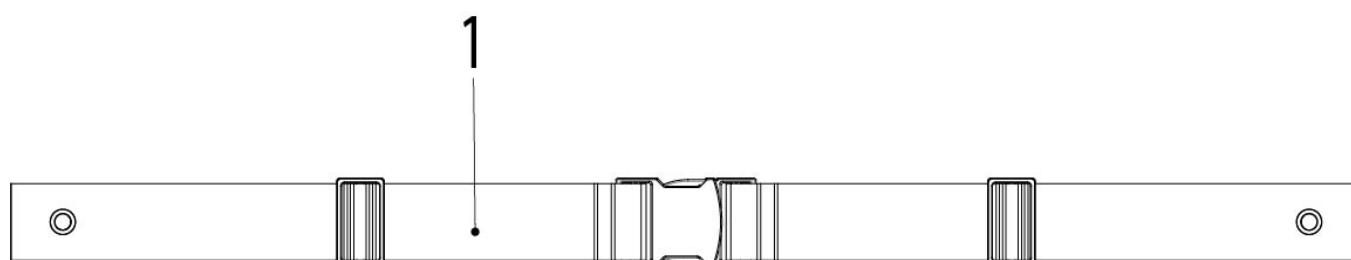

Spare parts

Item no.	Article no.	Description	Remark
1	035040-000	Lap strap	for kids seat
1	060859	Lap strap	For adult/teens seat


Rear view mirror

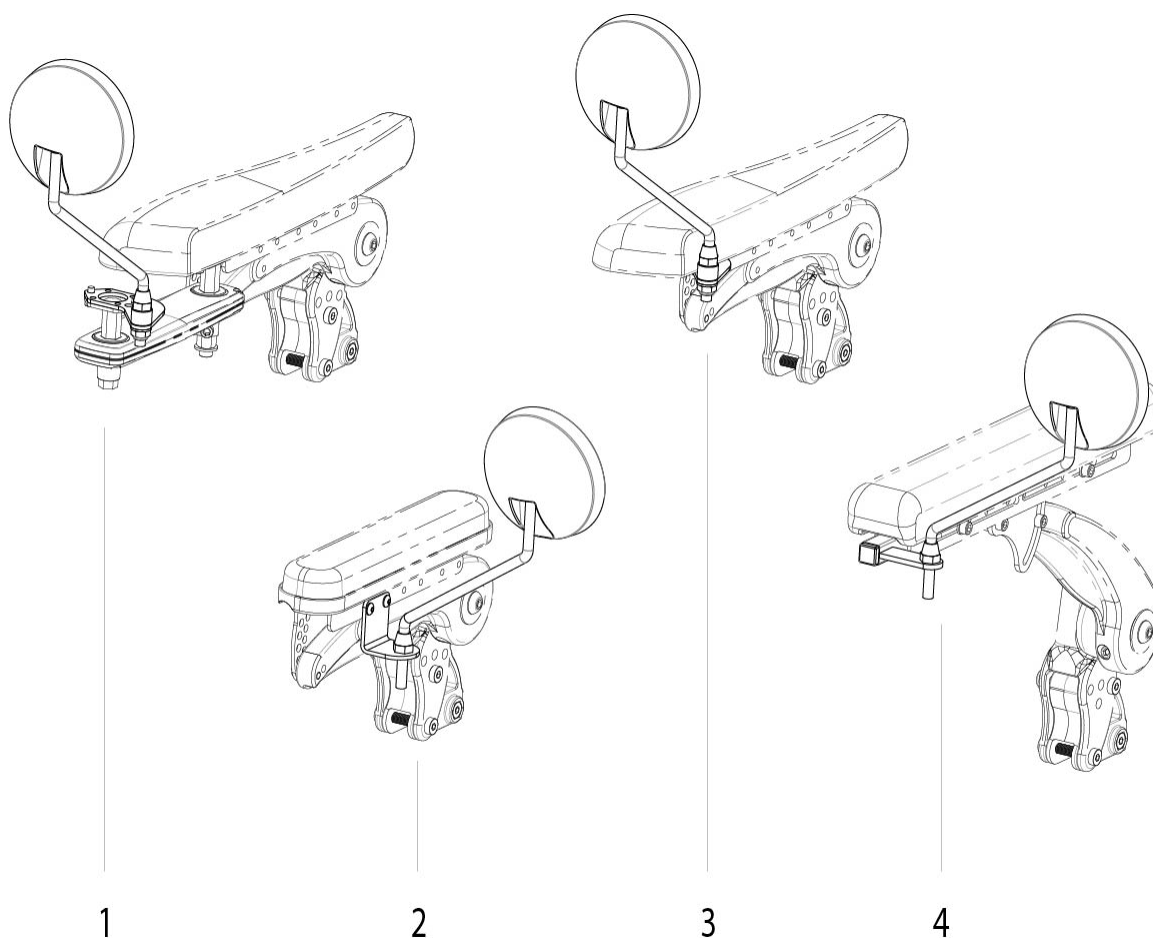

Spare parts

Item no.	Article no.	Description	Remark
2	060611	Rear view mirror	
1	22509080-001	Rear view mirror, same side of joystick	left
1	22509080-002	Rear view mirror, same side of joystick	right
3	22509080-011	Rear view mirror, opposite side of joystick	left
3	22509080-012	Rear view mirror, opposite side of joystick	right
4	22809090-001	Mirror assembly	left
4	22809090-002	Mirror assembly	right

Tray table, swing away

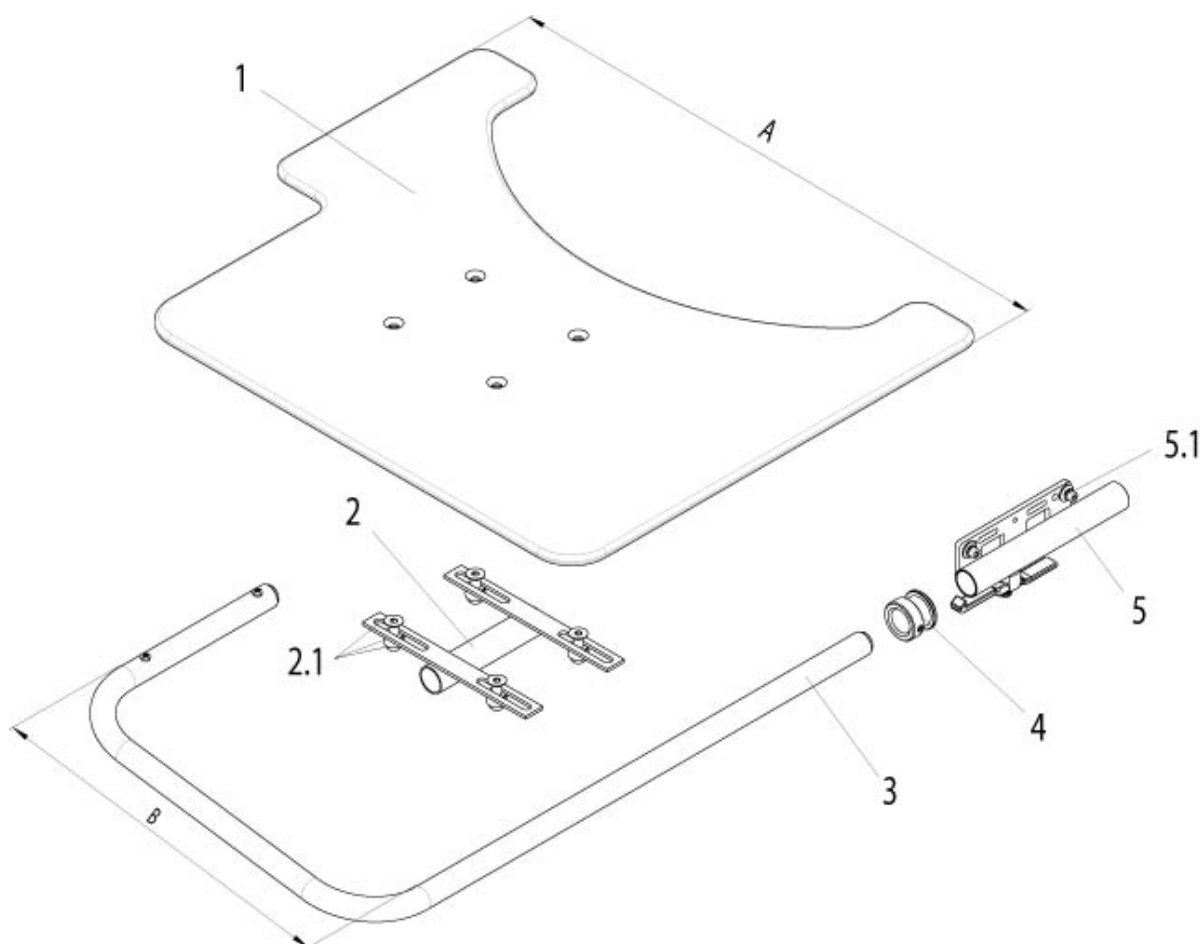

Spare parts

Item no.	Article no.	Description	Remark
3	20009002-210	Stay tube used with swingaway tray, width B = 210 mm	SW 300 - 360 mm
3	20009002-280	Stay tube used with swingaway tray, width B = 280 mm	SW 380 - 400 mm
1	20009003-485	Swingaway tray plate, plastic, thickness = 8 mm, width A = 485 mm	SW 340 - 360 mm
1	20009003-535	Swingaway tray plate, plastic, thickness = 8 mm, width A = 535 mm	SW 380 - 400 mm
2, 2.1	20009009	Swivel, tray table, inclusive mounting material	
2.1	20009010	Swivel mounting material	
5, 5.1	20009021-003	Support used with swingaway	left, note: for all Salsa standard

Tray table, swing away


Item no.	Article no.	Description	Remark
		tray table, centre mount LCD remote control,cpl.cpl.	seat
5, 5.1	20009021-004	Support used with swingaway tray table, centre mount LCD remote control,cpl.cpl.	right, note: for all Salsa standard seat
4	21009128	Clip ring with pin	
1	696000.3706	Swingaway tray plate, plastic, thickness = 8 mm, width A = 485 mm	SW 300 - 320 mm

Shrouds

Spare parts

Description
Q300 M - Battery cover
Coloured shroud elements

Q300 M - Battery cover

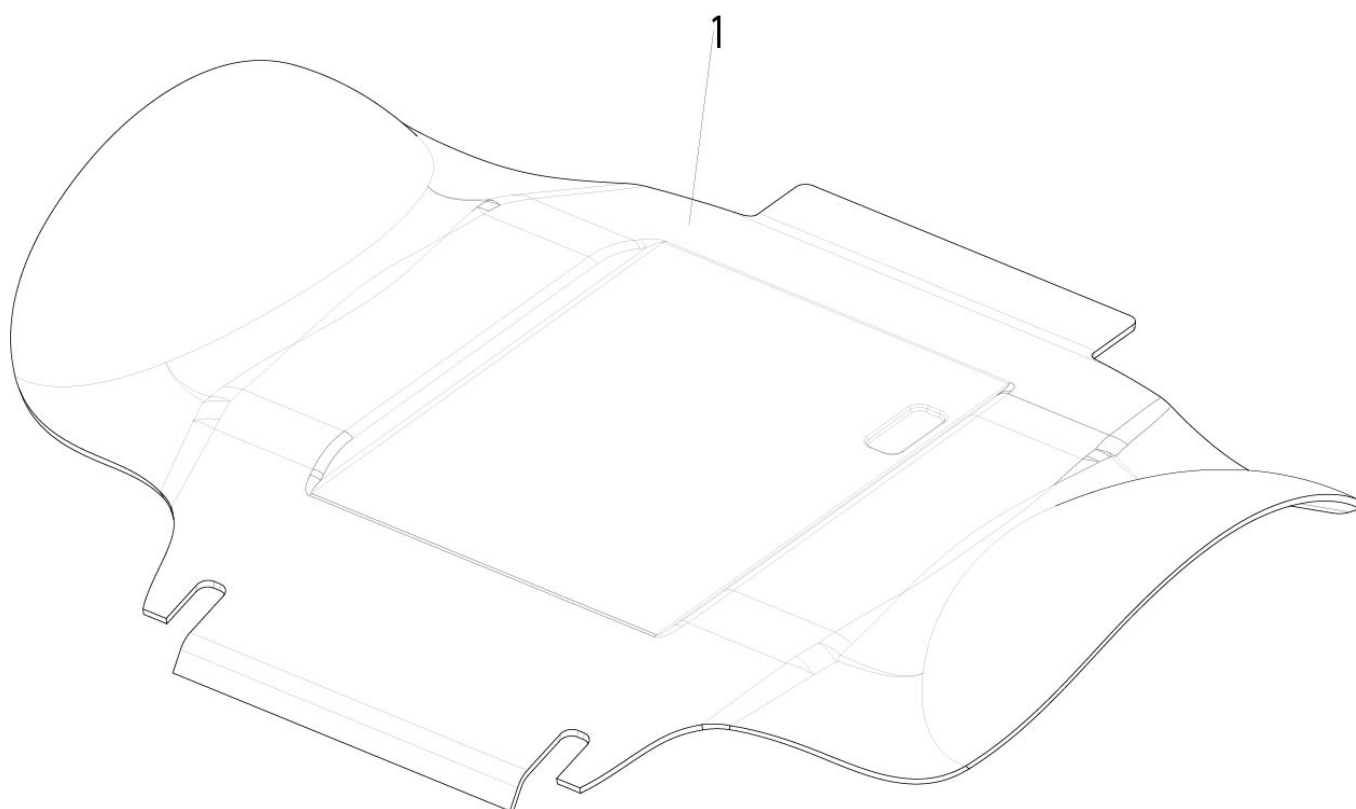

Spare parts

Item no.	Article no.	Description
1	21514650-001	Top shroud - red
1	21514650-002	Top shroud - blue
1	21514650-003	Top shroud - white
1	21514650-004	Top shroud - black
	21514661-001	Top shroud - grey burst - including -Q300M Mini- decals and mounting material
	21514661-002	Top shroud - red burst - including -Q300M Mini- decals and mounting material
	21514661-003	Top shroud - blue burst - including -Q300M Mini- decals and mounting material
	21514661-004	Top shroud - white burst - including -Q300M Mini- decals and mounting material
	21514661-005	Top shroud - flames grey - including -Q300M Mini- decals and mounting material
	21514661-006	Top shroud - flames red - including -Q300M Mini- decals and mounting material

Q300 M - Battery cover

Item no.	Article no.	Description
	21514661-007	Top shroud - flames blue - including -Q300M Mini- decals and mounting material
	21514661-008	Top shroud - flames white - including -Q300M Mini- decals and mounting material
	21514661-009	Top shroud - floral white - including -Q300M Mini- decals and mounting material
	21514661-010	Top shroud - carbon black - including -Q300M Mini- decals and mounting material
	21514661-011	Top shroud - purple burst - including -Q300M Mini- decals and mounting material
	21514661-012	Top shroud - pink burst - including -Q300M Mini- decals and mounting material

Coloured shroud elements



Spare parts


Item no.	Article no.	Description	Remark
	161948	Colour accent kit, black (RAL 9005)	Q500 M / Q400 M / Q300 M
	161949	Colour accent kit, white (RAL 9010)	Q500 M / Q300 M
	161950	Colour accent kit, orange (RAL 060 60 80)	Q500 M / Q400 M / Q300 M
	161951	Colour accent kit, red (RAL 040 40 67)	Q500 M / Q300 M
	161952	Colour accent kit, apple green (RAL 110 70 77)	Q500 M / Q400 M / Q300 M
1	161953	Colour accent kit, blue (RAL 260 40 45)	Q500 M / Q300 M
	162538	Colour accent kit, silver (RAL 9006)	Q500 M / Q400 M

Drive wheel / castor wheel

Spare parts

Description
Q 200 R - 9 Inch Castor Wheel
Q300 M - 13 inch drive wheel, knobbly
14 inch drive wheel, knobbly
12 inch drive wheel, knobbly
Q300 M - 7 inch castor wheel

Q 200 R - 9 Inch Castor Wheel

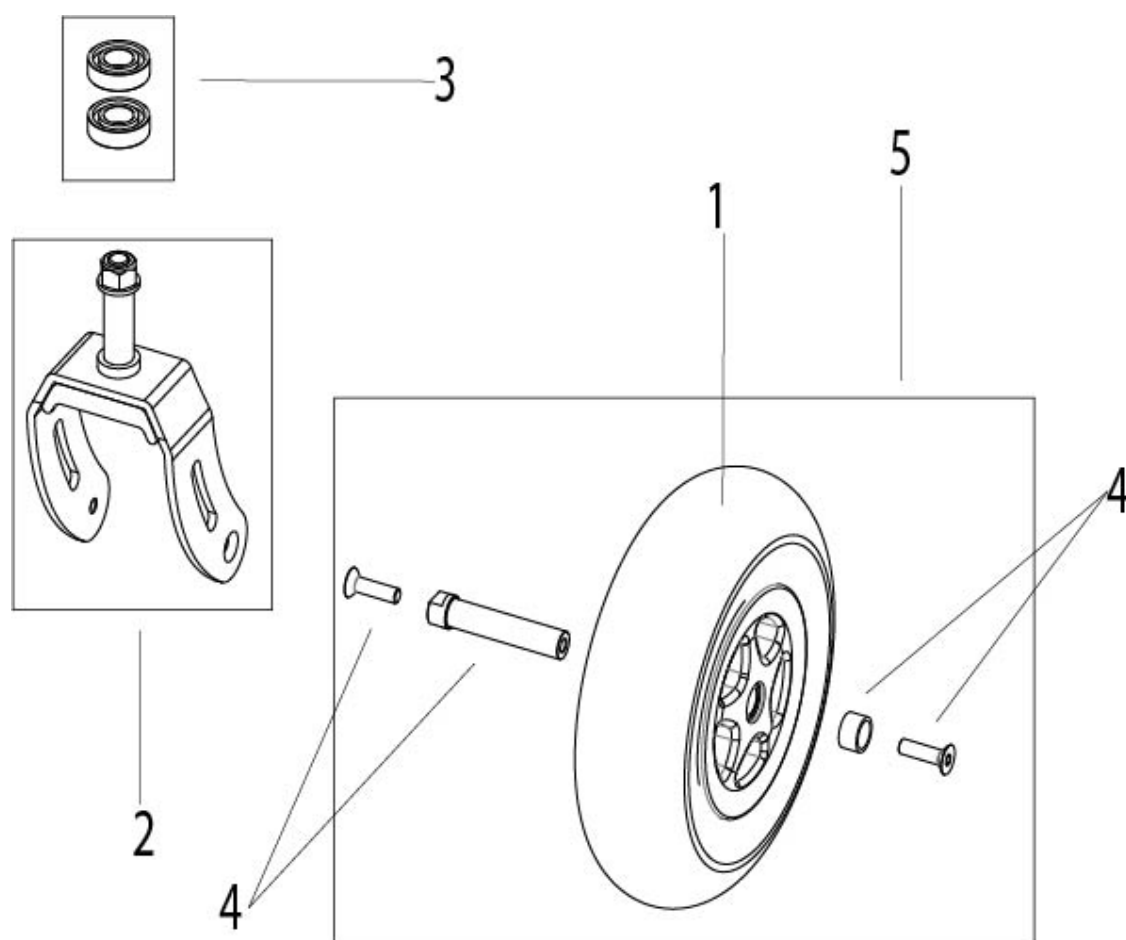


Spare parts

Item no.	Article no.	Description	Remark
2	22608008	Castor fork, 9 inch, black	
3	22608051	Bearing kit castor fork	
5	22608052	Castor wheel 9 x 2.5 inch, solid, black	
4	22700087	9 inch axle, inclusive mounting material	Q700 F/R
1	22708015	9 inch 3 spoke wheel assembly, solid, black	Q400 F/R / Q500 F/R/H / Q700 F/R
	22708018	9 inch tyre, solid (2,8/2,5-4,00) (2,8/2,5-4,00)	
	22708019	9 inch tyre, pneumatic (2,8/2,5-4,00) (2,8/2,5-4,00)	
	22708020	9 inch inner tube for pneumatic castor (2,8/2,5-4,00)	

Q 200 R - 9 Inch Castor Wheel

Item no.	Article no.	Description	Remark
		(2,8/2,5-4,00)	
1	22708026	9 inch castor wheel, 5-spoke, pneumatic, black	Q500 F/R, Q400 F/R, Q 200R


Q300 M - 13 inch drive wheel, knobbly

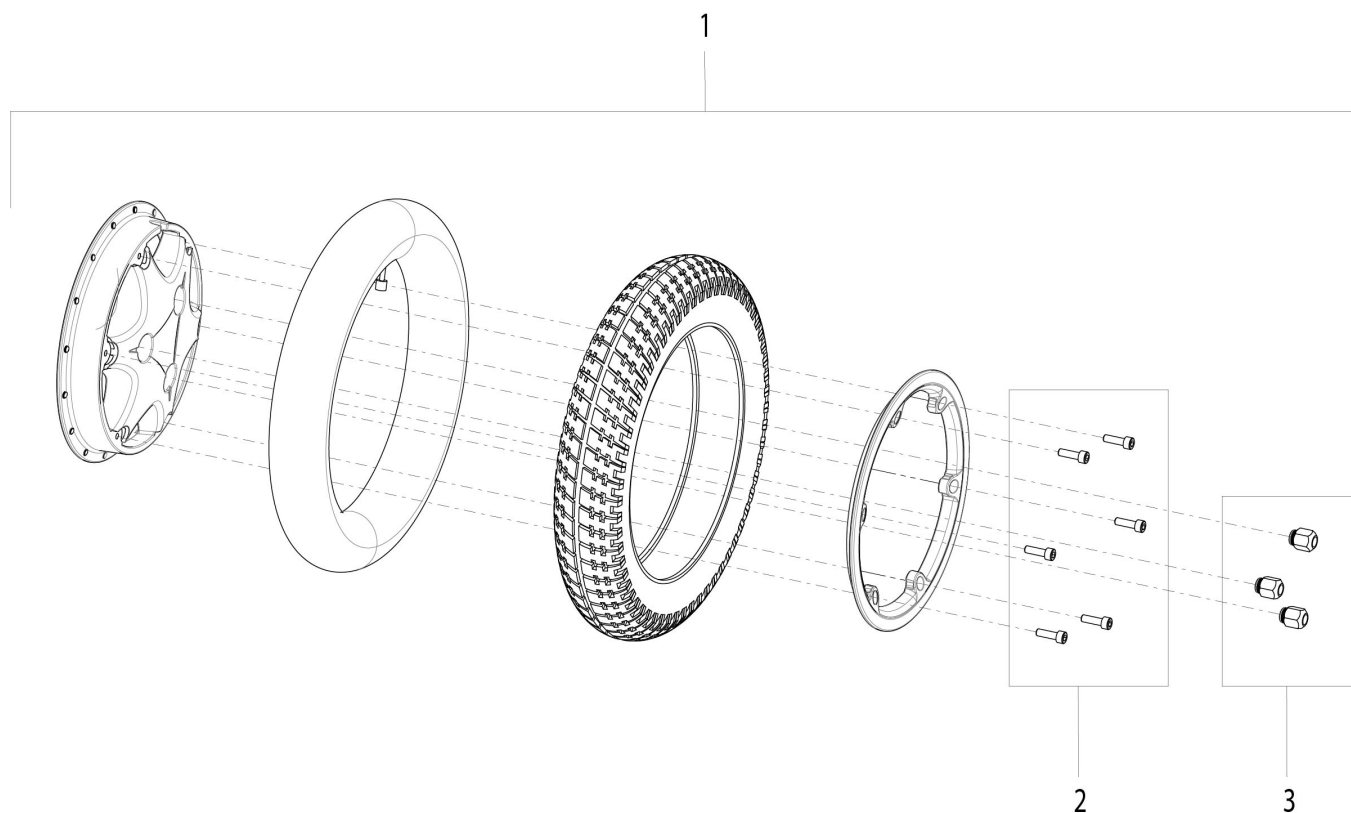

Spare parts

Item no.	Article no.	Description	Remark
4	010498	Inner tube 13 x 2.5 x 8 inch	
7	161965	Mounting kit lug nut M10 x 1.25	3 pcs./pack, (for Drivewheels and Motor with articlenumber: 22710010-013,22710010-014,22710010-015,22710010-016,22710010-017,22710010-018)
6	162361	Mounting kit screw	6 pcs./pack
2	243565	Inner ring - 6 spoke	
3	243567	Outer ring	
1	244823	13 inch drive Wheel, knobbly, pneumatic	
1	244824	13 inch drive Wheel, knobbly, solid	
5	244927	13 inch tyre, solid, black	
5	247671	13 inch tyre, pneumatic, black	

14 inch drive wheel, knobbly

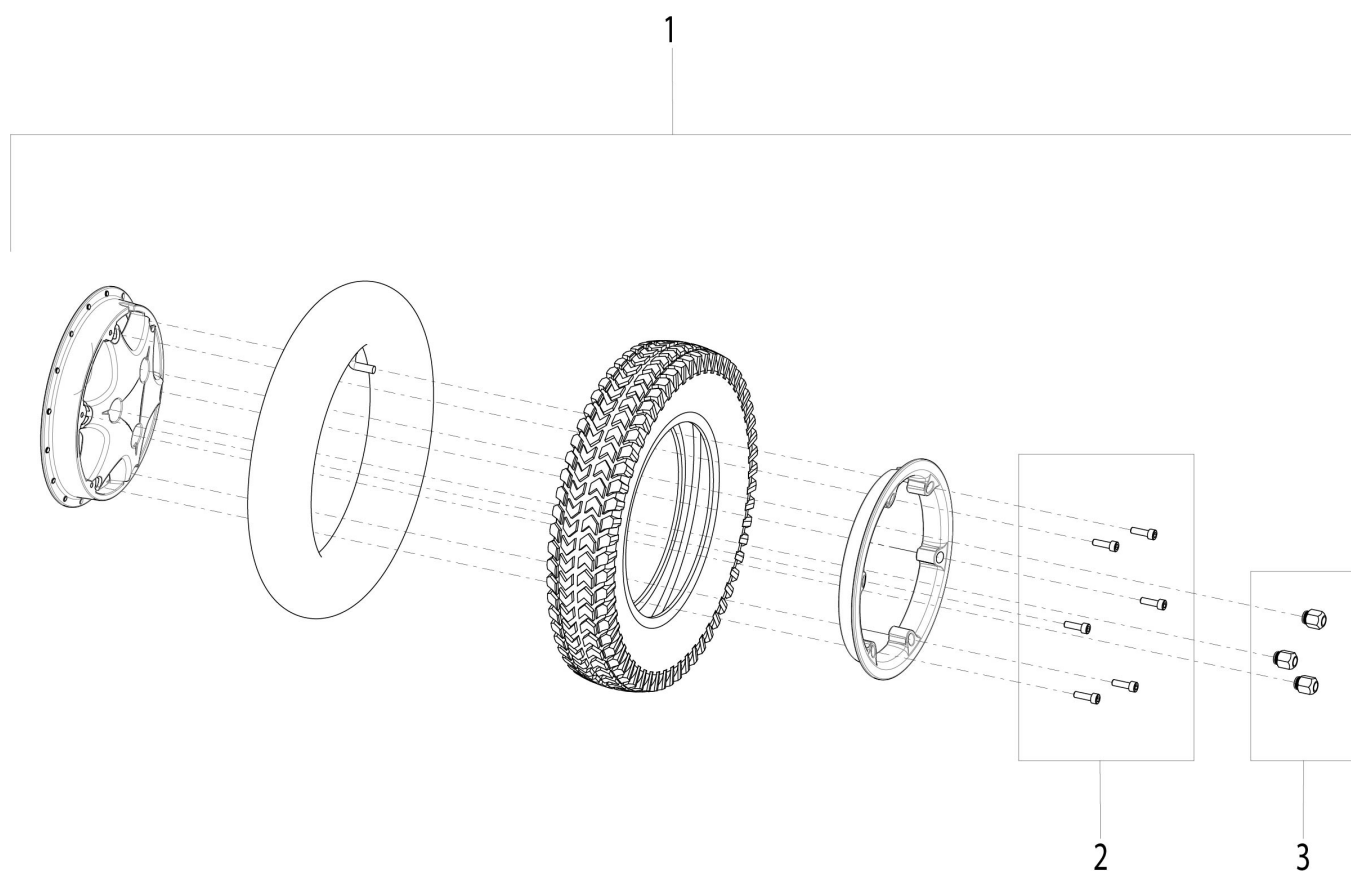


Spare parts

Item no.	Article no.	Description	Remark
5	125284	14in tyre, knobbly, pneumatic, marking on tyre: 3.00-8	
7	161965	Mounting kit lug nut M10 x 1.25	3 pcs./pack, (for Drivewheels and Motor with articlenumber: 227100 10-013,22710010-014,22710010-015,22710010-016,22710010-017,22710010-018)
6	162361	Mounting kit screw	6 pcs./pack
2	243565	Inner ring - 6 spoke	
3	243568	Outer ring	
5	244928	14 inch tyre, knobbly, solid	
1	245399	14 inch drive Wheel, knobbly, pneumatic	
1	245400	14 inch drive wheel, knobbly, solid	

14 inch drive wheel, knobbly

Item no.	Article no.	Description	Remark
4	384739	14 inch inner tube	


12 inch drive wheel, knobbly

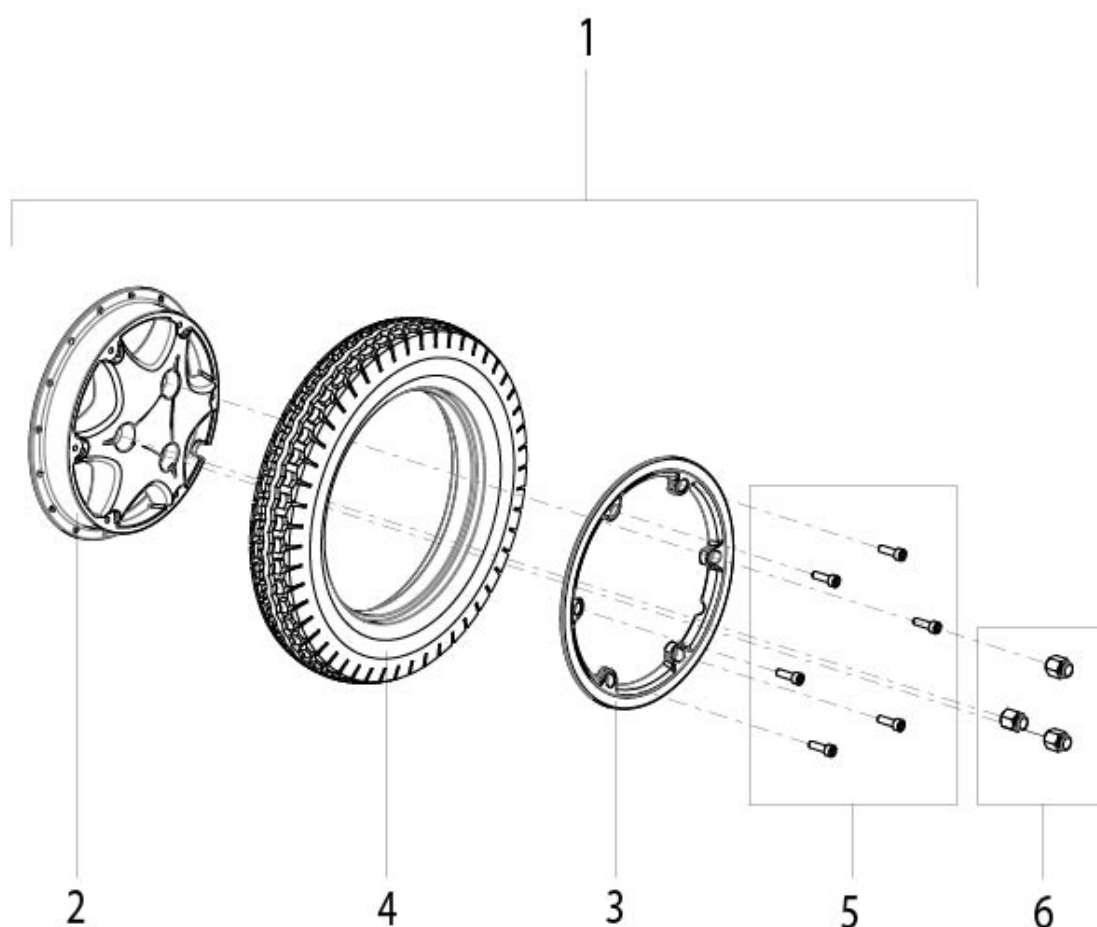

Spare parts

Item no.	Article no.	Description	Remark
6	161965	Mounting kit lug nut M10 x 1.25	3 pcs./pack, (for Drivewheels and Motor with articlenumber: 22710010-013,22710010-014,22710010-015,22710010-016,22710010-017,22710010-018)
1	22607000	12.5 inch, 6 spoke, solid, knobbly drive wheel	
4	22607010	12.5 inch solid tyre, knobbly, 12.5 x 2.25 mm	
5	22607050	Mounting kit screw	6 pcs./pack
2	243565	Inner ring - 6 spoke	
3	243567	Outer ring	

Q300 M - 7 inch castor wheel

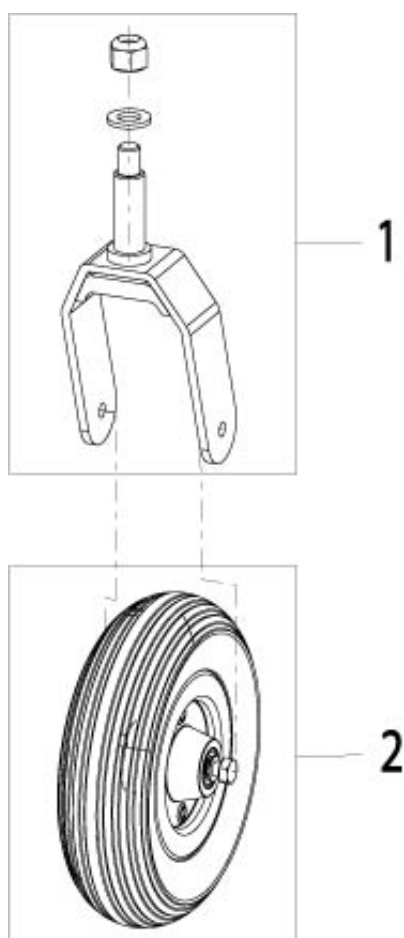

Spare parts

Item no.	Article no.	Description
1	21508003	7 inch castor fork assembly, black
2	21508006	7 inch castor with axle, black rim


Chassis

Spare parts

Description

Q300 M - Chassis

Q300 M - Chassis

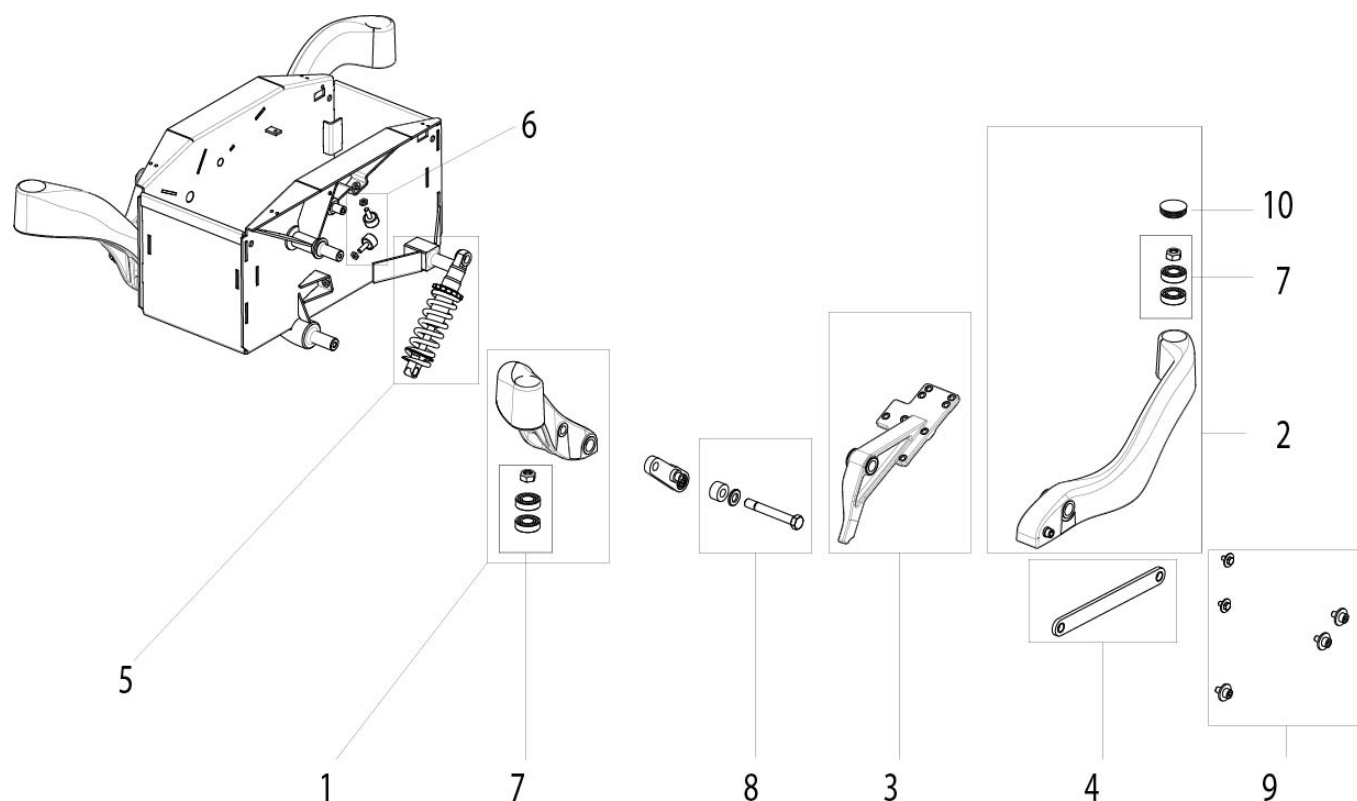

Spare parts


Item no.	Article no.	Description	Remark
6	21501030	Rubber bumper kit	2 pair per chair
4	21501034-001	Midlink assembly	2 per chair
7	21501036	Castor bearing kit (2 bearings, 1 nut)	4 sets per chair
8	21501037	Front stop bushing kit	2 per chair
9	21501038	Screw kit	5 screws, 5 washers
1	21501042-021	Front castor arm	left
	21501042-022	Front castor arm	right
2	21501046-021	Rear castor arm	left
	21501046-022	Rear castor arm	right
10	21501048	Plug for castor bearing	4 per chair
3	21501055-001	Motor mount kit	left
	21501055-002	Motor mount kit	right
5	21515000	Suspension spring 350 LBS	2 per chair

Looms

Spare parts

Description
Looms and electronic
Battery looms
R-Net bus cable
Actuator looms
Inhibit looms
Light looms

Looms and electronic

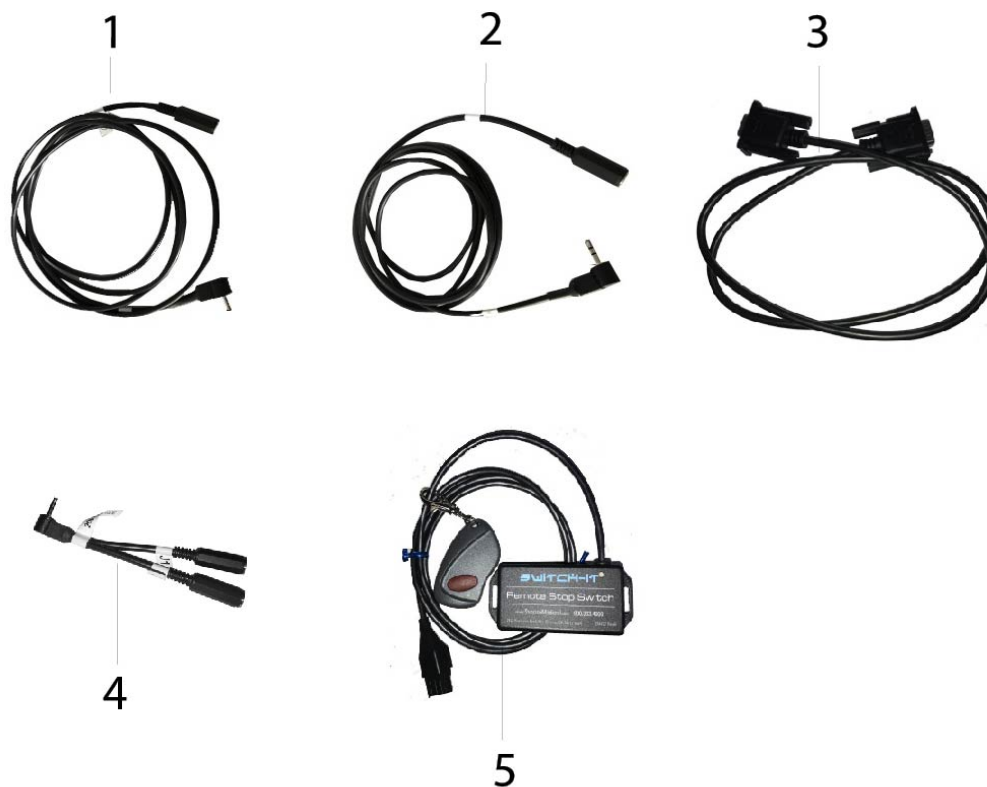

Spare parts

Item no.	Article no.	Description
3	015526	DB9 extension cable 800 mm
1	015734	Cordon jack mono 3.5 mm, 2000 mm long
4	20011063	Y - loom, 2 female mono to 1 male stereo
2	20011065	Stereo extension cable
5	RSS-OBC	Remote stop switch

Battery looms



Spare parts

Item no.	Article no.	Description	Remark
1	060107	Battery terminal loom	2 pcs. per wheelchair, used with 55 Ah battery
2	065146	R-Net / VR2 connection loom controller / battery with auxiliary	note: R-Net / VR2 Salsa starting with manufacturing date 2011-04-19
	065146UK	R-Net / VR2 connection loom controller / battery without auxiliary	

R-Net bus cable



Spare parts

Item no.	Article no.	Description	Remark
2	015993	Extension loom, R-Net BUS, 1000 mm	
1	108316-030	Harness R-Net BUS, 300 mm	
1	108316-050	Harness R-Net BUS, 500 mm	
1	108316-100	Harness R-Net BUS, 1000 mm	
1	108316-120	Harness R-Net BUS, 1200 mm	
1	108316-150	Harness R-Net BUS, 1500 mm	
1	108316-200	Harness R-Net BUS, 2000 mm	
1	108316-250	Harness R-Net BUS, 2500 mm	
2	108557-030	Extension loom, R-Net BUS, 300 mm	
3	20011032-120	R-Net loom with angle plug, 1200 mm	Note: only in use with Joystick LCD, Heavy Duty Article number:

R-Net bus cable

Item no.	Article no.	Description	Remark
			20011031
5	20011034	R-Net Y-loom	
4	20011035	R-Net loom with angle plug, 300 mm	

Actuator looms




Spare parts

Item no.	Article no.	Description
1	020063-060	Actuator loom extension, 600 mm
1	020063-100	Actuator loom extension, 1000 mm
1	020063-120	Actuator loom extension, 1200 mm
2	21211004-090	Actuator loom direct to ISM, 900 mm
2	21211004-120	Actuator loom direct to ISM, 1200 mm
2	21211004-135	Actuator loom direct to ISM, 1350 mm
2	21211004-200	Actuator loom direct to ISM, 2000 mm
2	21211004-225	Actuator loom direct to ISM, 2250 mm
3	21211005-060	Actuator extension loom, 600 mm
6	21211005-120	Actuator extension loom, 1200 mm
4	21511651	Actuator extension loom, 300 mm

Actuator looms

Item no.	Article no.	Description
4	22511004	Legrest harness controller side
5	22511005	Legrest harness actuator side

Inhibit looms

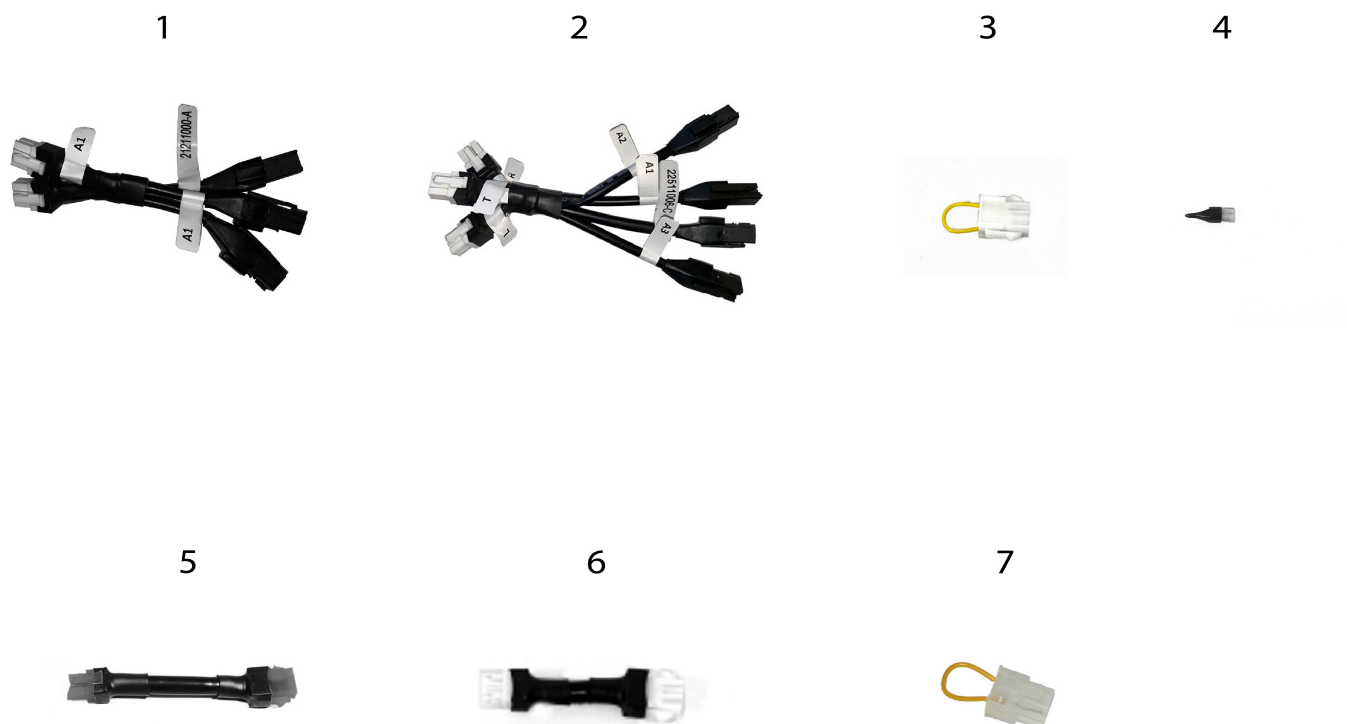

Spare parts

Item no.	Article no.	Description
7	015585	Actuator inhibit loop
5	060389	Inhibit loop (in line)
3	060393	Inhibit loop terminator
6	060663	Inhibit resistor
4	060665	VR2 Inhibit loom
1	21211000	Actuator adaptor loom
2	22511006	Actuator inhibit loom

Light looms



Spare parts

Item no.	Article no.	Description	Remark
1	21511650-001	Light loom R-Net	right
	21511650-002	Light loom R-Net	left
2	21511652	Light loom VR2	


Lights

Spare parts

Description

Q300 M - LED lights and indicators (StVZO)
--

Q300 M - LED lights and indicators (StVZO)

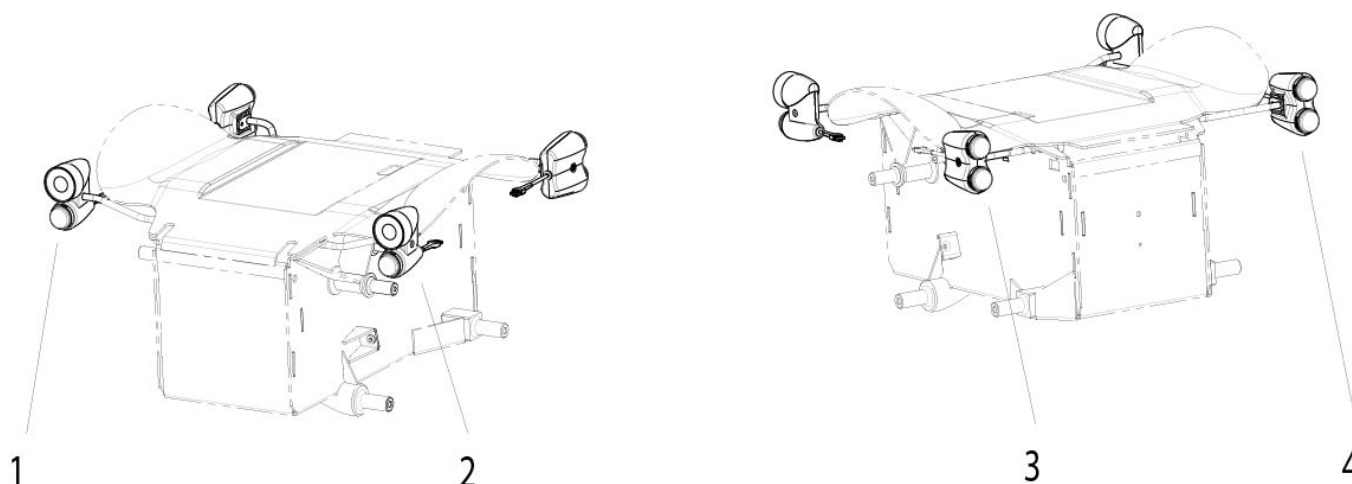

Spare parts


Item no.	Article no.	Description	Remark
1	21509000-001	Front LED lights and indicator	right, Note: SALSA 2, ZIPPIE SALSA 2 - due to a design change in March 2016 we recommend to replace the lights as pair
2	21509000-002	Front LED lights and indicator	left, Note: SALSA 2, ZIPPIE SALSA 2 - due to a design change in March 2016 we recommend to replace the lights as pair
4	21509600-001	Rear LED lights and indicator	right
3	21509600-002	Rear LED lights and indicator	left

Motor

Spare parts

Description
Motor 2 - pole standard
Q300 M - Motor 4 - pole
Q300 M - Motor 2 - pole performance


Motor 2 - pole standard

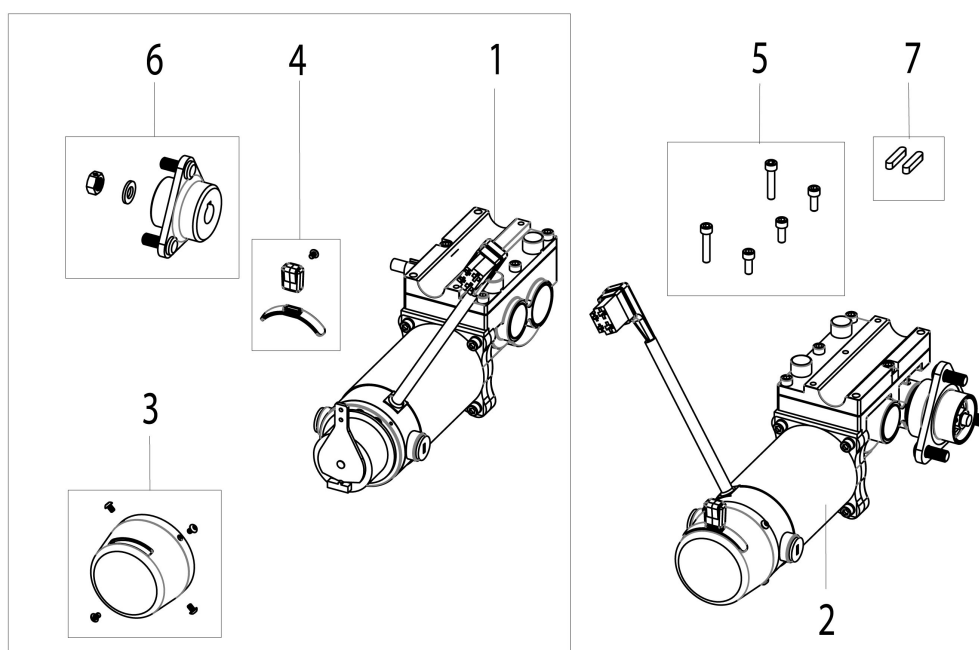

Spare parts

Item no.	Article no.	Description	Remark
5	22610001	Motor fastener kit	
7	22610002	Key	2 pcs./pack
3	22610003	Brake cover	
4	22610004	Freewheel lever	
6	22610005	Wheel hub	
	245244	Motor brush kit	4 pcs./pack
2	247356	Motor assembly 6 kph 2-pole standard	MWD/FWD = right, RWD = left
1	247357	Motor assembly 6 kph 2-pole standard	MWD/FWD = left, RWD = right

Q300 M - Motor 4 - pole

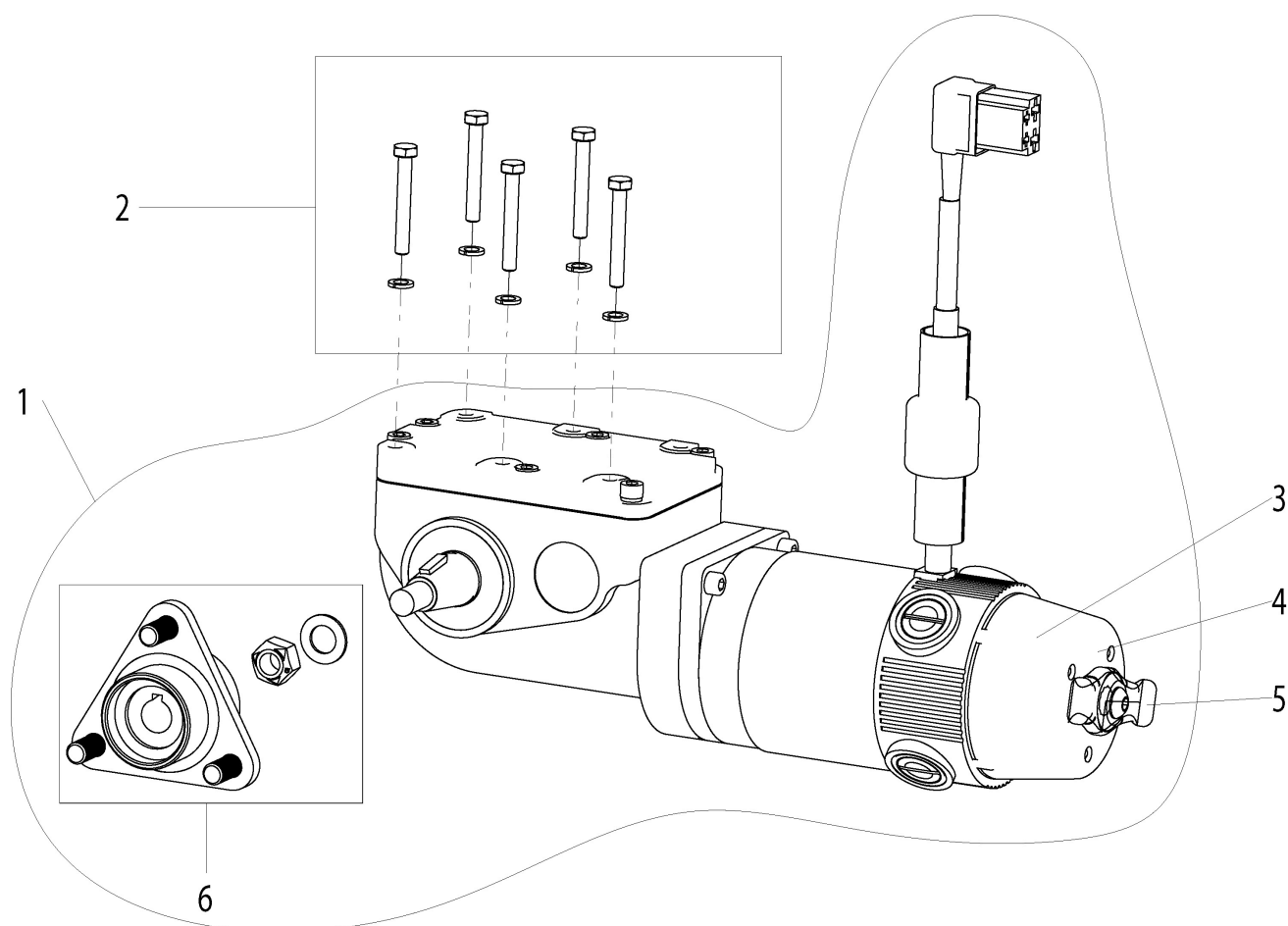


Spare parts

Item no.	Article no.	Description	Remark
	112281	Motor brush kit for 4 - pole motor	4 pcs./pack
3	112283	Brake cover, inclusive mounting material	
3-4	112770	Brake cpl.	MWD/FWD = right, RWD = left, (by Sedeo Pro/Lite/Ergo/UP/Q300 for articlenumber: 22710010-002, 22710010-004,22710010-006)
3-4	112771	Brake cpl.	MWD/FWD = left, RWD = right (by Sedeo Pro/Lite/Ergo/UP/Q300 for articlenumber: 22710010-001, 22710010-003,22710010-005)
5	162370	Brake release lever kit	for Motor (by Sedeo Pro/Lite/Ergo/UP/Q300 for articlenumber: 22710010-001,22 710010-002,22710010-003,2271 0010-004,22710010-005,227100

Q300 M - Motor 4 - pole

Item no.	Article no.	Description	Remark
			10-006)
2	20010990-002	Motor fixing kit	MWD 4-pole
2	20010990-004	Motor fixing kit	Q700 FWD/RWD, for one 4-pole Motor
6	22700083	Hub assembly for motor	
1	22710010-003	Motor, 4 pole, 8 kph (heavy duty 6kph), Motor P/N: 241236	RWD = right, MWD / FWD = left, (Identify the order number via the P/N on themotor)
1	22710010-004	Motor, 4 pole, 8 kph (heavy duty 6kph), Motor P/N: 241237	RWD = left, MWD / FWD = right, (Identify the order number via the P/N on themotor)

Q300 M - Motor 2 - pole performance

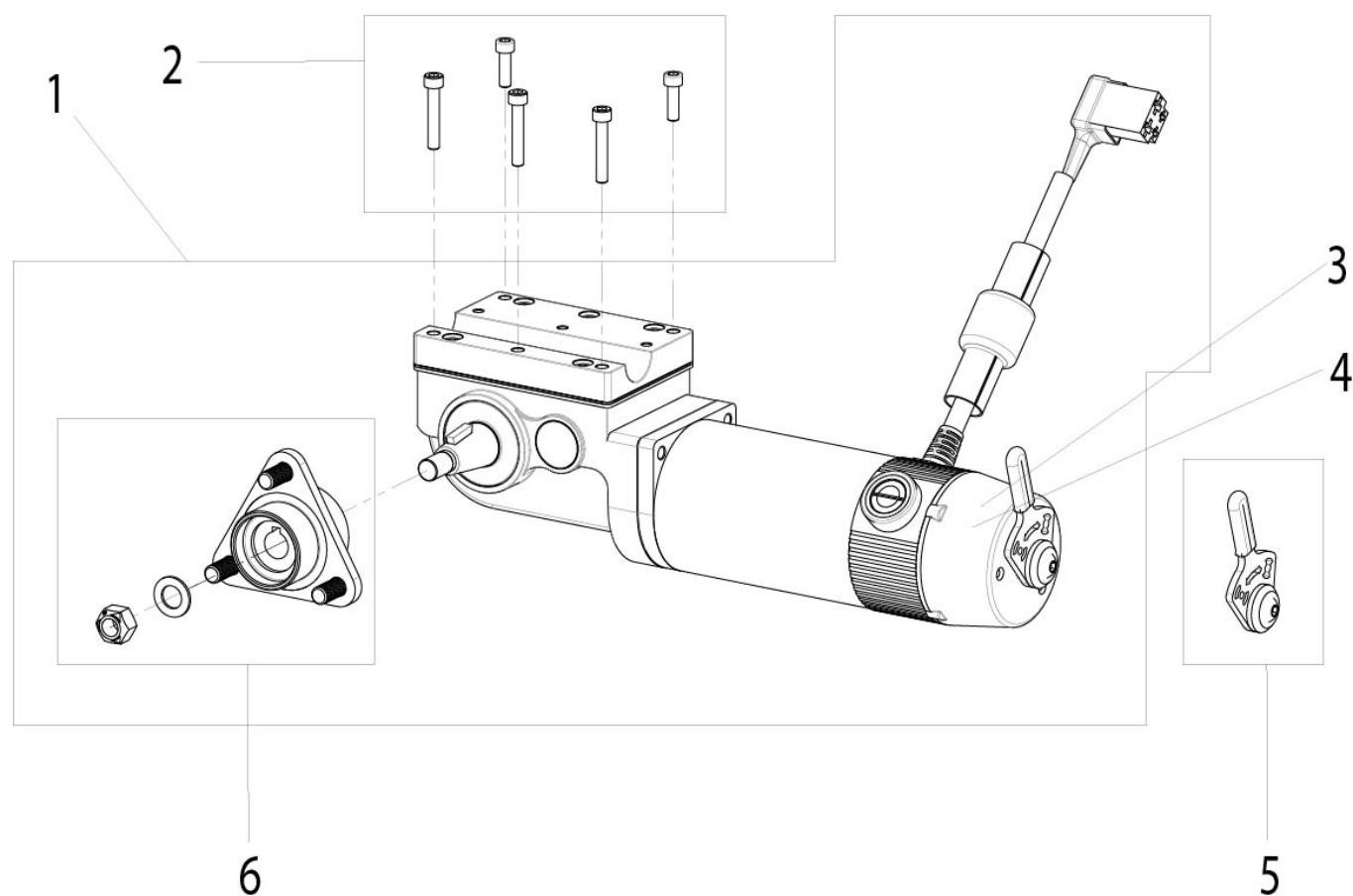

Spare parts

Item no.	Article no.	Description	Remark
5	061543	Brake release lever kit	
	112282	Brush kit 2 - pole, inclusive caps	for one motor
3	112283	Brake cover, inclusive mounting material	
3-4	112770	Brake cpl.	MWD/FWD = right, RWD = left, (by Sedeo Pro/Lite/Ergo/UP/Q300 for articlenumber: 22710010-002, 22710010-004,22710010-006)
3-4	112771	Brake cpl.	MWD/FWD = left, RWD = right (by Sedeo Pro/Lite/Ergo/UP/Q300 for articlenumber: 22710010-001, 22710010-003,22710010-005)
2	20010990-001	Motor fixing kit	MWD 2-pole
2	20010990-003	Motor fixing kit	FWD/RWD 2-pole
	21510010-001	Motor, 2-pole 6 kph	MWD / FWD = left, RWD = right

Q300 M - Motor 2 - pole performance


Item no.	Article no.	Description	Remark
1	21510010-002	Motor, 2-pole 6 kph	MWD / FWD = right, RWD = left
	21510010-003	Motor, 2-pole 10 kph	MWD / FWD = left, RWD = right
1	21510010-004	Motor, 2-pole 10 kph	MWD / FWD = right, RWD = left
6	22700083	Hub assembly for motor	

Control

Spare parts

Description
R-Net electronic components
R-Net control modules
VR2 control modules
Dongle


R-Net electronic components

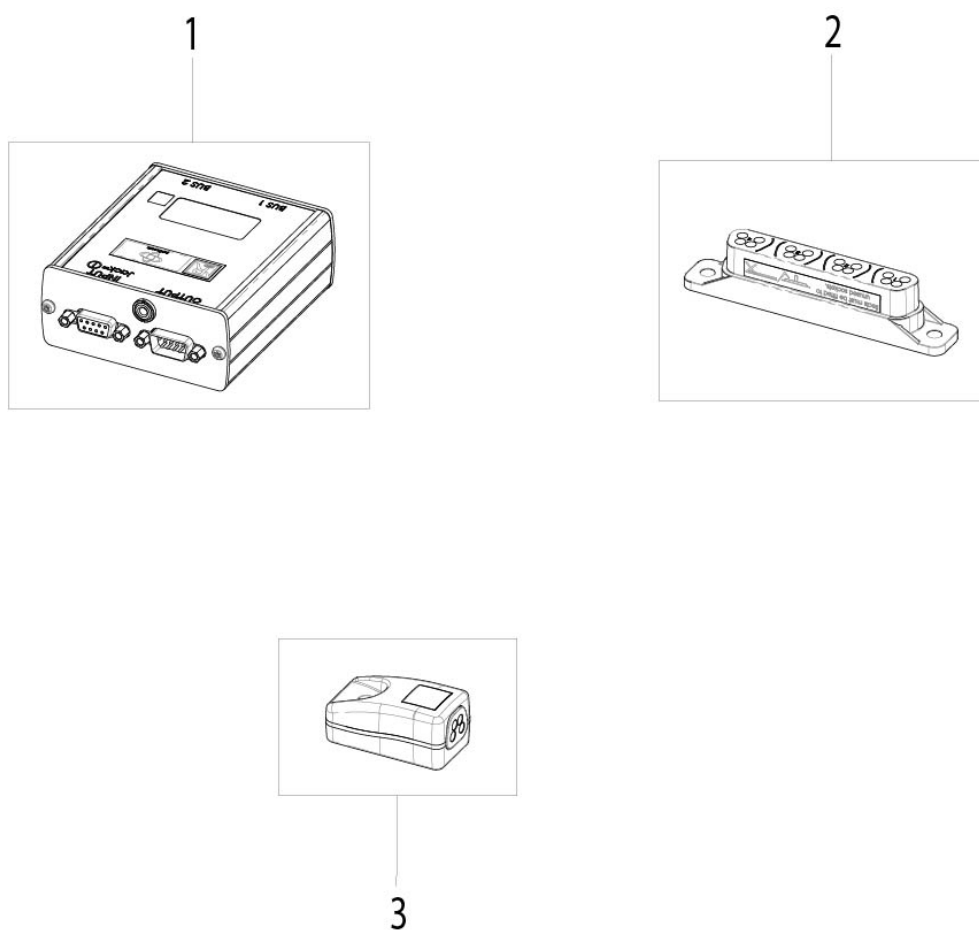

Spare parts

Item no.	Article no.	Description
1	108217	IOM (input-output module) for 7 switch functions
2	108312	R-Net distribution block
3	21809080	Gyro module kit


R-Net control modules

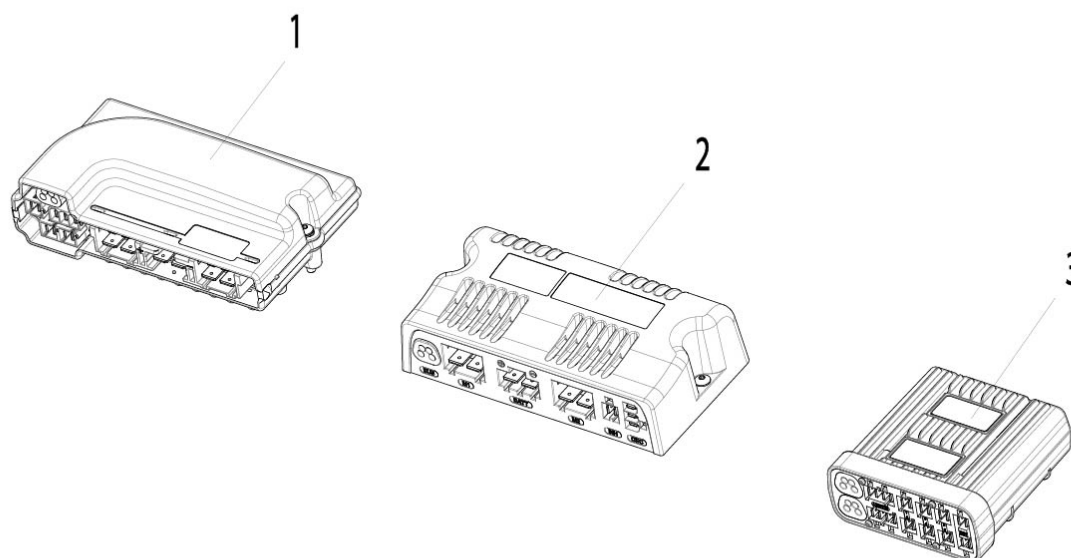

Spare parts

Item no.	Article no.	Description	Remark
3	015583	R-Net ISM (intelligent seating module), lights only	
1	20011080-003	R-Net Power module 90 A, without programming	
2	20011080-004	R-Net Power module 120 A, without programming	
1	20011081-003	R-Net Power module 90 A, with programming	Note: please note serial number when ordering
2	20011081-004	R-Net Power module 120 A, with programming	Note: please note serial number when ordering
3	21011013	R-Net ISM-6L	Note: 6 actuators and lights

VR2 control modules

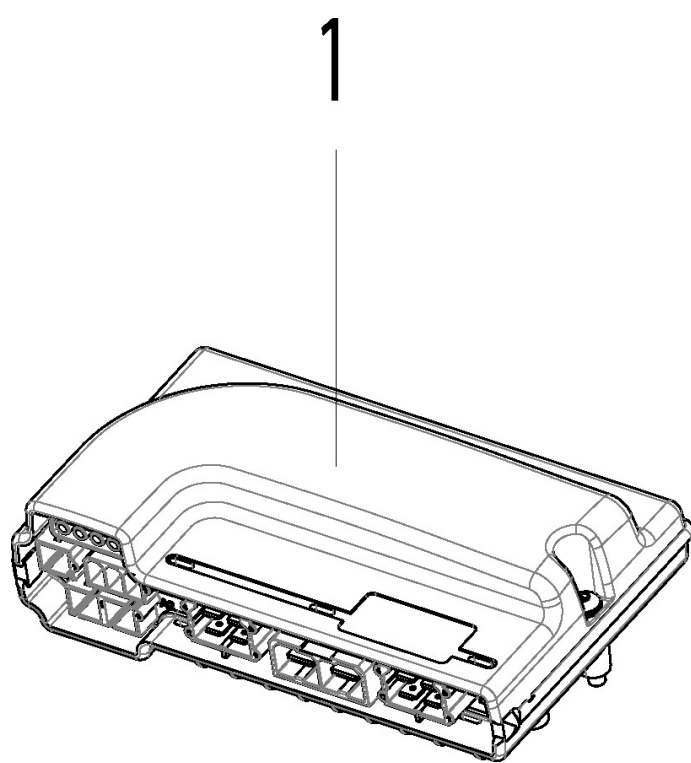

Spare parts

Item no.	Article no.	Description	Remark
1	20011080-001	VR2 Power module 70 A, without programming	
1	20011080-002	VR2 Power module 90 A, without programming	
1	20011081-001	VR2 Power module 70 A, with programming	Note: please note serial number when ordering
1	20011081-002	VR2 Power module 90 A, with programming	Note: please note serial number when ordering
2	21011027	VR2 lighting module	

Dongle



Spare parts

Item no.	Article no.	Description	Remark
	000501044	VR2 PC programmer software (D501044/3)	
	015574	R-Net programming cable DTT	
	015886	VR2 programming cable, DTT	
1	108532	R-Net PC programmer (dealer version), inclusive USB cable	Supplied with software, dongle and cable

Sunrise Medical GmbH

Kahlbachring 2-4
D-69254 Malsch/Heidelberg
Germany
Tel.: +49 (0) 7253 980-0
Fax: +49 (0) 7253 980-222
E-Mail: kundenservice@sunrisemedical.de
www.sunrisemedical.de

Sunrise Medical AG

Erlenauweg 17
CH-3110 Münsingen
Switzerland
Tel.: +41 (0)31 958 3838
Fax: +41 (0)31 958 3848
E-Mail: info@sunrisemedical.ch
www.sunrisemedical.ch

Sunrise Medical SAS

ZI de la Vrillonnerie
17 Rue Michaël Faraday
F-37170 Chambray-Lès-Tours
France
Telephone: +33 (0) 2 47 55 44 00
Fax: +33 (0) 2 47 55 44 03
Email: info@sunrisemedical.fr
www.sunrisemedical.fr

Sunrise Medical S.L.

Polígono Bakiola, 41
48498 Arrankudiaga
Vizcaya
España
Teléfono: 902 14 24 34 / 94 633 31 92
Fax.: 94648 15 75
www.sunrisemedical.es