

# WHEELCHAIR LIFTING GUIDANCE



## FOR CHAIR OWNER

Please mark and label:

1. Motor freewheel (brake release) lever
2. Safe points for lifting chair (e.g. lift from the frame and not the armrests)
3. Circle items that will be removed from wheelchair and stowed in a dedicated cabin bag

## BRAKE RELEASE



Fold or cut page here

Fold or cut page here

Powered Wheelchair Airline Travel Tag		
Manufacturer:		
Model:		Serial Number:
Manoeuvrability:		
Driving With Controller <input type="checkbox"/> Push In Freewheel Only <input type="checkbox"/>		
Joystick Safety:		
Ensure powered off and use cover <input type="checkbox"/>		
Unplug cable in transit near joystick controller <input type="checkbox"/>		
Locking key available? Yes <input type="checkbox"/> No <input type="checkbox"/>		
Other:		
Removable Parts	Will stay on w/c	Stow in Cabin
Seat Cushion	<input type="checkbox"/>	<input type="checkbox"/>
Head Support	<input type="checkbox"/>	<input type="checkbox"/>
Arm Supports	<input type="checkbox"/>	<input type="checkbox"/>
Leg Supports	<input type="checkbox"/>	<input type="checkbox"/>
Control Device	<input type="checkbox"/>	<input type="checkbox"/>
Secondary Control Device (Joystick, Attendant Joystick)	<input type="checkbox"/>	<input type="checkbox"/>
Backrest	<input type="checkbox"/>	<input type="checkbox"/>
Tray	<input type="checkbox"/>	<input type="checkbox"/>
Belts/Straps	<input type="checkbox"/>	<input type="checkbox"/>
Other:		

Owner's Personal Information
Name:
Email:
Mobile:
Flight Number:



Wheelchair weight:
Wheelchair weight without removable components:
Back folds down? Yes <input type="checkbox"/> No <input type="checkbox"/>
Can back be reclined to reduce height? Yes <input type="checkbox"/> No <input type="checkbox"/>
Can entire seat tilt back to reduce height? Yes <input type="checkbox"/> No <input type="checkbox"/>
Release lever for free wheel marked on chair? Yes <input type="checkbox"/> No <input type="checkbox"/>

Battery Type
<input type="checkbox"/> IATA-approved batteries (see owner's manual for further details):
IATA approved Lithium-Ion batteries on a compact folding powered wheelchair (with these options underneath):
<input type="checkbox"/> Wheelchair must be folded and transported in original box, with batteries discharged to 30% or lower.
<input type="checkbox"/> Carbon: Lithium Ion batteries less than 300Wh combined (less than 160Wh each). Move to blank spare battery compartment for transport, discharge to 30% capacity. Transport folded wheelchair in original box or approved protective container.