

# ATTENTION

## PLEASE HANDLE WITH CARE

### Please take care of the following when handling this equipment:

Powered wheelchairs can be heavy, see equipment tag for details, requires multiple-person lift. Lift only from the base of the wheelchair and never from the armrests.

Transport in upright orientation only and **do not put the wheelchair on the side**. Stowing or transporting on the side could cause damage to components.

Ensure the controller (joystick) is **turned off** and there are no lights illuminated.

**Lock/unlock** – levers on the motors lock the wheels, allowing the chair to move via the electronic controller only, and stopping the wheelchair once turned off. Do not attempt to push the chair manually when the locks are engaged.

Further Instructions:

### Contact and Chair Info

Name: \_\_\_\_\_

Email: \_\_\_\_\_

Phone number: \_\_\_\_\_

Chair model: \_\_\_\_\_ Colour: \_\_\_\_\_

Battery Type: \_\_\_\_\_ Weight: \_\_\_\_\_

Please refer to your owner's manual and contact your authorised Sunrise Medical provider for further details on safe transportation.