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Camo Forest

Carbon Fibre

Note: Colours shown on screen may not exactly match product. Please see physical product for actual colour.

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Liner fits in between tube and tyre, providing a robust protective barrier

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Note: Max. tilt angle may be slightly restricted on some seating setups to prevent impact with drive wheels

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FLOOR TO SEAT HEIGHT

-25 front, +25 rear: 410mm

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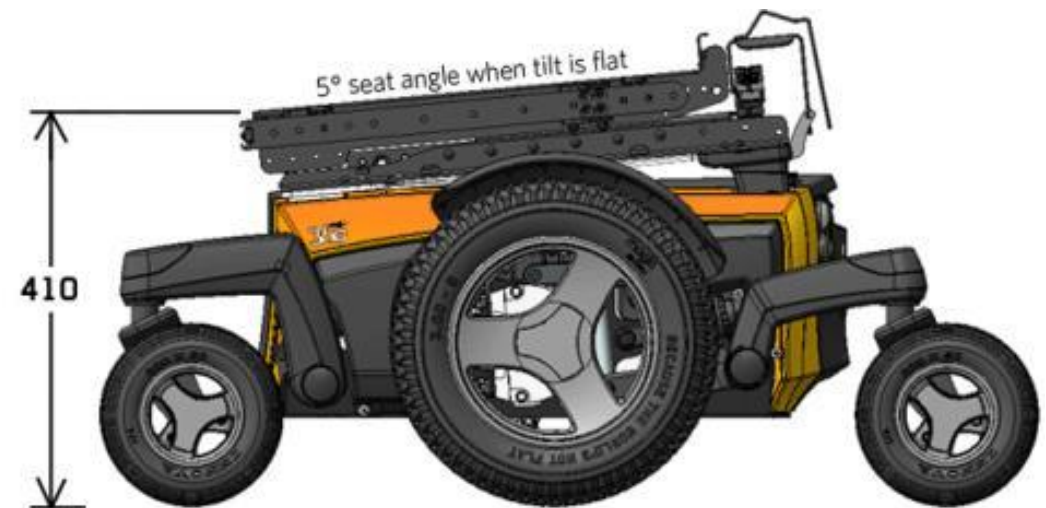
+25: 460mm

+50: 485mm

USO mount brackets are adjustable upwards to change seat height

Std

435mm FTS Height

-25 front, +25 rear

Std


MINIMUM recommended spacing based on legrest length:

Legrest Length	Spacing
0-370	-
375-400	25
405+	50

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
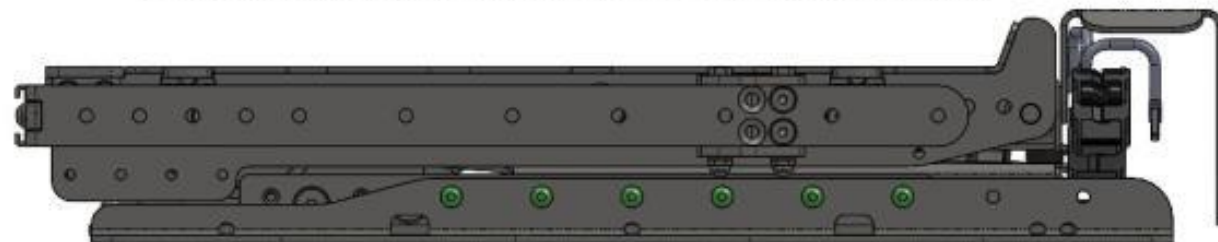
GOOD FOR GETTING MORE WEIGHT ON DRIVE WHEELS ON REAR WHEEL DRIVE

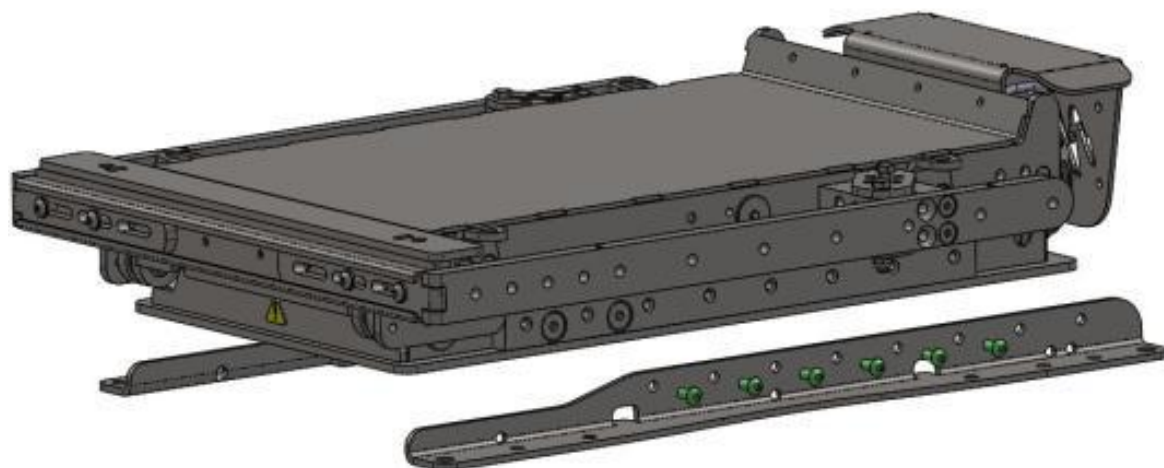1. Undo M6 screws on sides (some may need to be heated to help release Loctite)

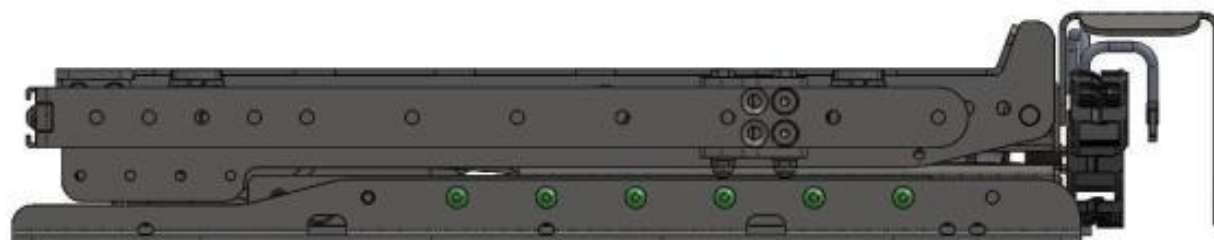2. Shift Lift and Tilt unit back one hole position (45mm)

3. Add Loctite to M6 screws and fit back in place

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© Magic Mobility 2016

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Reflective strips on power module cover

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Rear lights on power module cover
Front lights on base

- Front lights sprung mount to protect from impact
- Front lights will point forward regardless of tilt angle or seat lift

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Rear lights on power module cover
Front lights on side rail

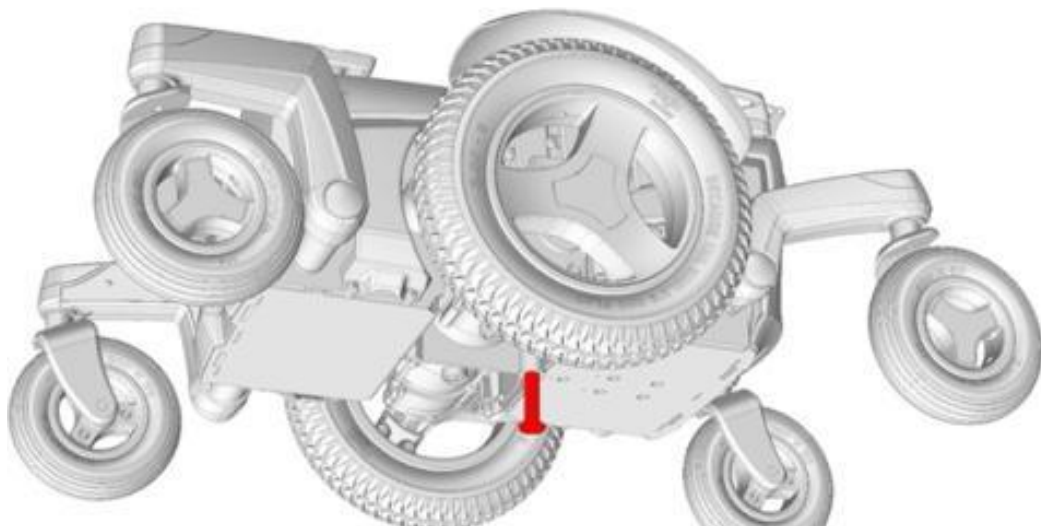
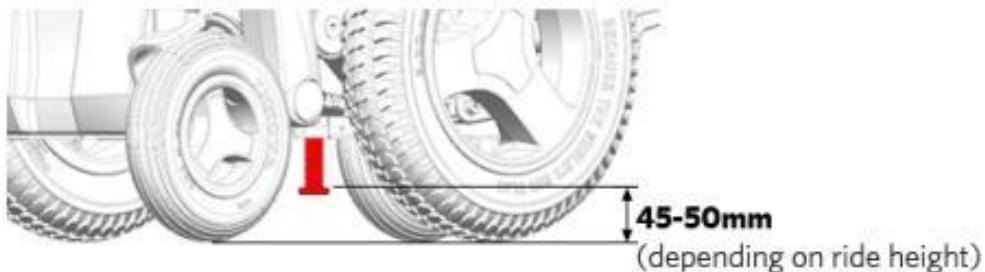
- Front lights will follow tilt angle and seat lift

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
- Magic 360 has been successfully crash tested to ISO 7176-19 with Retractable Docking Pin
- **Maximum user weight 136kg**
- Pin diameter is $\varnothing 16\text{mm}$ and outer head of pin is $\varnothing 28\text{mm}$. Head is 6mm thick.
- In **up** position, bottom of pin is 5mm below bottom of battery box
- In **down** position, approx. distance from **top face of pin to ground** is 47mm. Many aftermarket docks have an approx. engagement height of $\leq 40\text{mm}$, so therefore, may need spacing up


Docking pin position comparison between Magic 360 and other models:

With seat unit in same position, dimension 'X' is as follows:

V6 All Terrain/Hybrid:	121mm
V6 C40/C73:	121mm
V4 FWD:	171mm
V4 RWD:	-68mm

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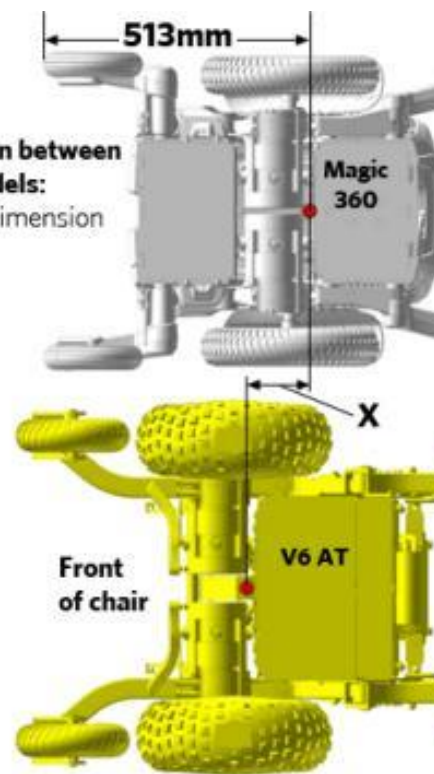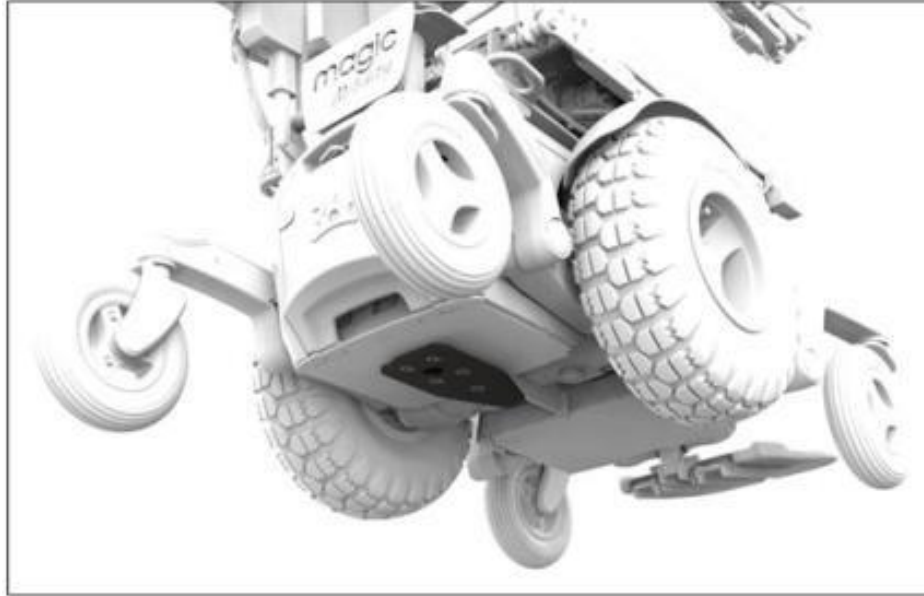
Dahl docking plate supplied fitted to Magic 360 by Magic Mobility

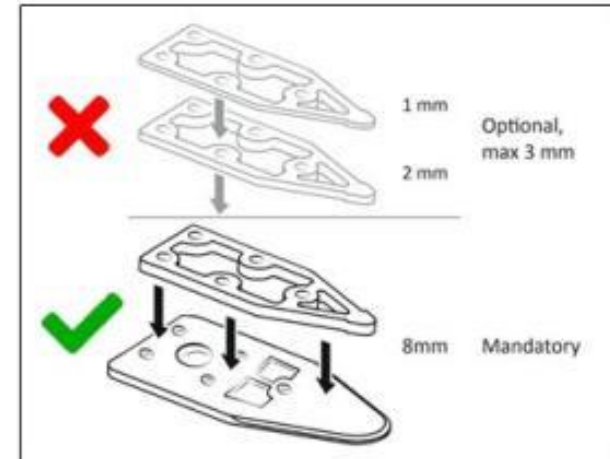- Magic 360 has been successfully crash tested to ISO 7176-19 with Dahl docking plate
- **Maximum user weight 136kg**
- 8mm spacer fitted as standard. Optional spacers not fitted unless specifically requested
- Dahl dock and associated accessories **not** supplied by Magic Mobility

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

Some dock manufacturers require the fitment of a stabiliser bracket when the user is driving a vehicle from their wheelchair.

These Magic Mobility kits provide an interface to engage the wheelchair with the aftermarket stabiliser brackets.

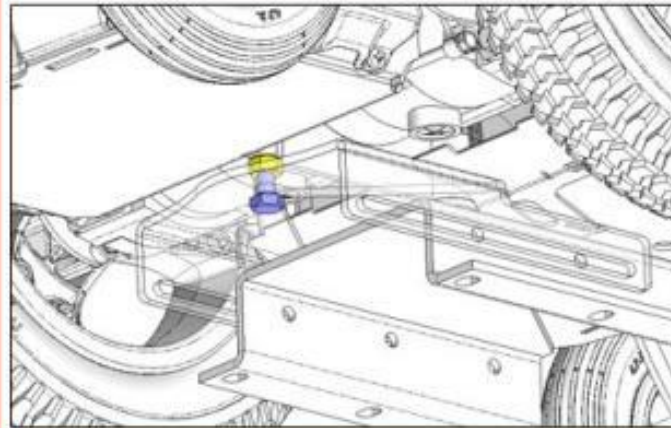
The aftermarket stabiliser brackets that attach to the dock are not provided with these kits. They must be sourced separately from the dock supplier.

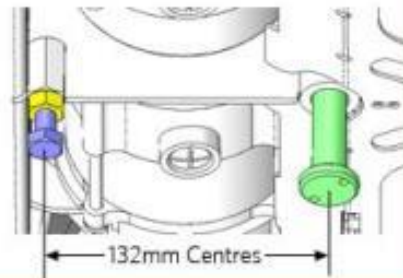
QLK-150 on Magic 360
(It's just an M10 bolt with a nut)


EZ Lock on Magic 360

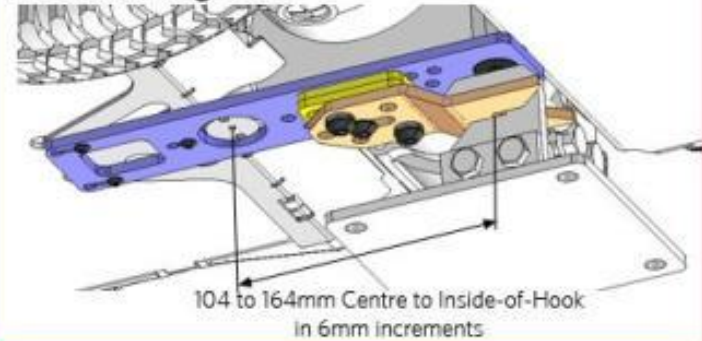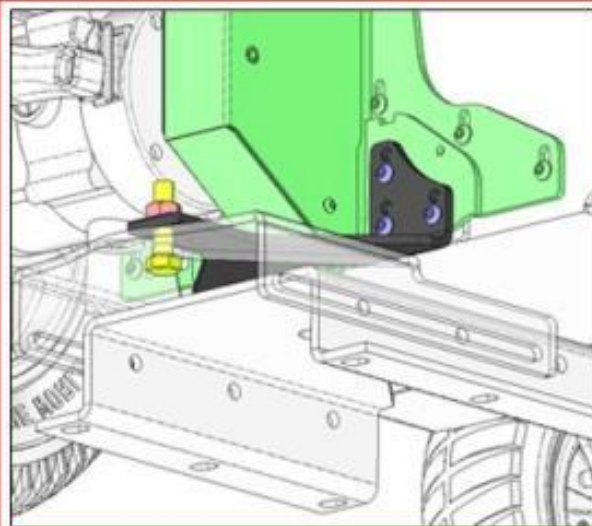104 to 164mm Centre to Inside-of-Hook
in 6mm increments


QLK-150 on V4 and V6

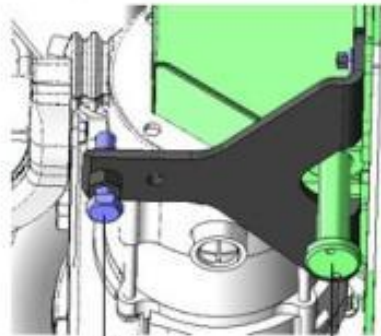88mm or 113mm Centres V4 FWD and V6
110mm or 135mm Centres V4 RWD

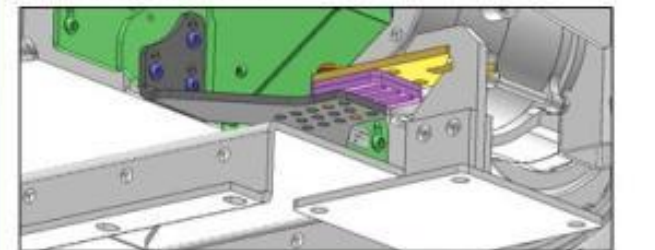EZ Lock on V4 and V6

110 to 152mm Centre to Inside-of-Hook
in 6mm increments

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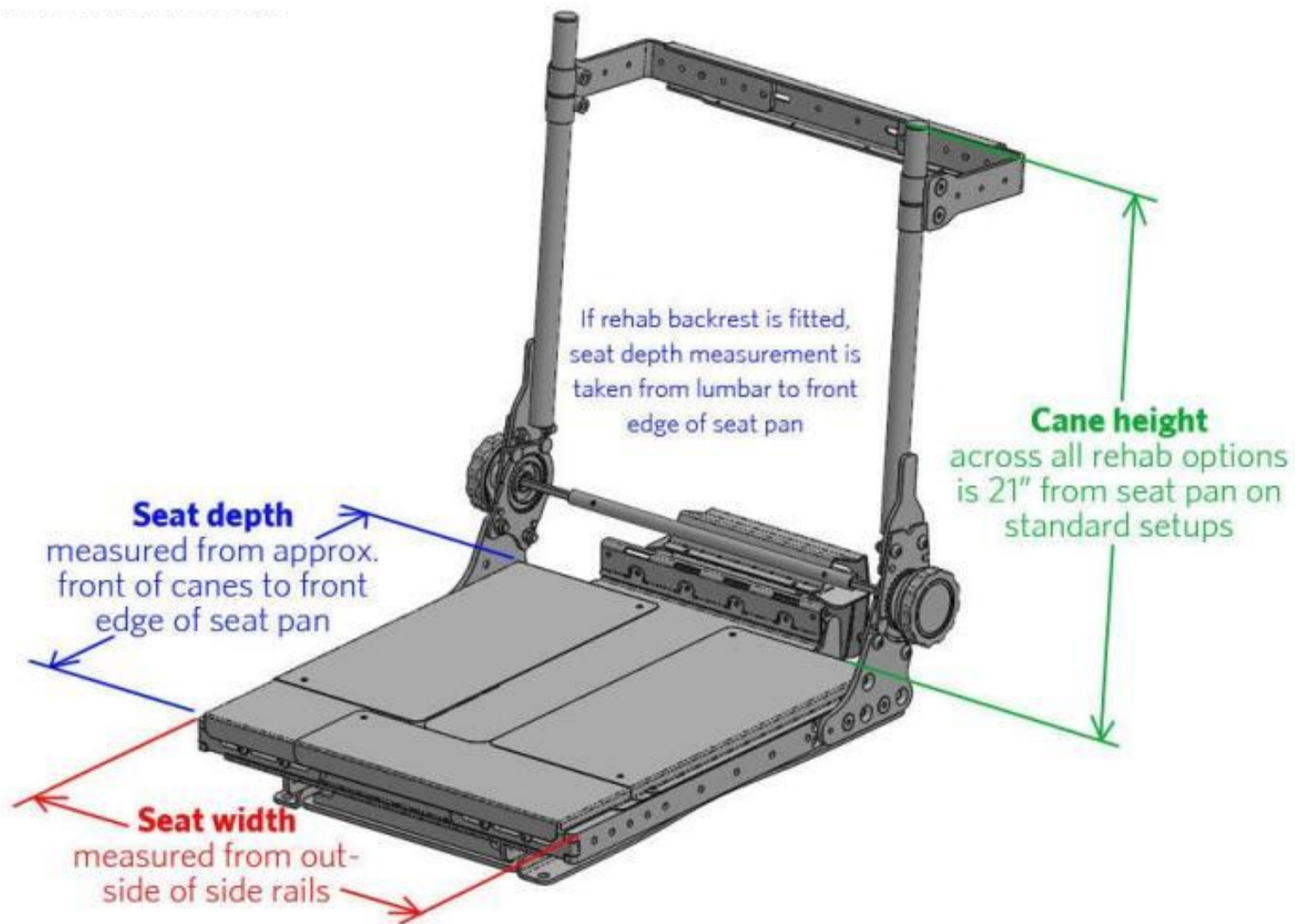
All rehab seats come with an adjustable seat pan

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© 2016 Magic Mobility. All rights reserved. This document is the property of Magic Mobility and is not to be distributed outside of Magic Mobility.

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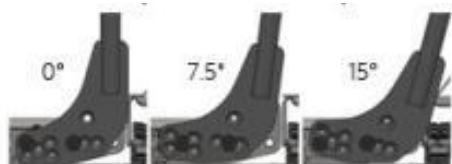
Adjust bolt pattern to change back angle

Rotate turnbuckles to change back angle between 85° and 107°

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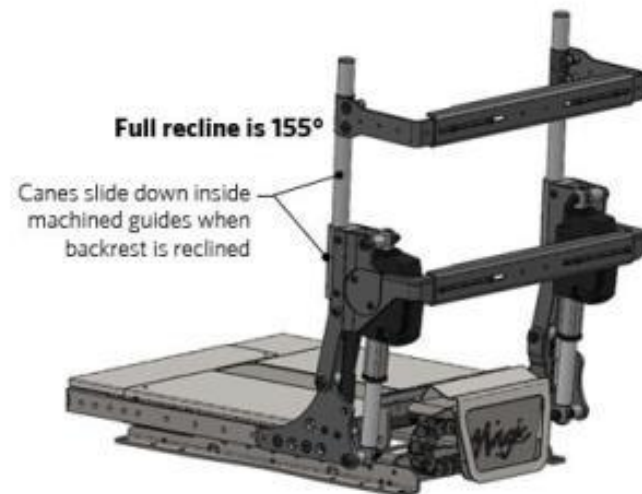
Full recline is 160°

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
Full recline is 155°


Canes slide down inside machined guides when backrest is reclined

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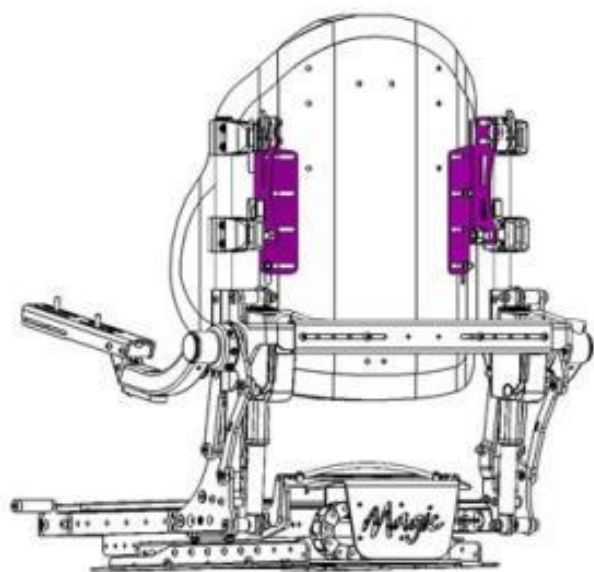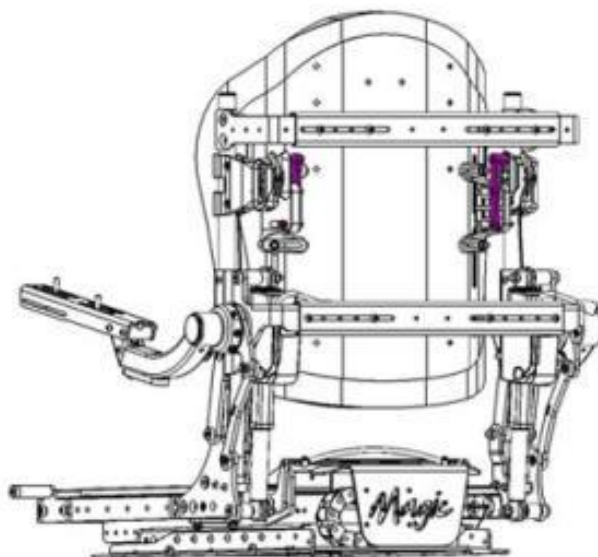
**FIT JAY BACK WITH FS CLAMPS
TO MAGIC REHAB POWER BACKS
(STANDARD AND ANTI-SHEAR)**

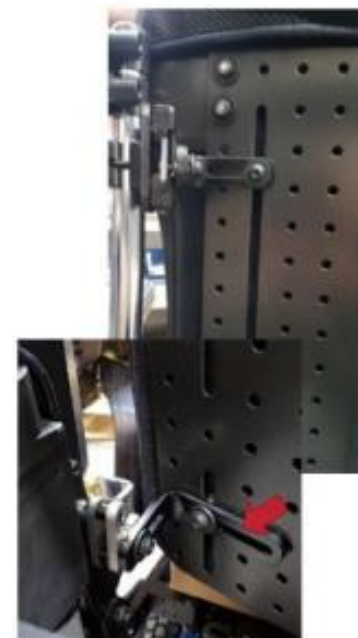**FIT JAY BACK WITH TH CLAMPS
TO MAGIC REHAB ANTISHEAR BACK**

**FIT SPEX BACK WITH QUAD MOUNT
TO MAGIC REHAB ANTISHEAR BACK**

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Chest Width 'C'	Height 'Y' (mm)
330-470mm	
Large (Xtra tall)	530
Large	480
Large (Short)	430
280-420mm	
Medium (Xtra tall)	530
Medium	480
Medium (Short)	430
260-400mm	
Junior (Xtra tall)	410
Junior	350

- Bolsters can be mounted upside down for increased armpit support
- Bolster heights can be offset up and down, up to 25mm between each side (asymmetric)

MAGIC REHAB BACKREST MADE TO ORDER
PLEASE COMPLETE ALL 6 STEPS FOR US TO ACCURATELY PROCESS YOUR ORDER

Client Name
 Quote number

mm

1. Take chest measurement, C

mm

2. Take armpit to bottom measurement, B

3. Select Backboard size

JUNIOR

For "C" = 10-15.5" / 260-400mm

Junior - std height 350mm

Junior P60H 410mm

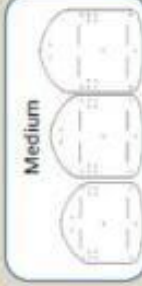
MEDIUM

For "C" = 11-16.5" / 290-420mm

Medium M50H 430mm

Medium - std height 480mm

Medium P50H 530mm

LARGE

For "C" = 13-18.5" / 330-470mm

Large M50H 430mm


Large - std height 480mm

Large P50H 530mm


4. Select bolster sizes for L and R

Junior Bolster			
Shallow 75mm	Standard 125mm	<input type="checkbox"/>	<input type="checkbox"/>
Left	Right	<input type="checkbox"/>	<input type="checkbox"/>

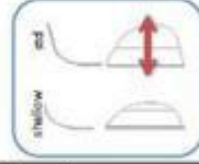Medium/Large Bolster			
Shallow 127mm	Standard 177mm	Deep 237mm	
Left	Right	<input type="checkbox"/>	<input type="checkbox"/>

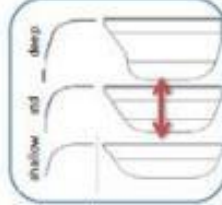5. Select lumbar support

None Shallow Std Deep

6. Select upholstery

Fabric Vinyl Dartex

Add itonal Customisation - additional pricing

Bolster up/down (to support higher trunk)

Custom bolster - please amend the drawing

Custom backboard - please amend the drawing

Note: on final set up, the angle on the bolster can also be adjusted

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Depth 'D' (mm)
Front of Backrest to
Front Edge of Bolster

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Rehab Spreader Bar

- Keeps canes parallel and can be used for pushing
- Fitted with neoprene grips when attendant control is selected

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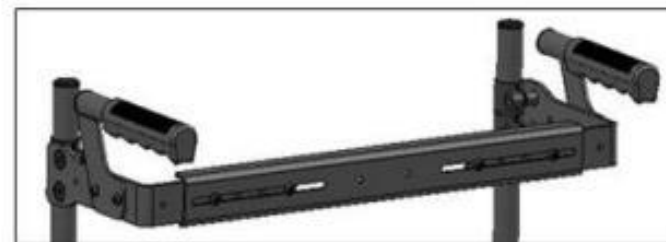
Rehab Peg Handles

- Softer grip than push rail
- Do not protrude as much as spreader bar

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Rehab Peg Handles with Spreader

- A better option when a spreader bar is fitted
- Uses less cane height

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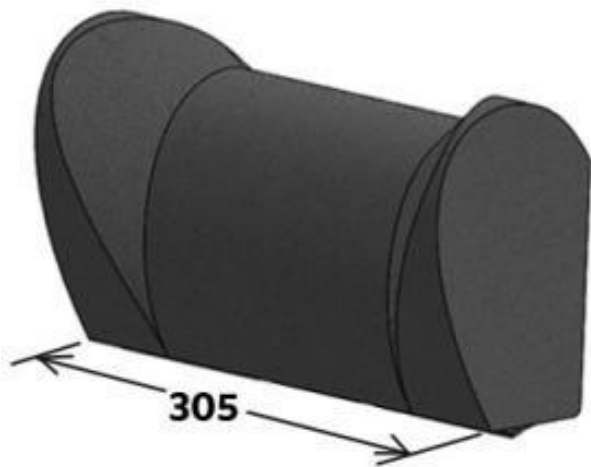
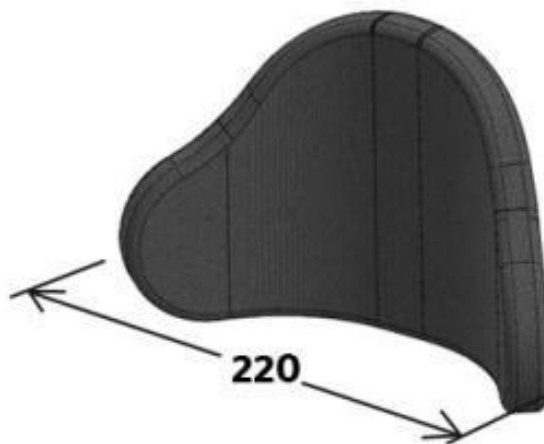
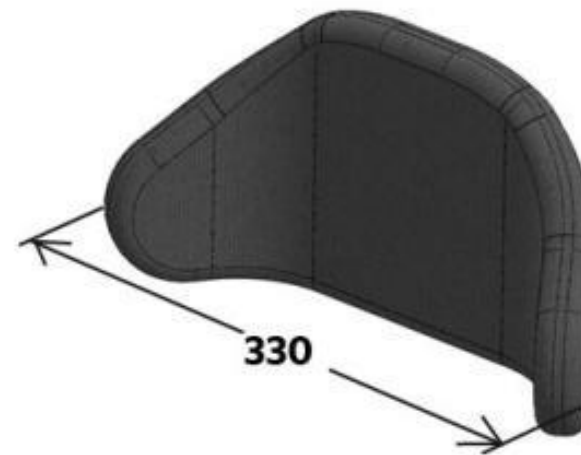
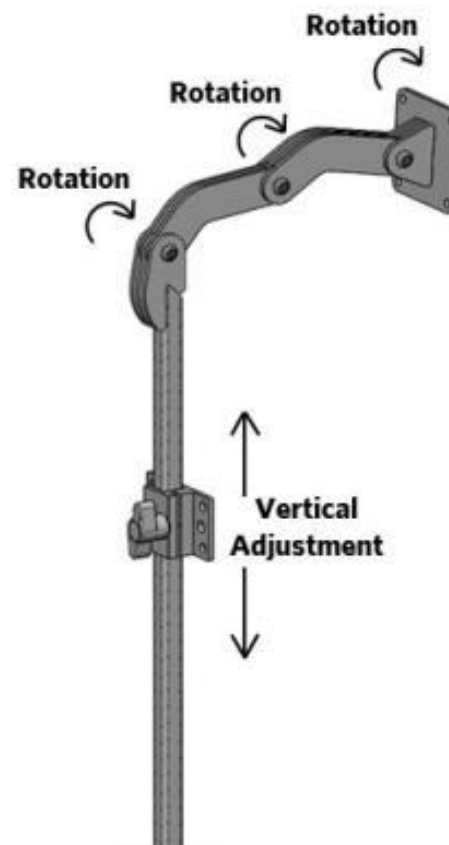
(Double Prongs - MPS Headrest
into MPS backrest only)

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(Rehab or MPS Headrest)

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Open camlock
to remove arm

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Note: 25mm of extra adjustment
is added to bottom

Flexi arms for rehab
backs shown here

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Flip-up, paralleling

Note: Depending on requested armrest height the Flexi arms
will be assembled using slightly different parts. Therefore the
range for each armrest will not be the full 220-440mm:

Requested Height	Actual Range (Flexis on MPS back)
220-330mm	220-370mm
340-370mm	250-400mm
380-440mm	330-440mm


Requested Height	Actual Range (Flexis on Rehab back)
220-330mm	180-360mm
340-440mm	275-440mm

ARMPAD ADJUSTMENTS (ALL ARMREST TYPES)


Left to Right - 7mm increments

Front to Back - 70mm

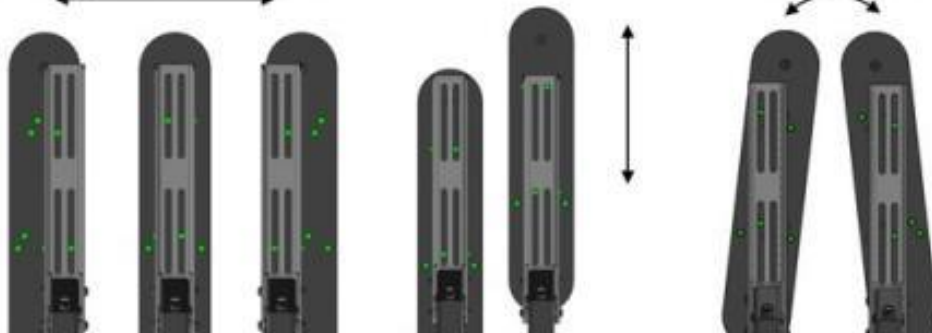Rotation +/- 12°


FLEXI ARMREST ADJUSTMENTS

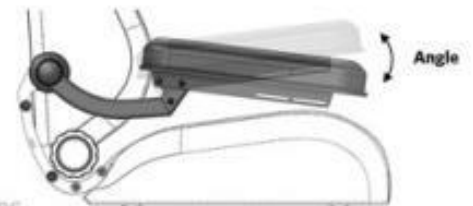
Flexi arms for MPS
backs shown here

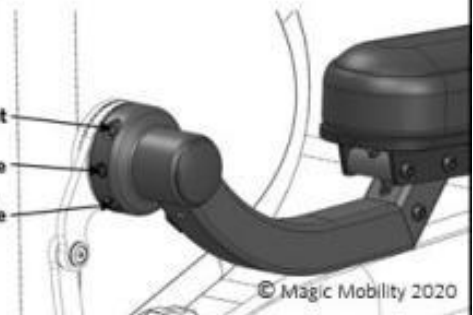Height
Rotating Resistance
Flip-up Angle

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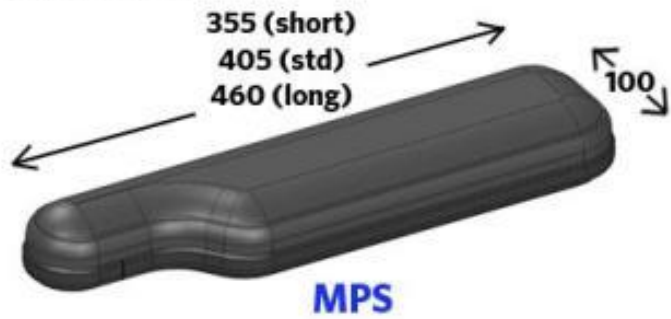
ARMPAD WIDTHS VS SEAT WIDTHS VS BASE WIDTHS - FOR FLEXI ARMS ONLY

Armpad Type	Seat Width	Rehab Backrest												MPS Backrest													
		12"	13"	14"	15"	16"	17"	18"	19"	20"	21"	22"	23"	24"	12"	13"	14"	15"	16"	17"	18"	19"	20"	21"	22"	23"	24"
50mm		450	475	500	525	550	575	600	625	650	675	700	725	750	450	475	500	525	550	575	600	625	650	675	700	725	750
75mm		500	525	550	575	600	625	650	675	700	725	750	775	800	490	515	540	565	590	615	640	665	690	715	740	765	790

Outer armpad widths shown in green boxes.

Select an armpad type and seat width, then compare the outer armpad dimension to the base width.

360 Urban	610
360 Crossover	635
360 Off-road	660

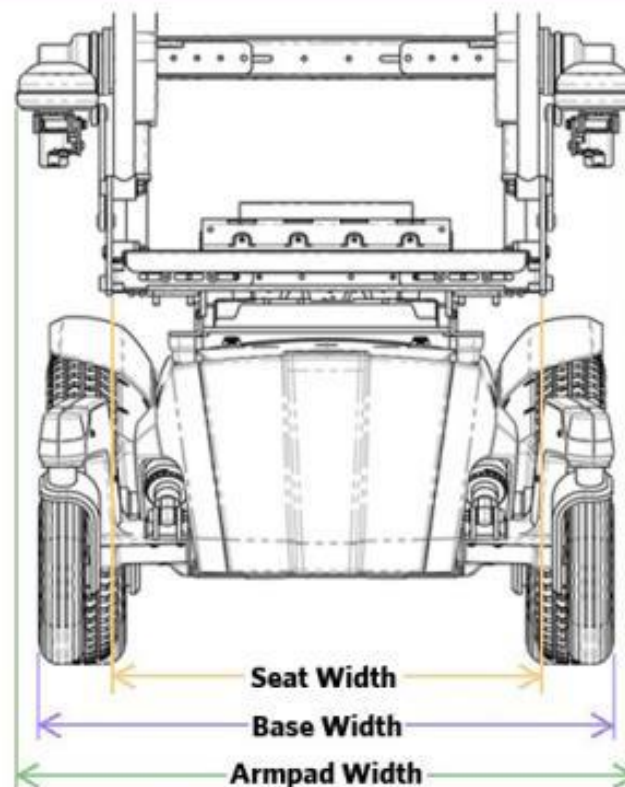Note:

- Armpads are generally fitted as far inboard as possible while still allowing flip-up function, unless requested otherwise
- All dimensions are approximate

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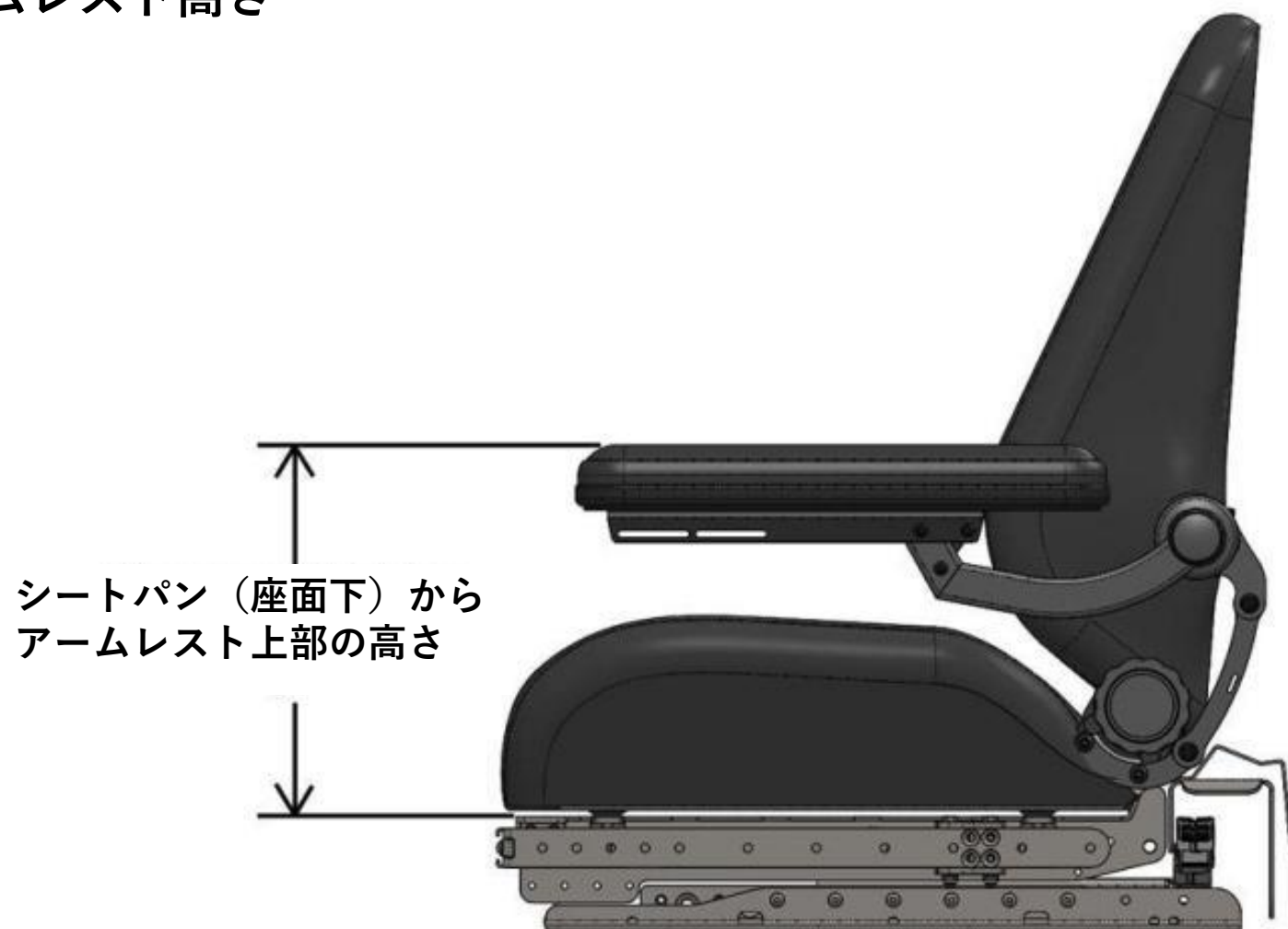
LCD COLOUR SCREEN
PLUS ONE CLICK ACTIVATOR

BASIC LED (NO SCREEN)

CJSM2 - IR + BT
COMES WITH INFRARED AND BLUETOOTH
CJSM2 - IR
COMES WITH INFRARED

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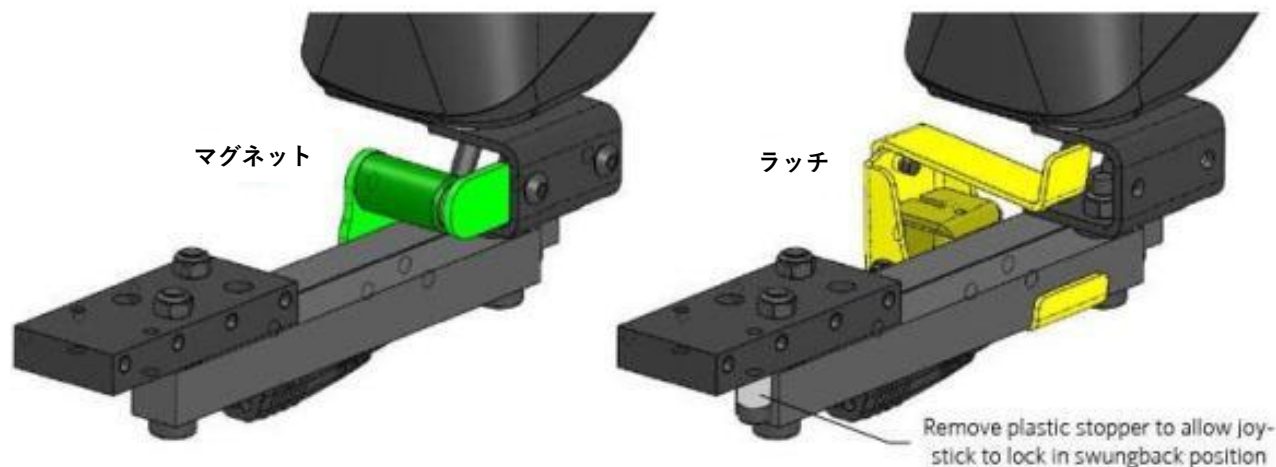
Swingback Joystick Release Options	
Magnet	Latch
<ul style="list-style-type: none"> • Requires more force but no dexterity to release • Can potentially release if wheelchair is being driven erratically or over rough terrain <ul style="list-style-type: none"> • Releases more easily if impacted 	<ul style="list-style-type: none"> • Requires minimal force but some dexterity to release • Will stay locked at all times unless latch is pushed • Less likely to release if impacted, potentially causing more damage • Can be locked in fully swung-back position (optional) <ul style="list-style-type: none"> • Custom levers can be screwed to latch to make releasing it easier for clients with less hand movement
-	
-	

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Front to back adjustment
Bracket also allows some side
to side adjustment

Note: If drinks holder or table is selected
then CTRL+5 mount will be fitted with
socket receiver.
This is not fitted on the standard setup.

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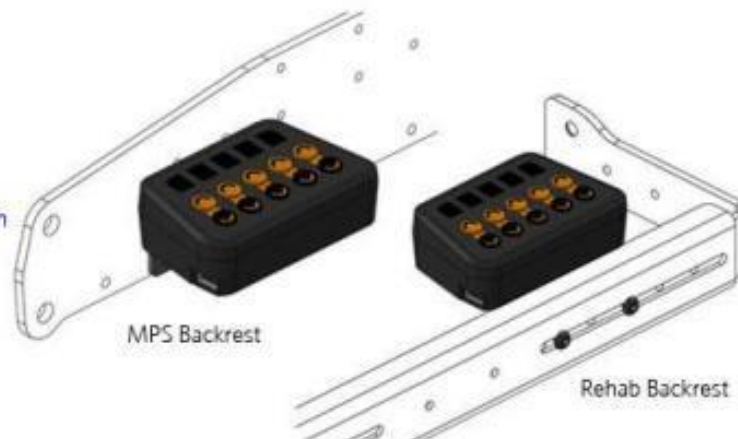
Note: The One CTRL+5 will be mounted
on the opposite side from the attendant
control (unless requested otherwise)

MPS Specific Notes:

CTRL+5 is not compatible with MPS push
rail select peg handles

CTRL+5 is not compatible with BOTH a
harness and rehab headrest. Select
alternative mounting

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OMNI 2

OMNI 2 - STANDARD

A standard Omni 2 is available with different switch options that are selectable on the order form

This Omni has the standard scanning functionality with 4 directions only

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Chair will be fitted with short hanger if foot-plate height is 320mm or less

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Note: Calf Pads must be selected if required

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This bracket can be adjusted up and down to change legrest angle

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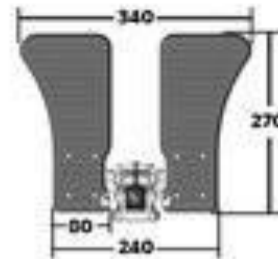
Note: Footplates are all angle adjustable on centre post, swingaway and CMPE legrests

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15


CENTRE MOUNT POWER ELEVATING WITH 2-PIECE FLIP UP

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
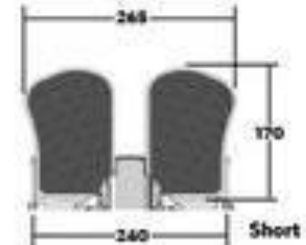
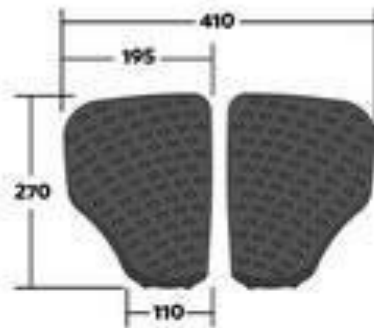
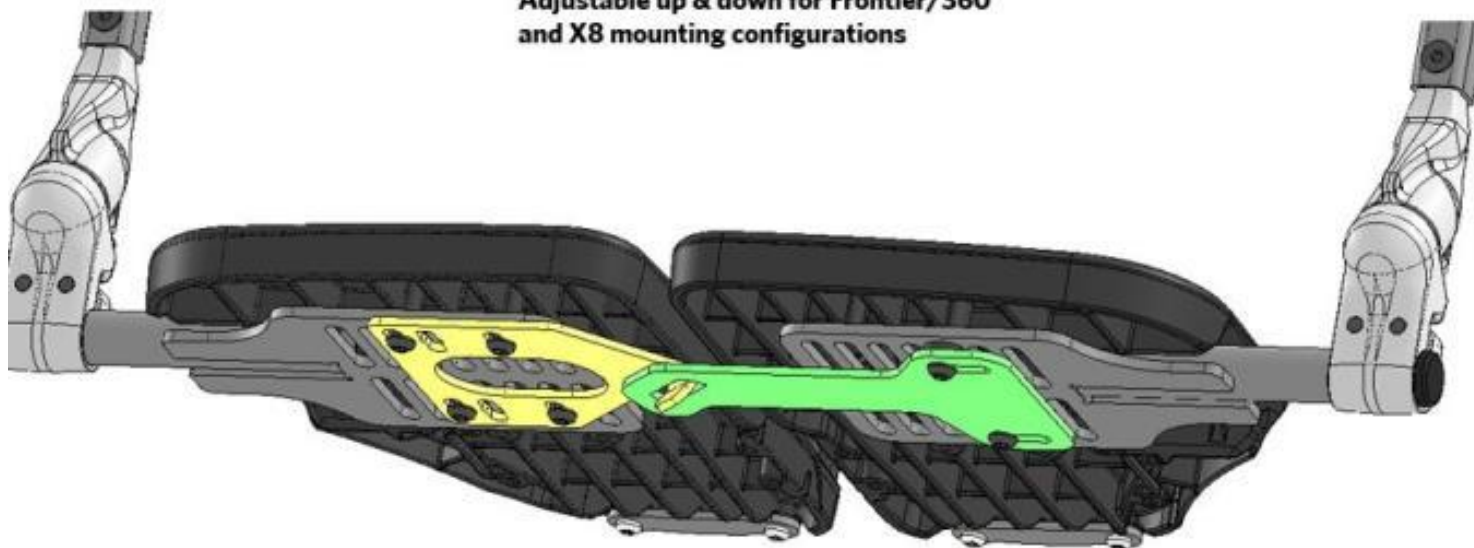
Adjustable up & down for Frontier/360
and X8 mounting configurations

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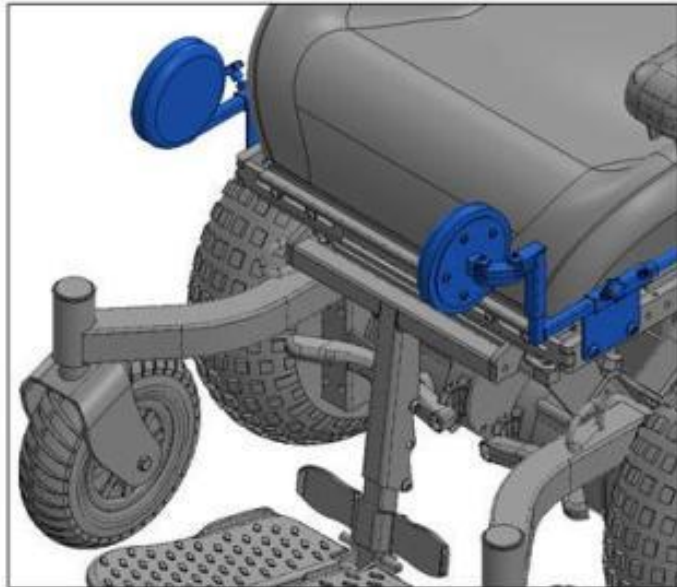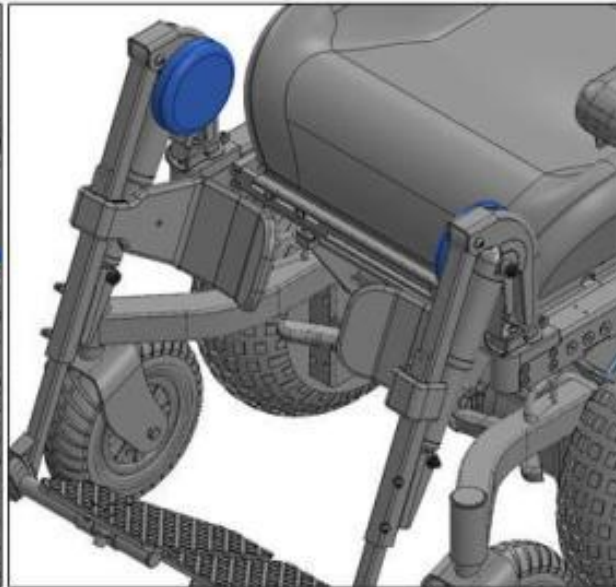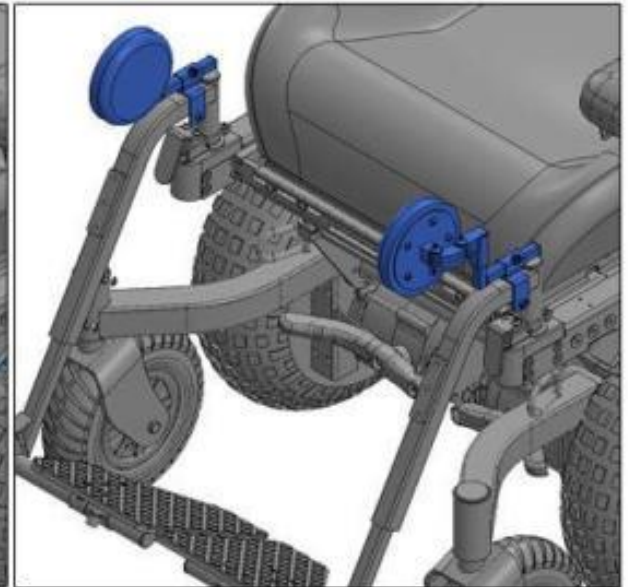
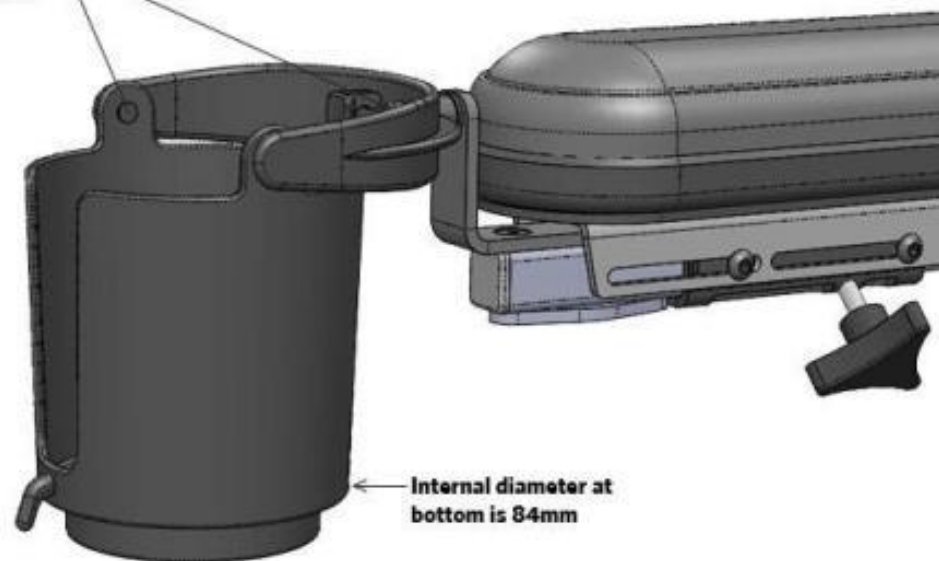
Gimbal configuration allows cup to stay flat when tilting and when driving on bumpy surfaces

Note: This assembly will be supplied loose if selected in conjunction with a table as both share the same mounting position

Internal diameter at bottom is 84mm

If swingaway legrests are fitted the cup holder mount bracket may be offset higher to avoid top of legrest (depending on armrest height)

Note: Previous cup holder can be fitted if required. See option on order form.

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