JAY[®]

JAY[®] ZIP+ CUSHION
JAY[®] LOWZONE SWITCH CUSHION
JAY[®] LOWZONE+ CUSHION
JAY[®] GELZONE CUSHION
JAY[®] CLIMAZONE CUSHION

Owner's Manual

EN

Important Consumer Information

NOTE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.

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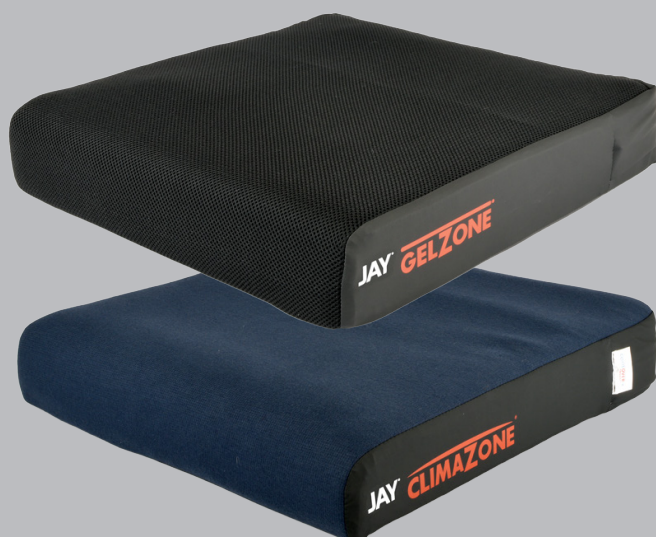
CAUTION: Check all parts are free from shipping damage. In case of damage DO NOT use. Contact HH Pressure Care or your supplier for further instructions.

Helping Hand Pressure Care recommends that a clinician such as a doctor, therapist or rehab engineer experienced in seating and positioning be consulted to determine if the cushion is appropriate. Cushion should only be installed by:

- an authorised Helping Hand Pressure Care supplier
- a qualified healthcare professional who has undergone appropriate training by either HH Pressure Care or their approved supplier.

GENERAL WARNINGS!

- DO NOT install this equipment without first reading and understanding this manual. If you are unable to understand the Warnings, Cautions and Instructions, contact a qualified clinician or supplier - otherwise injury or damage may occur.
- Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury.
- Prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.
- Please note that no cushion can completely eliminate sitting pressure.
- If you need more pressure distribution for skin protection, ask your authorised supplier for a free demonstration of any of the other JAY cushion products.
- None of the JAY cushions are a substitute for good skin care including; proper diet, cleanliness, and regular pressure relief.
- Allow the cushion to warm to room temperature before using. Sitting on extremely cold or hot surfaces can cause skin damage.
- AVOID SHARP OBJECTS or exposure to excessive heat or open flame.
- OBSTRUCTIONS: Do not place any obstructions between the user and the cushion because it will reduce product effectiveness.
- CUSHION AND COVER ORIENTATION: Product must be used with the cushion cover facing up. If the cover is not used correctly it may reduce or eliminate the cushion's benefits and could increase risk to the skin and soft tissue.

- When putting the cover back on please ensure the cover has been put back onto the foam composite in the correct orientation, and that the orientation compass is in the correct place.

INTRODUCTION:

- Intended purpose of device: JAY ZIP+, LOWZONE SWITCH, LOWZONE+, GELZONE, CLIMAZONE are designed for giving varying levels of pressure management, postural control and microclimate regulation.
- The intended lifetime of this product is 5 years.
- Maximum user weight: JAY ZIP+, LOWZONE SWITCH 114 kg, LOWZONE+, GELZONE 158 kg, CLIMAZONE 254 kg

FITTING THE CUSHION

- Select a JAY cushion that matches the width of your wheelchair.
- The cushion length should extend to within 25 – 50 mm (1" – 2") from the back of the knees.
- Adjust the footrests so your legs rest comfortably on the cushion.
- Lowering the footrests on your wheelchair or seating device may help reduce the pressure on your ischials, (seat bones) and coccyx (tailbone).

NOTE: Proper footrest adjustment may enhance sitting comfort.

Avoid using the cushion on overly stretched seat upholstery. Replace stretched seat upholstery or purchase a JAY solid seat insert for your cushion. Contact an authorised supplier for a free demonstration.

EASY MAINTENANCE AND CLEANING

- Monthly cleaning and regular maintenance may help extend the life of your cushion. Check the cover for tears and excessive wear.
- Cover: All covers can be machine washed. **Please check the labels on the individual cushion.** Remove covers from foam base before machine washing.
- Foam Base: Foam composites may be covered with a polyurethane film. Providing this film has not been damaged, clean with soft cloth moistened with water and a neutral detergent. Foam base cleaning instruction in case of removing the PU film: Wipe lightly with a damp cloth. Do not use soap and do not submerge in water. Wipe off with a clean cloth and let it completely air-dry before reassembly.


STORAGE

- DO NOT leave the cushion outside - extended exposure to the elements could compromise the effectiveness of the cushion.
- Store in a dry (moisture free) and clean environment when not in use.

WARRANTY

All JAY cushions are guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided they are used correctly for the purpose they were intended. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover. The removable outer covers are guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use.

Medical device symbol glossary			
This symbol means Medical Device	UKCA Mark	CE Mark	Consult owner's manual
Manufacturer's address	European Authorised Representative	Swiss Representative's address	Importer's address
Batch code	Catalogue number	Unique device identifier	Serial number
DANGER! Warning messages regarding possible risks of serious accident or death	WARNING! Warning messages regarding possible risks of injury	CAUTION! Warning messages regarding possible technical damage	Do not use if package is damaged and consult the operator's manual
Date of manufacture XXXX-XX-XX	Load Maximum Please refer to individual cushion XXX Kg	Distributor's address	Keep dry

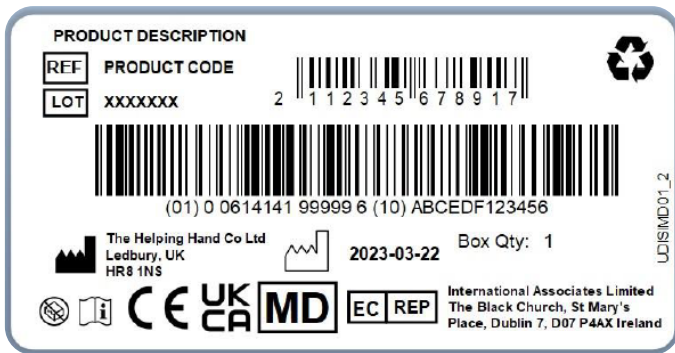← SAMPLE LABEL

CE As the manufacturer, Helping Hand, declares that this product conforms to the Medical Device Regulation (2017/745).

UKCA As the manufacturer, Helping Hand, declares that the product conforms to the UK Medical Devices Regulation 2002 No.618.

We at Helping Hand have been awarded the ISO-13485 certificate, which affirms the quality of our products at every stage, from R&D to production. This product meets the requirements in accordance with EU and UK regulations.

The varieties of cushions fitting variants, as well as the different positioning components to deliver higher positioning and skin protection benefits according to each user requirement and indication, mean that it can be used by a range of users with different positioning and skin protection needs and indications as follows:

- Elderly
- Less Mobile (in the home)
- Wheelchair users

The cushion shall not be used in case of:

- if washed or tumble dried above 60 degrees
- User weight higher than allowed (Maximum User Load stated on individual cushion).

NOTE: General user advice. Not following these instructions may result in physical injury, damage to the product or damage to the environment!

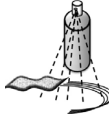
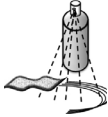
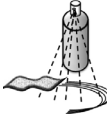
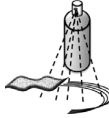
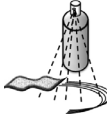
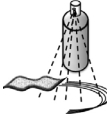
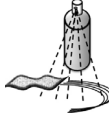
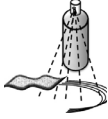
Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority the Member State in which the user and/or patient is established.

Helping Hand Pressure Care strongly recommends that in order to ensure that your cushion operates and performs as intended, all information supplied within this operating manual is read, understood and stored safely for future reference.

Medical Device Combinations

It may be possible to combine this Medical Device with one or more Medical Devices, or other products. All combinations have been validated to meet the General Safety and Performance Requirements, Annex 1 No. 14.1 of the Medical Device Regulation 2017/745.

Approved Cleaning Procedures for JAY Helping Hand Pressure Care Cushions

Product	JAY ZIP+	LOWZONE SWITCH	LOWZONE+	LOWZONE GEL SWITCH	GELZONE	CLIMAZONE
1. Cushion OUTER COVER - CoolOver / Spacer fabric / Wipedown PU* / Wipedown PU non-slip* / Toweling						
Wipe clean with a soft cloth moistened with water & a neutral detergent.						
Machine wash with detergent up to 60°C or dry clean. Chlorine based cleaning solutions (i.e. bleach) can be diluted to 1%. DO NOT use biological cleaner						
HANG COVERS TO AIR DRY BEFORE RE-USE DO NOT TUMBLE DRY DO NOT IRON						
2. Cushion INNER COVER - polyurethane film						
Gently clean with a soft cloth moistened with water & a neutral detergent						
3. Cushion CORE BASE - DO NOT SUBMERGE IN WATER						
Steam autoclave up to 105 °C						
Gently clean with a soft cloth moistened with water & a neutral detergent						
Immersion isinfection						
60°C is the maximum recommended temperature for ALL covers. * Possible wrinkling may occur with PU if washed at 60°C but the functional aspect of the cover is not diminished						

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Sunrise Medical HCM B.V.
Sunrise Medical S.A.S
Certification Ltd